

Spare Parts List

MARCH 2022



Mobile Lifts
Slings
Spare Parts Accessories
Service Tools



Changes made from previous version of the Spare Parts List:

Prod No.	Name		Page	Proj No.
2006110	Battery Lithium Ion	Assembly Instruction 3EN690296 added	26, 75, 91	CAPA03789
2006109	Battery Lithium Ion	Assembly Instruction 3EN690296 added	107	CAPA03789
20090029	Cable for charging JP	Spare part only for the Japanese market	44	SUS003812
21090000	Extension Cable JP	Spare part only for the Japanese market	45	SUS003812
20290024JP	Control Box Sa II, JP	New spare part only for the Japanese market.	45	SUS003812
20490013JP	Control Box Viking M, JP	New spare part only for the Japanese market.	95	SUS003813
20090029	Cable for charging JP	Spare part only for the Japanese market	95	SUS003813
21090000	Extension Cable JP	Spare part only for the Japanese market	95	SUS003813

NOTE

Make sure the spare part is mounted according to the assembly instruction. Check all functions. Repairs and maintenance may only be performed by personnel authorized by Hill-Rom and using original Liko™ spare parts. For information regarding Service and Maintenance education, contact your Hill-Rom representative. For contact information please visit our website: www.hillrom.com.

Product changes

Products are constantly being updated and refined. Hill-Rom reserves the right to change aspects of the products without prior notice. Contact your local Hill-Rom representative for updated information and advice.

Pictures

Pictures may not show all components. The Spare Part Assembly Instruction contains information regarding components included in each spare part.

Accessories

Accessories can be found in the Product Lists and Instruction Guides.

Spare part statement

Liko AB will as manufacturer of Overhead & Mobile Lifts keep spare parts available for 10 years after the last manufacturing date of the product. Spare parts are available in the spare parts list or in the Service Manual for each model of lift. *Components (manufacturing parts) are not available after last manufacturing date.*

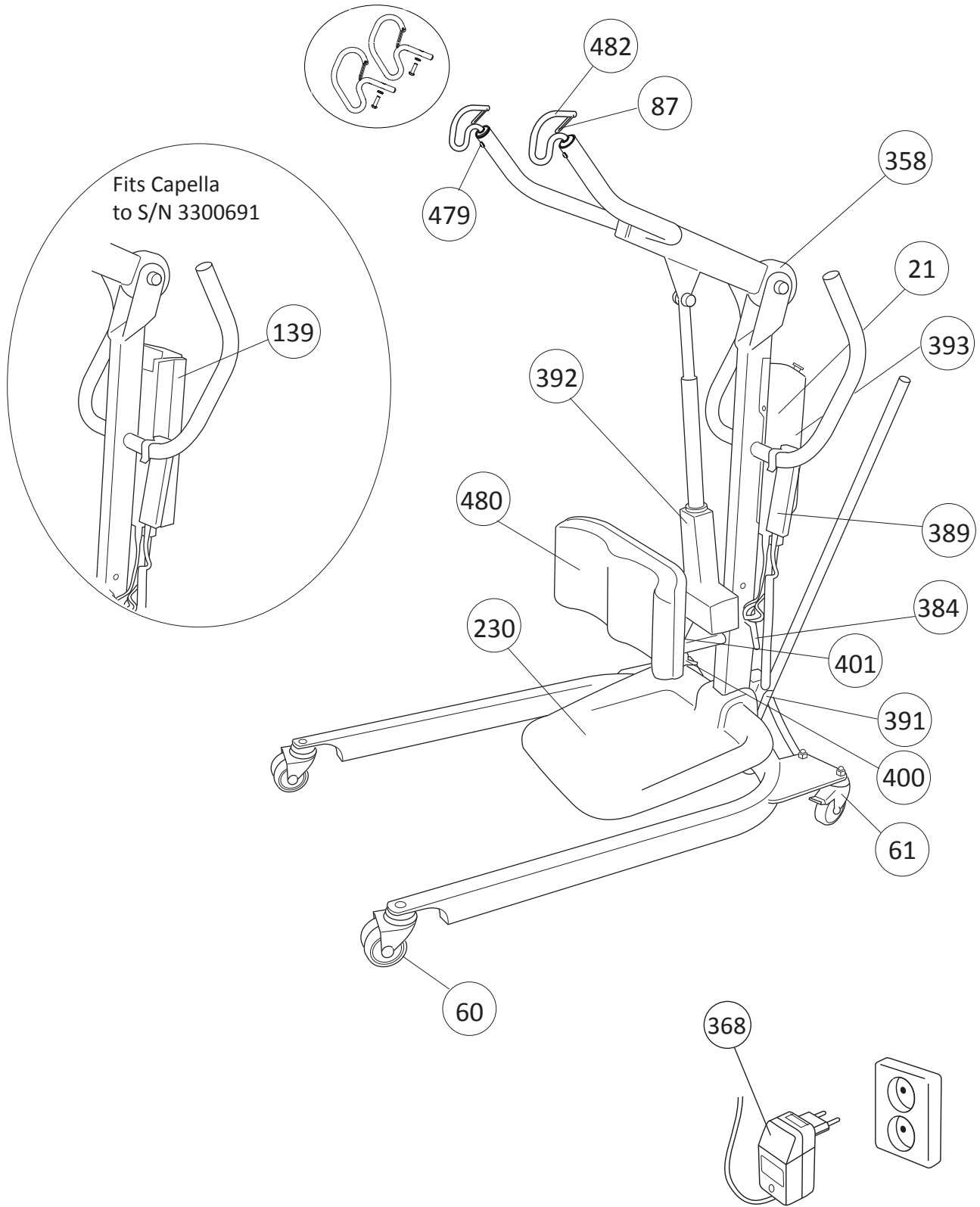
Table of content

Mobile Lifts

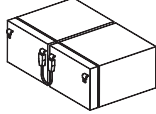
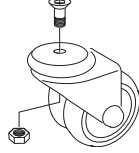
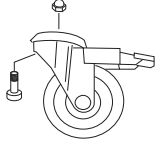
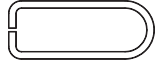
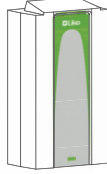
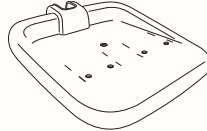

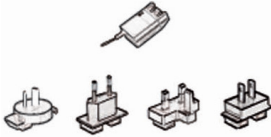


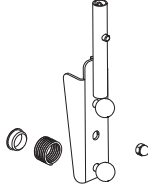
Capella™ 201	4
Caroli™ 101	7
Golvo™ 7000 / 7007 Spare Parts List.....	10
Golvo™ 8000 / 8008 Spare Parts List.....	17
Golvo™ 9000 Spare Parts List	24
Golvo™ Spare Parts Matrix	30
Liko M220™, Liko M230™	33
LikoLight™	37
RollOn™	41
Sabina™ II EE/EM	43
Sabina™ 200	48
Uno™ 100 EM/EE, 102 EM/EE, ES (up to S/N 51 999).....	51
Uno™ 102 EE from Rev 01 (S/N 52 000)	55
Uno™ 200	59
Viking™ XL Prod No. 2040003	62
Viking™ XL Prod No. 2040013	66
Viking™ XL Prod No. 2040033	70
Viking™ XL Prod No. 2040043	74
Viking™ L Prod No. 2040004	78
Viking™ L Prod No. 2040014	82
Viking™ L Prod No. 2040034	86
Viking™ L Prod No. 2040044	90
Viking™ M Prod No. 2040005	94
Viking™ M Prod No. 2040015	98
Viking™ M Prod No. 2040035	102
Viking™ M Prod No. 2040045	106
Viking™ S	110
 Viking™ XS	 114
 Slings	
Spare Parts, Slings.....	118
 Spare Parts Accessories	
Accessories (in alphabetical order).....	119
 Service Tools	
Service Tools (in alphabetical order).....	123

Spare Parts Positions Capella™ 201 mobile lift

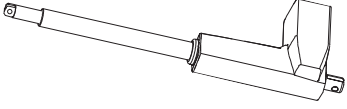
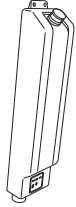



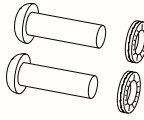
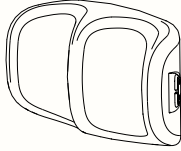
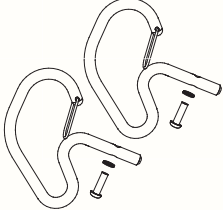
PROD. NO.	PRODUCT
5021001	Capella 201 (Discontinued 2009)



Capella™ 201 mobile lift

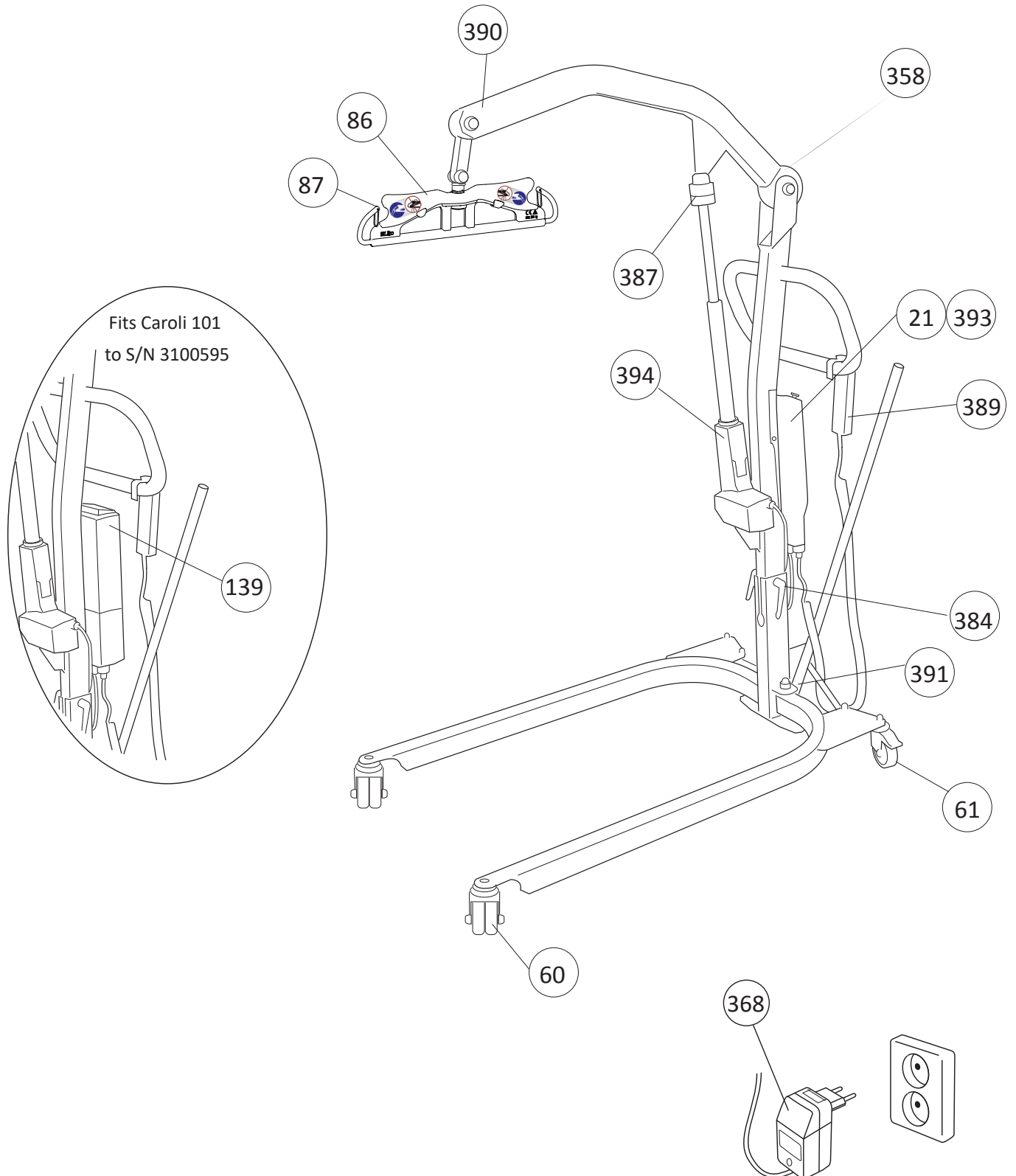
PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20090025	21	Battery 2.9 Ah/12V	pair		3SE380201
20190042	60	Front Wheel 75 mm	pce		3SE640283
20190043	61	Rear Wheel 75 mm	pce		3SE640282
31190018	87	Latch	pair		3EN640271
2006106	139	Battery Lead Acid	pce		3EN630224
20290026	230	Foot Tray cover , Sa II	pce		3SE630245
20190062	358	Lift arm, plastic cover	set		3SE650225
21090025	368	Charger CH01	set		3EN650232
2107101	384	Locking handles	pair		3SE650248
5019008	389	Hand control 2-Button	pce		3EN590401 Service Manual
5019014	391	Restoration set Manual Base	set		3EN590401 Service Manual

Capella™ 201 mobile lift

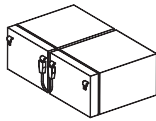
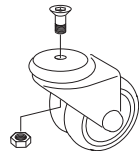
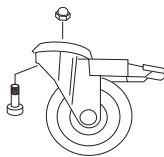

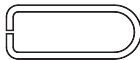
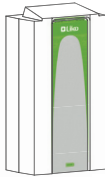

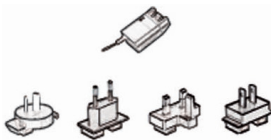



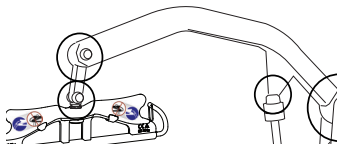
PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
5019021	392	Actuator Capella	pce		3EN590401 Service Manual
5019024	393	Control Unit <i>from S/N 3300692</i>	pce		3EN590401 Service Manual
5019026	395	Lacquer top coat for improve. (50 ml)	pce		3EN590401 Service Manual
20290033	400	Star wheel M8	pce		3SE650265
20290034	401	Locking rail & Star wheel M6	set		3SE650251
20290042	479	Hook assembly	set		3SE690202
20290043	480	Leg Support II	pce		3SE690205
20290046	482	Sabina Hook Retrofit kit II	set		3SE680204

Spare Parts Positions Caroli™ mobile lift

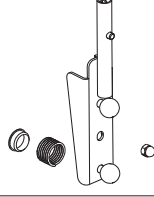
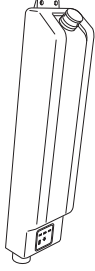
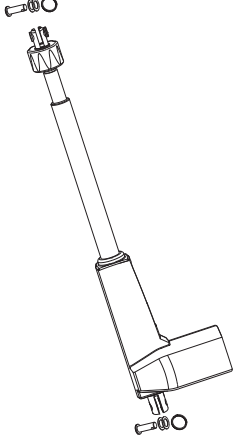
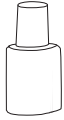
PROD. NO.	PRODUCT
5011001	Caroli 101 (Discontinued 2009)



Caroli™ 101 mobile lift

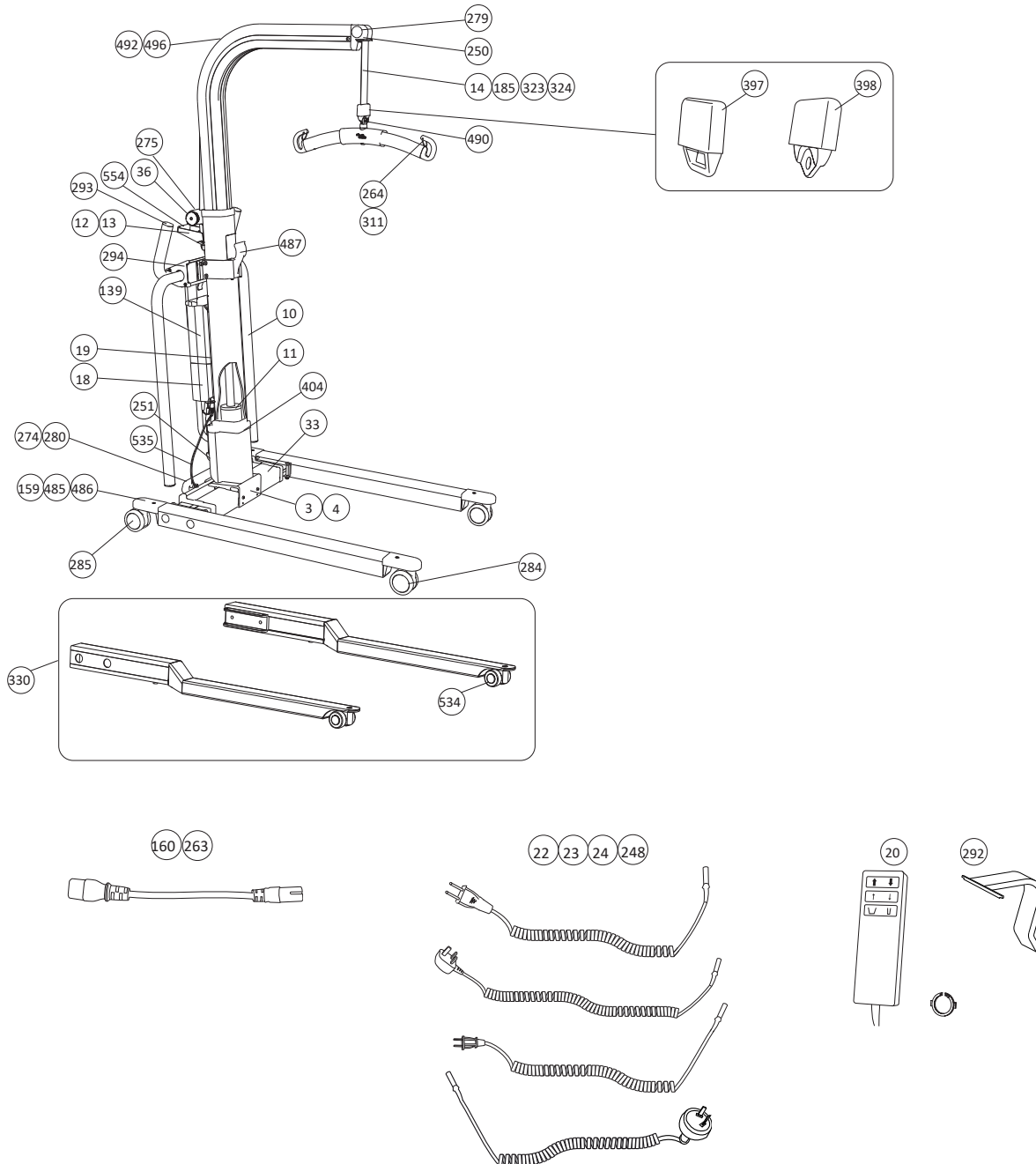
PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20090025	21	Battery 2.9 Ah/12V	pair		3SE380201
20190042	60	Front Wheel 75 mm	pce		3SE640283
20190043	61	Rear Wheel 75 mm	pce		3SE640282
3156075	86	Universal SlingBar 450	pce		7EN160185
31190018	87	Latch	pair		3EN640271
2006106	139	Battery Lead Acid	pce		3EN630224
20190062	358	Lift arm, plastic cover	set		3SE650225
21090025	368	Charger CH01	set		3EN650232
2107101	384	Locking handles	pair		3SE650248
5019004	387	Emergency Lowering Device Caroli	pce		3EN580401 Service Manual
5019008	389	Hand control 2-Button	pce		3EN580401 Service Manual
5019010	390	Overhead Beam Restoration set	pce		3EN580401 Service Manual

Caroli™ 101 mobile lift

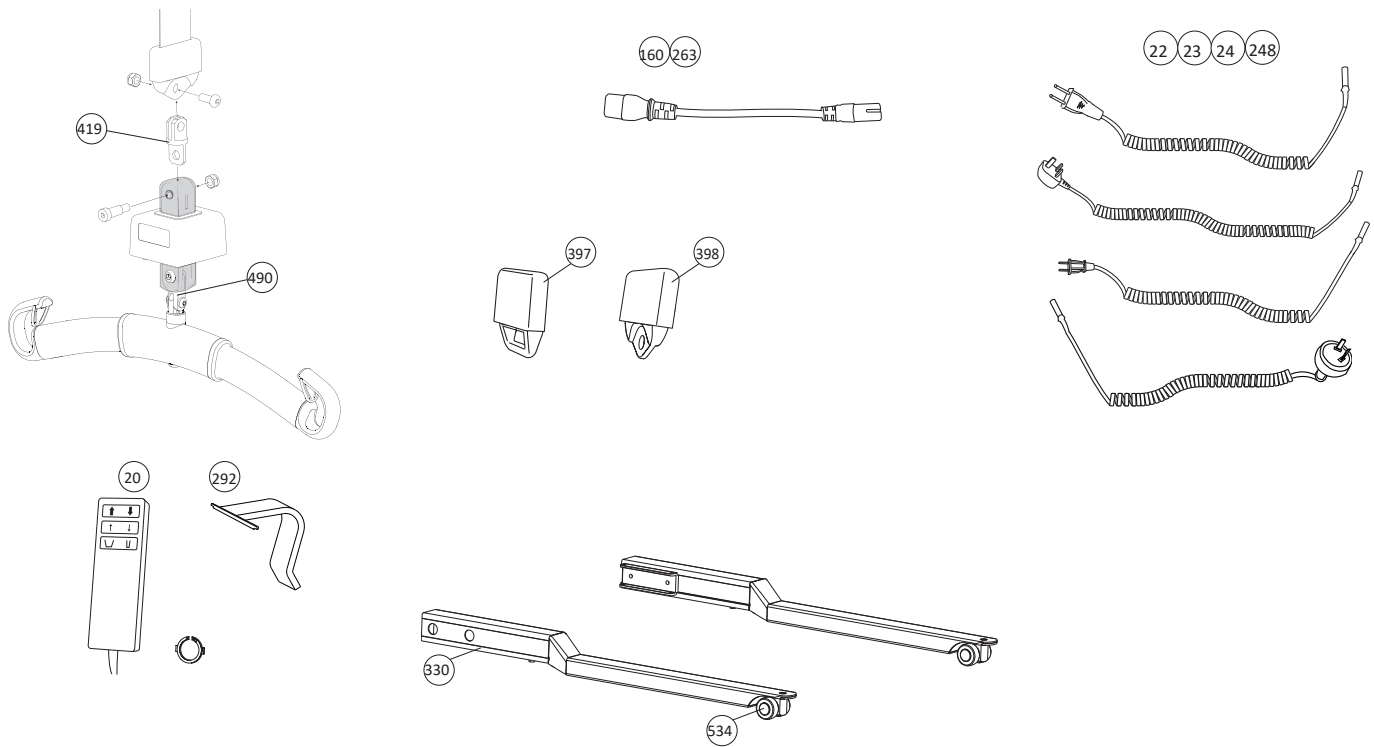
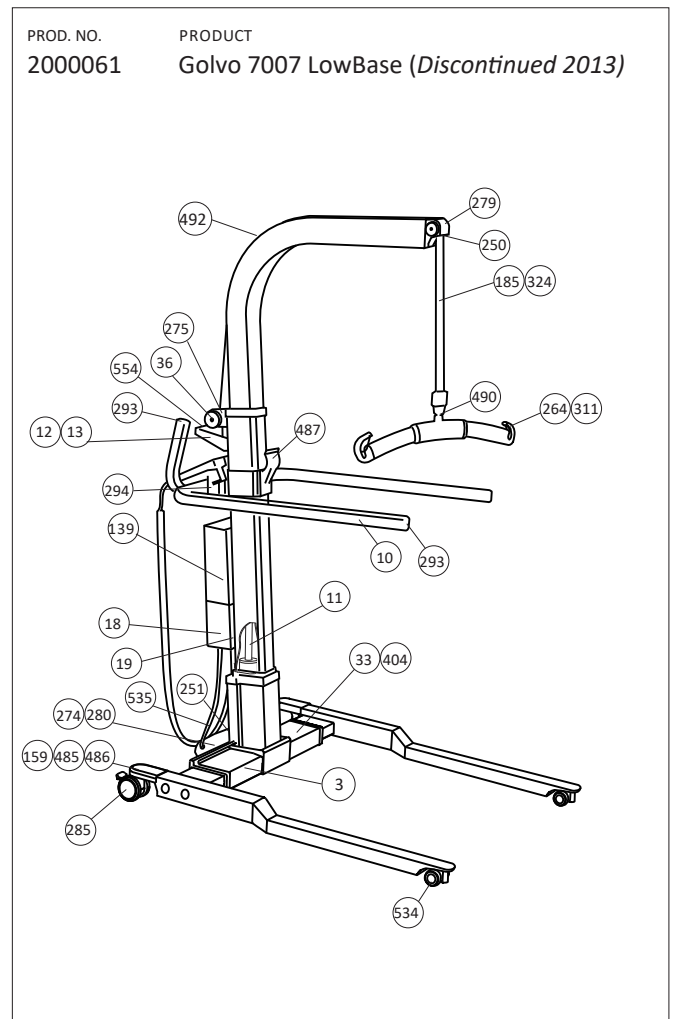
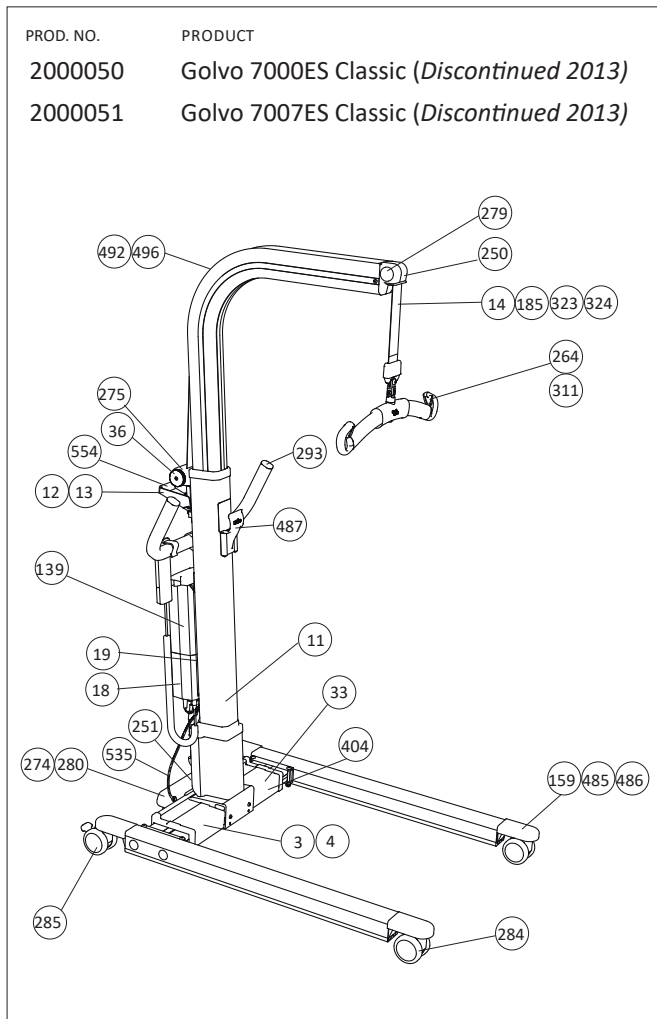
PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
5019014	391	Restoration set Manual Base	set		3EN580401 Service Manual
5019024	393	Control Unit <i>from S/N 3100596</i>	pce		3EN580401 Service Manual
5019025	394	Actuator Caroli MKII	set		3EN580401 Service Manual
5019026	395	Lacquer top coat for improve. (50 ml)	pce		3EN580401 Service Manual

Spare Parts Positions Golvo™ 7000/7007 mobile lift

2000009	Golvo 7000 ES,"facelift" from S/N 7024206 (production start 2005. Discontinued 2012)
2000010	Golvo 7007 ES "facelift" from S/N 7729722 (production start 2005. Discontinued 2012)

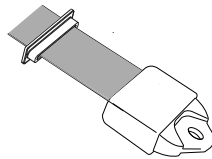
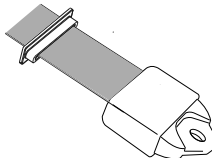
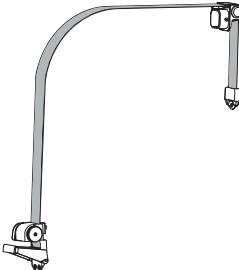
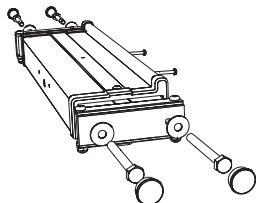
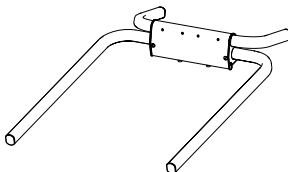
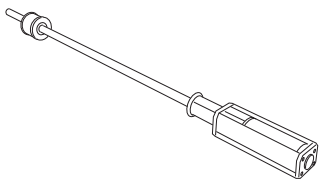
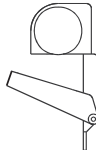
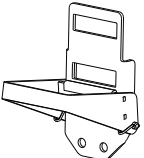


Spare Parts Positions Golvo™ 7000/7007 mobile lift



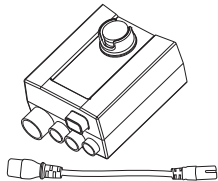
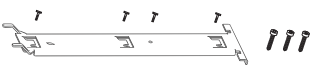
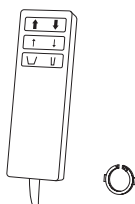
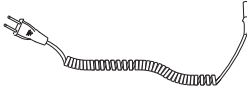
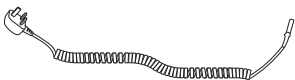
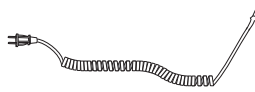
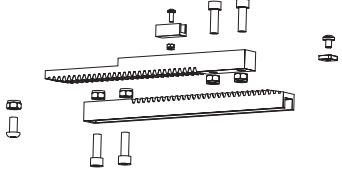
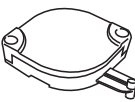
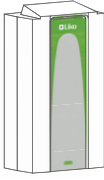
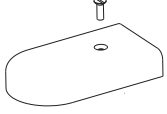

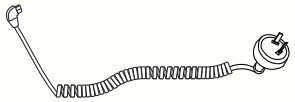
Golvo™ 7000/7007 mobile lift

(G7=Golvo 7000, G77=Golvo 7007, G8=Golvo 8000, G88=Golvo 8008, G9=Golvo 9000)

PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
Golvo LiftStrap					
20090018	14	Lift Strap G7 / G8 <i>Grey strap and grey plastic details</i>	pce		3EN650272
20090048	185	Lift Strap G77 (from S/N: 7723630) / G88 / G9 <i>Grey strap and grey plastic details</i>	pce		3EN650272
20090107	323	Easy Strapper G7 / G8 <i>Grey strap and grey plastic details</i>	set		3EN640248
20090106	324	Easy Strapper G77 from S/N 7723630 / G88 / G9 <i>Grey strap, grey plastic details and handle.</i>	set		3EN640248
Golvo Spare Parts					
20090094	3	Middle Beam 7000/8000 <i>(motor <u>not</u> included)</i>	pce		3EN690271
20090095	4	Middle Beam 7007/8008/9000 <i>(motor <u>not</u> included)</i>	pce		3EN690272
20090103	10	Golvo Armrest	set		3SE630280
20090014	11	Actuator G7 / 77 <i>Fits: From S/N:</i> <i>Golvo 7000 7024206</i> <i>Golvo 7007 7729722</i>	pce		3SE630249
20090104	12	Emergency Lowering Golvo <i>Grey plate</i>	pce		3EN640253
20090105	13	Body/Handle set for Emergency Lowering Device <i>Grey plate</i>	pce		3SE630285


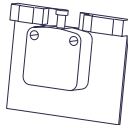
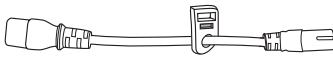
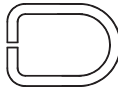
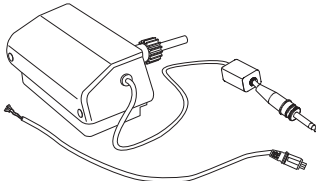
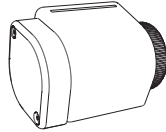
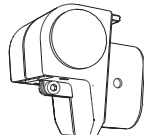
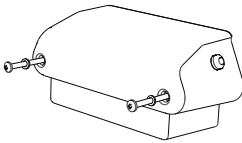


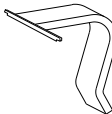

Golvo™ 7000/7007 mobile lift

(G7=Golvo 7000, G77=Golvo 7007, G8=Golvo 8000, G88=Golvo 8008, G9=Golvo 9000)

PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20090022	18	Control Box G7 / 77 ES	pce		3SE630223
20090022US	18	Control Box G7 / 77 ES, US <i>only for the US market</i>	pce		
20090023	19	Attachment for Control Box	pce		3EN630238
20090089	20	Hand control 6-Button	pce		3SE690232
20090026	22	Cable for charging EU	pce		3EN630237
20090027	23	Cable for charging UK	pce		
20090028	24	Cable for charging US	pce		
20090096	33	Gear Rack set	set		3EN690273
20090041	36	Spring Magazine	pce		3EN630214
2006106	139	Battery Lead Acid	pce		3EN630224
20390006	159	Fork Cover	pair		183916
21090001	160	Extension Cable	pce		3EN630225
20090050	248	Cable for Charging AU/NZ	pce		3EN630237

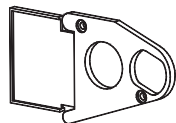

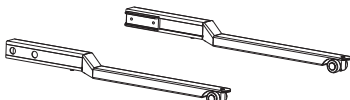

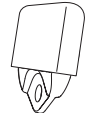
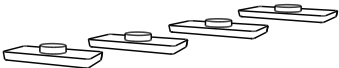
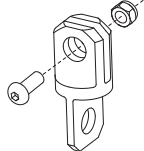
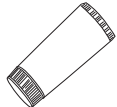
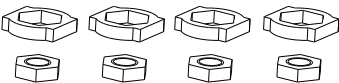
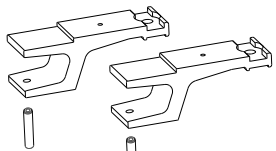
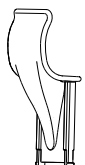
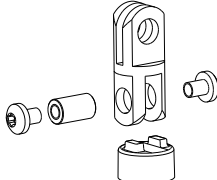
Golvo™ 7000/7007 mobile lift

(G7=Golvo 7000, G77=Golvo 7007, G8=Golvo 8000, G88=Golvo 8008, G9=Golvo 9000)

PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20090051	250	Reinforcement Plate Golvo <i>from S/N: 7022914 / 7725944</i>	pce		3EN630215
20090052	251	Limit Switch G7 / 77	pce		3SE640240
21090011	263	Extension Cable US	pce		3EN630218
31590011	264	Latch Universal SlingBar	pair		3SE630229
20090059	274	Motor Base Width Adjustment Golvo, grey <i>replaces 20090005 and 20090046</i>			3SE630216
20090060	275	Plastic Housing grey <i>from S/N 7024206 / 7729722</i>	pce		3EN630267
20090064	279	Roll Frame Attachment Golvo, grey <i>from S/N 7024206 / 7729722</i>	pce		3EN630269
20090065	280	Base Motor Cover grey <i>from S/N 7024206 / 7729722</i>	set		3EN630270
20490014	284	Front Wheel 75 mm Standard <i>from S/N 7024206 / 7729722</i>	pce		182904
20490015	285	Rear Wheel 75 mm Standard <i>from S/N 7024206 / 7729722</i>	pce		182903
20090058	292	Hook for Hand control 2,4,6	pce		3SE630210
20090066	293	End Cover Handle/Arm Support grey, 4 pce	set		3SE630271

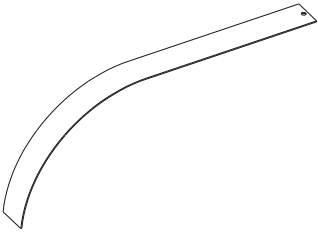
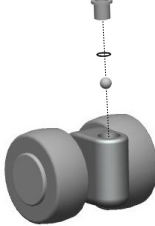
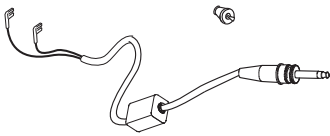
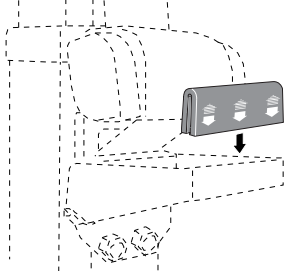
Golvo™ 7000/7007 mobile lift

(G7=Golvo 7000, G77=Golvo 7007, G8=Golvo 8000, G88=Golvo 8008, G9=Golvo 9000)

PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20090067	294	End Cover Armrest, grey	set		3SE630272
31590014	311	Latch, composite <i>Fits Universal SlingBar from S/N 1299511</i>	pair		3SE640242
20090071	330	Low base kit Golvo 7007/8008	set		3EN640262
31590005	397	Q-link	pce		3EN640236
31590025	398	Lift strap joint	pce		3EN640237
20090072	404	Friction plates 2.8-3.4 mm <i>20 pce of each model (2.8, 3.0, 3.2, 3.4 mm)</i>	set		3EN650254
2006010	419	Adapter Golvo (for Scale)	set		3SE650244
21090028	425	Grease SHC460 30g	pce		-
20090080	485	Fork locking attachment, 4 pair	set		3SE690208
20090081	486	Wheel Fork	pair		3SE690209
20090082	487	Parking bracket	pce		3SE690204
20490052	490	SlingBar Assembly set	set		3SE690210

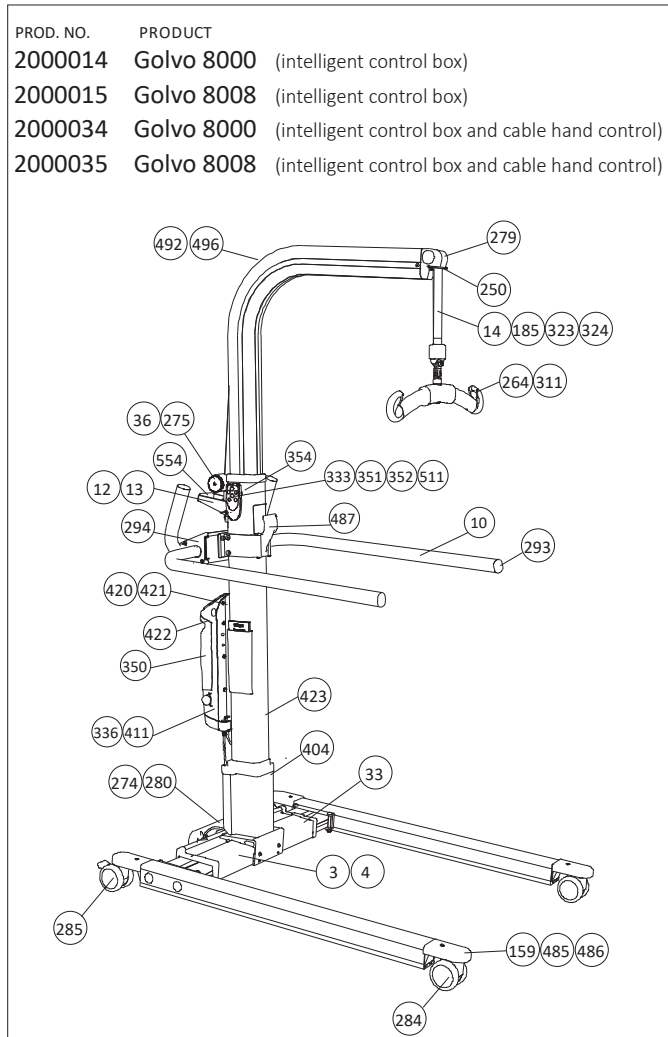
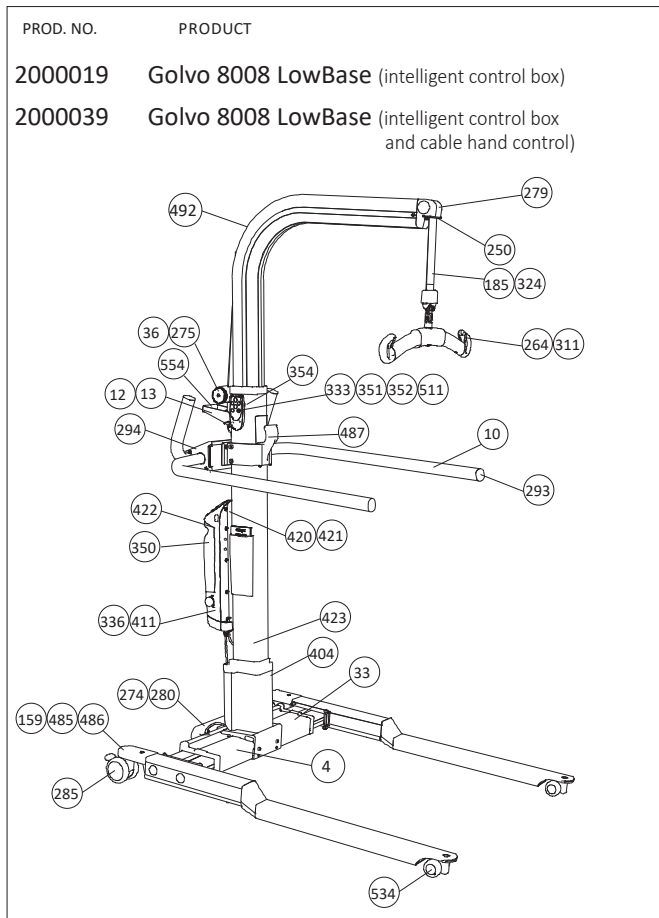
Golvo™ 7000/7007 mobile lift

(G7=Golvo 7000, G77=Golvo 7007, G8=Golvo 8000, G88=Golvo 8008, G9=Golvo 9000)

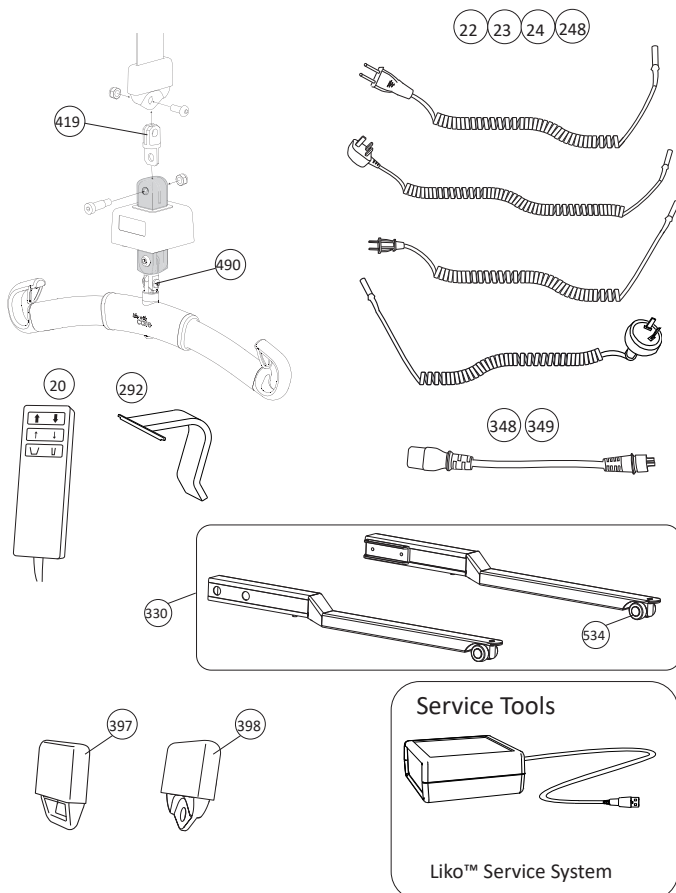
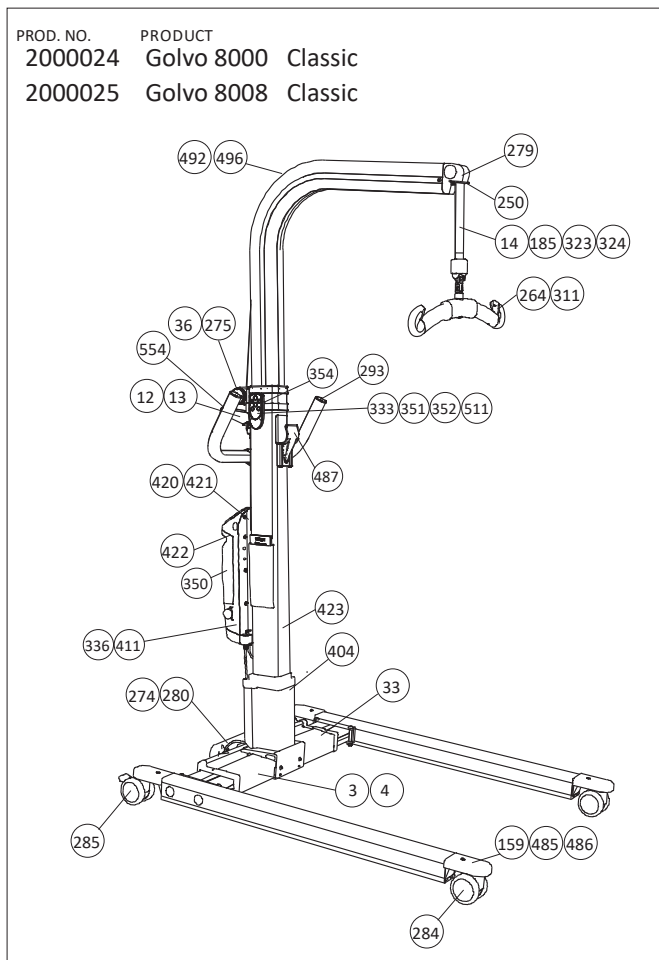
PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20090085	492	Lift strap Cover Strip, 7007/8008	pce		3EN690212
20090087	496	Lift strap Cover Strip, 7000/8000	pce		
20090092	534	Front Wheel LowBase <i>Fits: Golvo LowBase</i>	pce		3SE690241
20090091	535	Cable for Base Motor <i>Fits: Golvo 7000 to S/N 7024205</i> <i>Golvo 7007 to S/N 7729721</i>	set		3SE690240
20090102	554	Snap On Handle, 10 pce	set		-

(Discontinued 2017)

(Discontinued 2017)

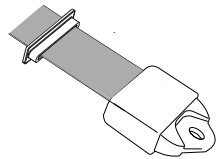
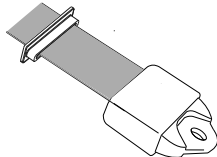
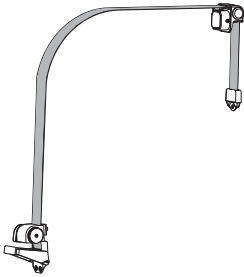






(Discontinued 2014)



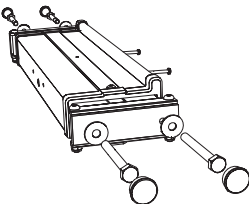
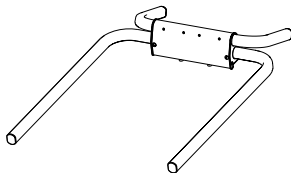
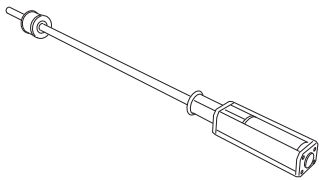
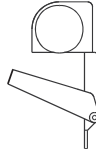
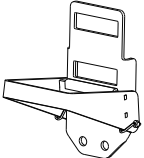
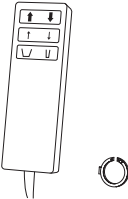
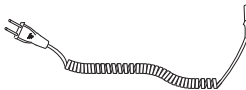

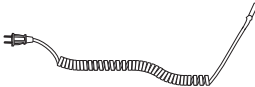
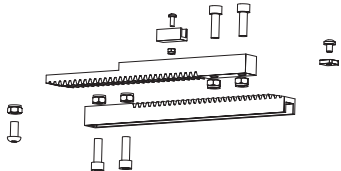
Golvo™ 8000/8008 mobile lift

(G7=Golvo 7000, G77=Golvo 7007, G8=Golvo 8000, G88=Golvo 8008, G9=Golvo 9000)

PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
Golvo LiftStrap					
20090018	14	Lift Strap G7 / G8 <i>Grey strap and grey plastic details</i>	pce		3EN650272
20090048	185	Lift Strap G77 (from S/N: 7723630) / G88 / G9 <i>Grey strap and grey plastic details</i>	pce		3EN650272
20090107	323	Easy Strapper G7 / G8 <i>Grey strap and grey plastic details</i>	set		3EN640248
20090106	324	Easy Strapper G77 from S/N 7723630 / G88 / G9 <i>Grey strap, grey plastic details and handle.</i>	set		
Golvo Control Units					
20490039	336	Control Unit CBL, by S/N <i>Can only be ordered if S/N (serial number) can be provided to Liko™ Customer Service.</i> <i>Will be programmed at Liko before delivery.</i> <i>2 weeks delivery time from Hill-Rom/Liko™ Alvik, Sweden</i>	pce		3SE650270
20490054	336C	Control Unit CBL by S/N, Cable <i>Can only be ordered if S/N (serial number) can be provided to Liko Customer Service.</i> <i>Will be programmed at Liko before delivery.</i> <i>2 weeks delivery time from Hill-Rom/Liko™ Alvik, Sweden</i>	pce		3SE690227
20090073	411	Control Unit CBL, Golvo <i>Before use; requires Software and, needs to be programmed with Liko™ Service System, Prod. No. 21090024</i>	pce		3SE630255
20090088	411C	Control Unit CBL Golvo, Cable <i>Before use; requires Software and, needs to be programmed with Liko™ Service System, Prod. No. 21090024</i>	pce		3SE690230


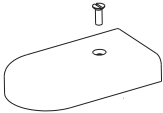
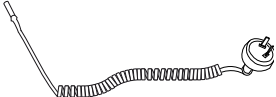

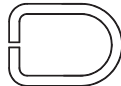
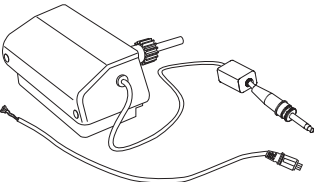
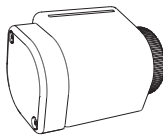
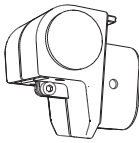
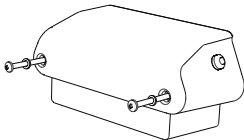



Golvo™ 8000/8008 mobile lift

(G7=Golvo 7000, G77=Golvo 7007, G8=Golvo 8000, G88=Golvo 8008, G9=Golvo 9000)

PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
Golvo Spare Parts					
20090094	3	Middle Beam 7000/8000 <i>(motor <u>not</u> included)</i>	pce		3EN690271
20090095	4	Middle Beam 7007/8008/9000 <i>(motor <u>not</u> included)</i>	pce		3EN690272
20090103	10	Golvo Armrest	set		3SE630280
20090014	11	Actuator G7 / 77 <i>Fits: From S/N:</i> <i>Golvo 7000 7024206</i> <i>Golvo 7007 7729722</i>	pce		3SE630249
20090104	12	Emergency Lowering Golvo <i>Grey plate</i>	pce		3EN640253
20090105	13	Body/Handle set for Emergency Lowering Device <i>Grey plate</i>	pce		3EN630285
20090089	20	Hand control 6-Button	pce		3SE690232
20090026	22	Cable for charging EU	pce		3EN630237
20090027	23	Cable for charging UK	pce		
20090028	24	Cable for charging US	pce		
20090096	33	Gear Rack set	set		3EN690273

Golvo™ 8000/8008 mobile lift

(G7=Golvo 7000, G77=Golvo 7007, G8=Golvo 8000, G88=Golvo 8008, G9=Golvo 9000)

PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20090041	36	Spring Magazine	pce		3EN630214
20390006	159	Fork Cover	pair		183916
20090050	248	Cable for Charging AU/NZ	pce		3EN630237
20090051	250	Reinforcement Plate Golvo	pce		3EN630215
31590011	264	Latch Universal SlingBar	pair		3SE630229
20090059	274	Motor Base Width Adjustment Golvo, grey <i>replaces 20090005 and 20090046</i>			3SE630216
20090060	275	Plastic Housing grey	pce		3EN630267
20090064	279	Roll Frame Attachment Golvo, grey	pce		3EN630269
20090065	280	Base Motor Cover grey	set		3EN630270
20490014	284	Front Wheel 75 mm Standard	pce		182904
20490015	285	Rear Wheel 75 mm Standard	pce		182903
20090058	292	Hook for Hand control 2,4,6	pce		3SE630210


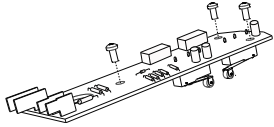

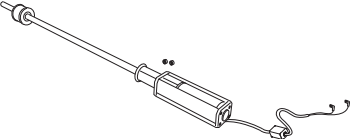
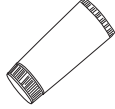
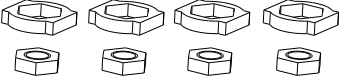
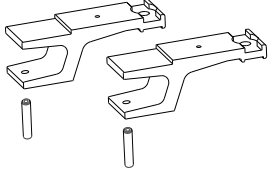

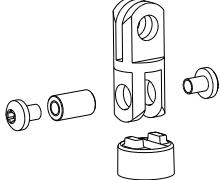
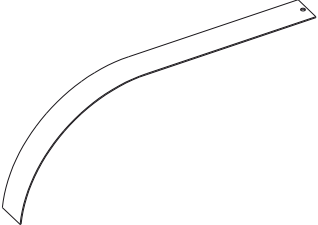
Golvo™ 8000/8008 mobile lift

(G7=Golvo 7000, G77=Golvo 7007, G8=Golvo 8000, G88=Golvo 8008, G9=Golvo 9000)

PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20090066	293	End Cover Handle/Arm Support grey, 4 pce	set		3SE630271
20090067	294	End Cover Armrest, grey	set		3SE630272
31590014	311	Latch, composite <i>Fits Universal SlingBar from S/N 1299511</i>	pair		3SE640242
20090071	330	Low base kit Golvo 7007/8008	set		3EN640262
20490031	333	Wireless hand control HBL20	set		3SE640298
21090018	348	Extension cable CBL	pce		3SE650216
21090019	349	Extension cable CBL, US	pce		
2006107	350	Extra Battery Box	pce		7EN140105 User Manual
21090020	351	Beltclip HBL20	pair		3SE650217
21090021	352	Clips with hook HBL20	pair		3SE650218
21090022	353	Charge cable hook , 10 pce	set		3SE650219
21090023	354	Hand control hanger, 10 pce	set		3SE650220
31590005	397	Q-link	pce		3EN640236
31590025	398	Lift strap joint	pce		3EN640237
20090072	404	Friction plates 2.8-3.4 mm <i>20 pce of each model (2.8, 3.0, 3.2, 3.4 mm)</i>	set		3EN650254
2006010	419	Adapter Golvo (for Scale)	set		3SE650244

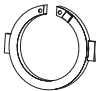
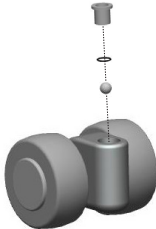
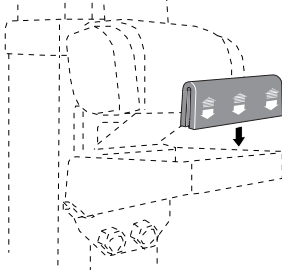
Golvo™ 8000/8008 mobile lift

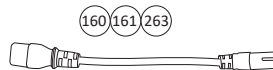
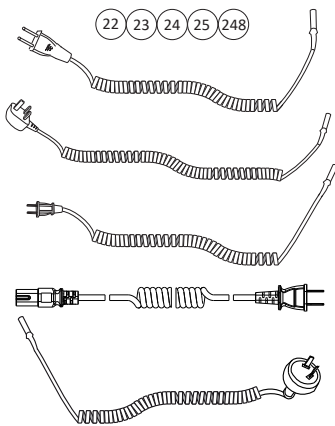
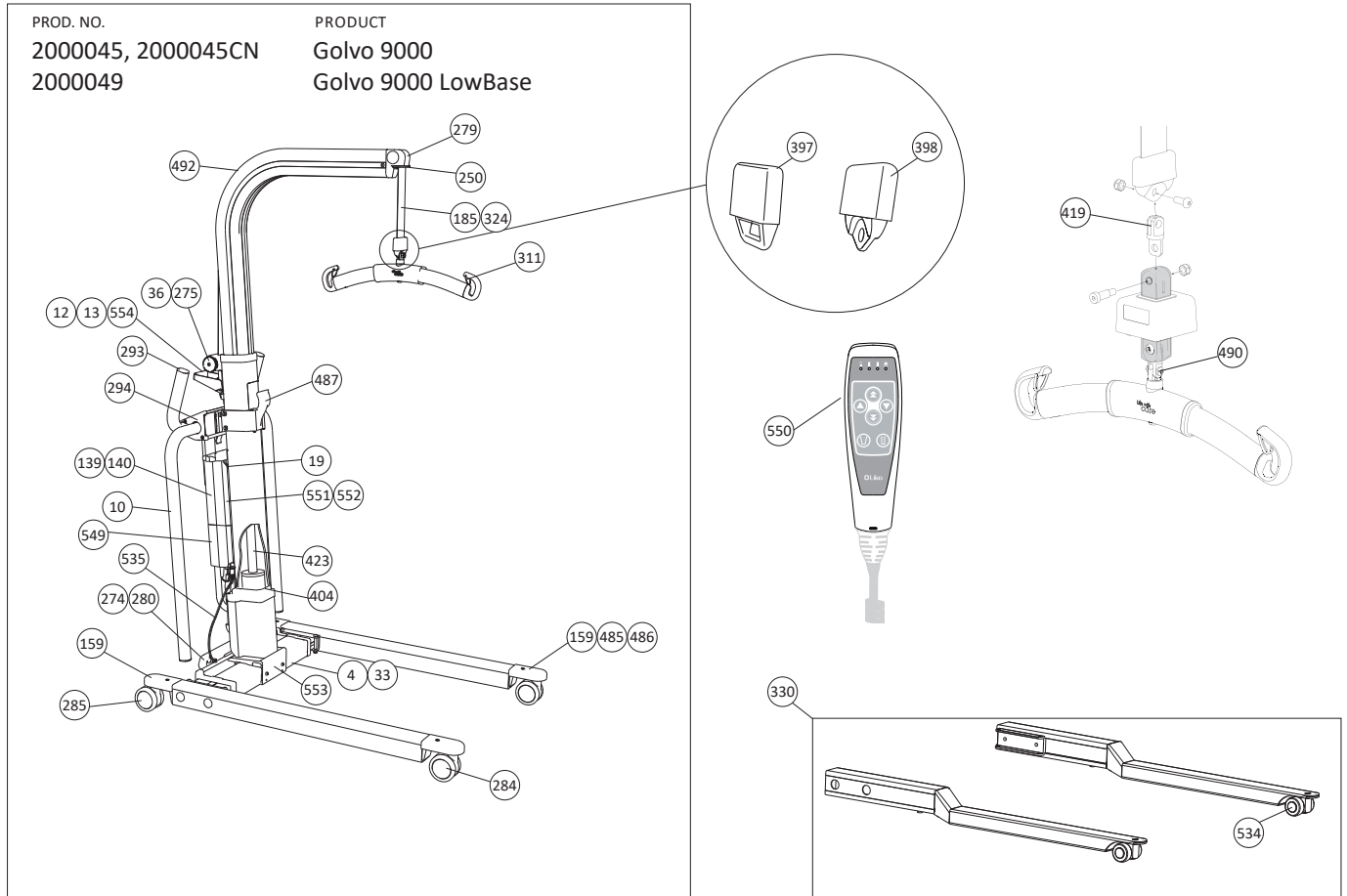
(G7=Golvo 7000, G77=Golvo 7007, G8=Golvo 8000, G88=Golvo 8008, G9=Golvo 9000)

PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20090074	420	Limit switch box	set		3SE630266
20090076	421	Limit Switch PCBA	set		3SE630287
20090077	422	Locking device CBL20, 10 pce	set		3SE640217
20090078	423	Actuator G8 / G88 / G9	pce		3EN630263
21090028	425	Grease SHC460 30g	pce		-
20090080	485	Fork locking attachment, 4 pair	set		3SE690208
20090081	486	Wheel Fork	pair		3SE690209
20090082	487	Parking bracket	pce		3SE690204
20490052	490	SlingBar Assembly set	set		3SE690210
20090085	492	Lift strap Cover Strip, 7007/8008	pce		3EN690212
20090087	496	Lift strap Cover Strip, 7000/8000	pce		

Golvo™ 8000/8008 mobile lift

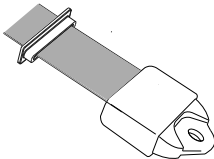
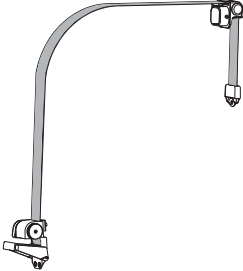
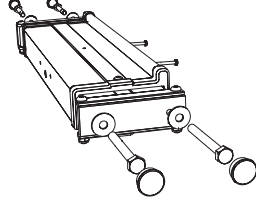
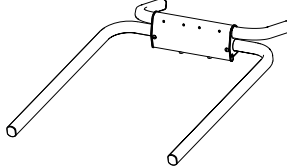
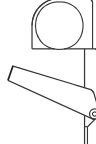
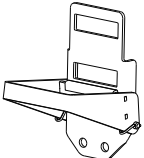
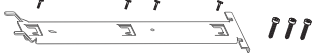




(G7=Golvo 7000, G77=Golvo 7007, G8=Golvo 8000, G88=Golvo 8008, G9=Golvo 9000)

PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20490057	511	Locking Ring CBL, 10 pce	set		3SE690231
20090092	534	Front Wheel LowBase <i>Fits: Golvo LowBase</i>	pce		3SE690241
20090102	554	Snap On Handle, 10 pce	set		-



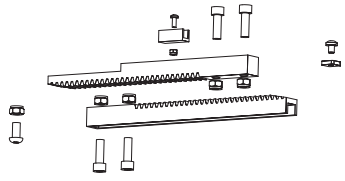
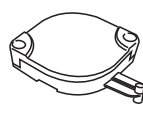
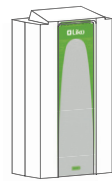
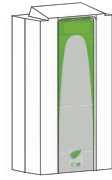
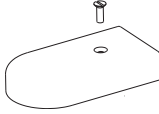

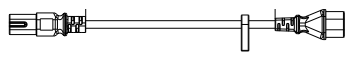


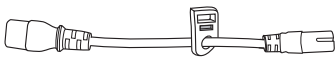
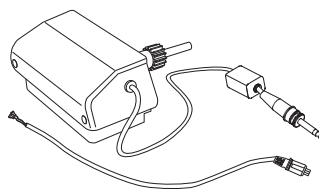
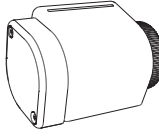
Golvo™ 9000 mobile lift

(G7=Golvo 7000, G77=Golvo 7007, G8=Golvo 8000, G88=Golvo 8008, G9=Golvo 9000)

PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
Golvo LiftStrap					
20090048	185	Lift Strap G77 / G88 / G9	pce		3EN650272
20090106	324	Easy Strapper G77 / G88 / G9	set		3EN640248
Golvo Spare Parts					
20090095	4	Middle Beam 7007 / 8008 / 9000 (motor <u>not</u> included)	pce		3EN690272
20090103	10	Golvo Armrest	set		3SE630280
20090104	12	Emergency Lowering Golvo Grey plate	pce		3EN640253
20090105	13	Body/Handle set for Emergency Lowering Device Grey plate	pce		3EN630285
20090023	19	Attachment for Control Box	pce		3EN630238
20090026	22	Cable for charging EU	pce		3EN630237
20090027	23	Cable for charging UK	pce		
20090028	24	Cable for charging US	pce		
20090029	25	Cable for charging JP	pce		

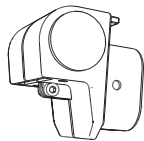
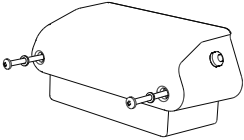

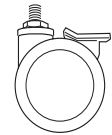

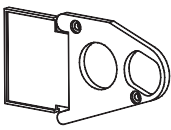

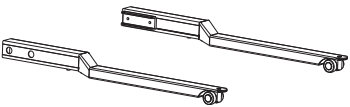
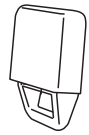
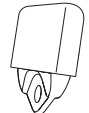
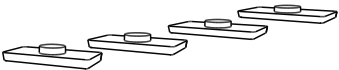
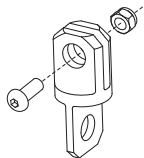
Golvo™ 9000 mobile lift

(G7=Golvo 7000, G77=Golvo 7007, G8=Golvo 8000, G88=Golvo 8008, G9=Golvo 9000)

PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20090096	33	Gear Rack set	set		3EN690273
20090041	36	Spring Magazine	pce		3EN630214
2006106	139	Battery Lead Acid	pce		3EN630224
2006110	140	Battery Lithium Ion	pce		3EN690296
20390006	159	Fork Cover	pair		183916
21090001	160	Extension Cable	pce		3EN630225
21090000	161	Extension Cable JP	pce		3EN690280
20090050	248	Cable for Charging AU/NZ	pce		3EN630237
20090051	250	Reinforcement Plate Golvo	pce		3EN630215
21090011	263	Extension Cable US	pce		3EN630218
20090059	274	Motor Base Width Adjustment Golvo, grey			3SE630216
20090060	275	Plastic Housing grey	pce		3EN630267

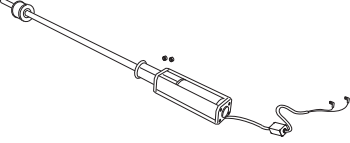
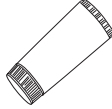
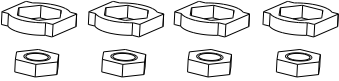
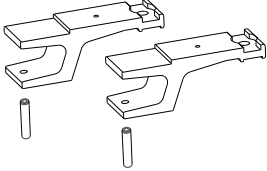

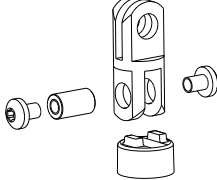
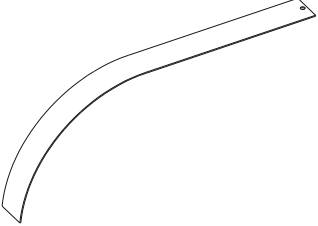
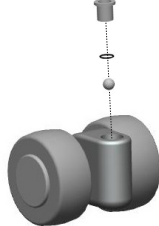
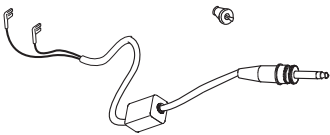
Golvo™ 9000 mobile lift

(G7=Golvo 7000, G77=Golvo 7007, G8=Golvo 8000, G88=Golvo 8008, G9=Golvo 9000)

PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20090064	279	Roll Frame Attachment Golvo, grey	pce		3EN630269
20090065	280	Base Motor Cover, grey	set		3EN630270
20490014	284	Front Wheel 75 mm Standard	pce		182904
20490015	285	Rear Wheel 75 mm Standard	pce		182903
20090066	293	End Cover Handle/Arm Support grey, 4 pce	set		3SE630271
20090067	294	End Cover Armrest, grey	set		3SE630272
31590014	311	Latch, composite <i>Fits Universal SlingBar from S/N 1299511</i>	pair		3SE640242
20090071	330	Low base kit Golvo 7007 / 8008 / 9000	set		3EN640262
31590005	397	Q-link	pce		3EN640236
31590025	398	Lift strap joint	pce		3EN640237
20090072	404	Friction plates 2.8-3.4 mm <i>20 pce of each model (2.8, 3.0, 3.2, 3.4 mm)</i>	set		3EN650254
2006010	419	Adapter Golvo (for Scale)	set		3SE650244

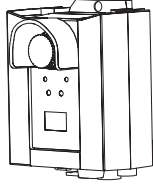

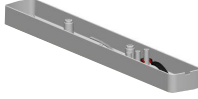

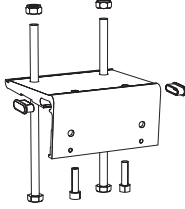
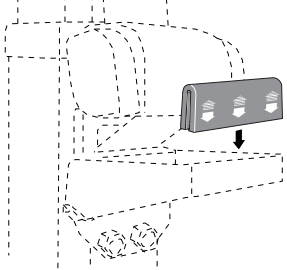
Golvo™ 9000 mobile lift

(G7=Golvo 7000, G77=Golvo 7007, G8=Golvo 8000, G88=Golvo 8008, G9=Golvo 9000)

PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20090078	423	Actuator G8 / G88 / G9	pce		3EN630263
21090028	425	Grease SHC460, 30g	pce		-
20090080	485	Fork locking attachment, 4 pair	set		3SE690208
20090081	486	Wheel Fork	pair		3SE690209
20090082	487	Parking bracket	pce		3SE690204
20490052	490	SlingBar Assembly set	set		3SE690210
20090085	492	Lift strap Cover Strip, 7007 / 8008 / 9000	pce		3EN690212
20090092	534	Front Wheel LowBase <i>Fits: Golvo LowBase</i>	pce		3SE690241
20090091	535	Cable for Base Motor	set		3SE690240

Golvo™ 9000 mobile lift

(G7=Golvo 7000, G77=Golvo 7007, G8=Golvo 8000, G88=Golvo 8008, G9=Golvo 9000)

PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20090097	549	Control Box Jumbo Care	pce		3EN690282
20090098	550	Hand Control 6 button, LED	pce		3EN690283
20090099	551	Limit Switch Box, Jumbo Care	pce		3EN690284
20090100	552	Limit Switch PCBA, Jumbo Care	pce		3EN690285
20090101	553	Mast attachment	set		3EN690286
20090102	554	Snap On Handle, 10 pce	set		-

Golvo™ mobile lift - Spare Parts Matrix

SPARE PART NO.	POS. NO.	NAME	2000009	2000010	2000050	2000051	2000061	2000019	2000039	2000014	2000034	2000015	2000035	2000024	2000025	2000045 2000045CN	2000049	
20090094	3	Middle Beam 7000/8000	7000 ES fr. S/N 7024206 (Face lift)	X						X				X			9000 LowBase	
20090095	4	Middle Beam 7007/8008/9000		X		X						X			X			X
20090103	10	Golvo Armrest		X		X		X	X	X								
20090014	11	Actuator G7 / 77		X	X	X												
20090104	12	Emergency Lowering Device Golvo, grey		X	X	X		X	X	X	X	X	X	X	X			
20090105	13	Body/Handle set for Em.Low. Device		X	X	X		X	X	X	X	X	X	X	X			
20090018	14	Lift Strap G7/G8		X	X					X				X				
20090022	18	Control Box G7-77 ES		X	X	X												
20090023	19	Attachment for Control Box		X	X	X												
20090089	20	Hand Control 6-Button		X	X	X			X		X							
20090026	22	Cable for charging EU		X	X	X		X	X	X	X	X	X	X	X			X
20090027	23	Cable for charging UK		X	X	X		X	X	X	X	X	X	X	X			X
20090028	24	Cable for charging US		X	X	X		X	X	X	X	X	X	X	X			X
20090029	25	Cable for charging JP																X
20090096	33	Gear Rack set		X	X	X		X	X	X	X	X	X	X	X			
20090041	36	Spring Magazine		X	X	X		X	X	X	X	X	X	X	X			
2006106	139	Battery Lead Acid		X	X	X		X	X									
2006110	140	Extra battery Lithium Ion																X
20390006	159	Fork Cover		X	X	X		X	X	X	X	X	X	X	X			X
21090001	160	Extension Cable		X	X	X		X	X									X
21090000	161	Extension Cable, JP																X
20090048	185	Lift Strap G77 ES / G88 / G9		X	X	X		X	X	X	X	X	X	X	X			X
20090050	248	Cable for Charging AU/NZ		X	X	X		X	X	X	X	X	X	X	X			X
20090051	250	Reinforcement Plate		X	X	X		X	X	X	X	X	X	X	X			X
20090052	251	Limit Switch G7 / 77		X	X	X		X	X	X	X	X	X	X	X			X
21090011	263	Extension Cable US		X	X	X		X	X	X	X	X	X	X	X			X

Golvo™ mobile lift - Spare Parts Matrix

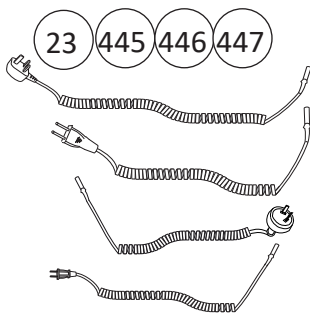
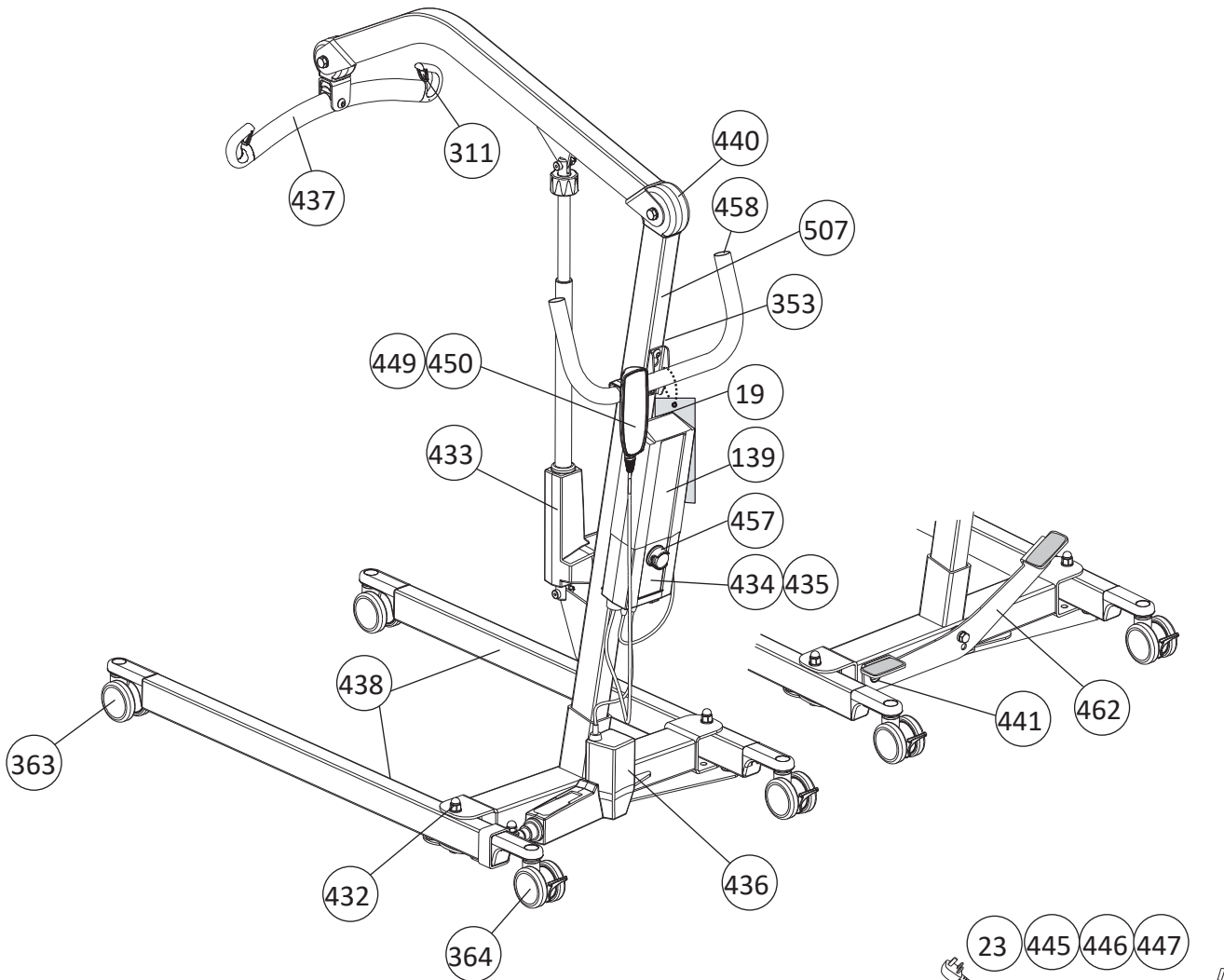
SPARE PART NO.	POS. NO.	NAME	2000009	2000010	2000050	2000051	2000061	2000019	2000039	2000014	2000034	2000015	2000035	2000024	2000025	2000045 2000045CN	2000049
31590011	264	Latch Universal SlingBar	X	X	X	X	X	X	X	X	X	X	X	X	X		
20090059	274	Motor for Base with Adjustment Golvo, grey	X	X	X	X	X	X	X	X	X	X	X	X	X		9000 LowBase
20090060	275	Plastic Housing, grey	X	X	X	X	X	X	X	X	X	X	X	X	X		
20090064	279	Roll Frame Attachment Golvo, grey	X	X	X	X	X	X	X	X	X	X	X	X	X		
20090065	280	Base Motor Cover, grey	X	X	X	X	X	X	X	X	X	X	X	X	X		
20490014	284	Front Wheel 75 mm Standard	X	X	X	X	X	X	X	X	X	X	X	X	X		
20490015	285	Rear Wheel 75 mm , Std	X	X	X	X	X	X	X	X	X	X	X	X	X		
20090058	292	Hook for Hand control	X	X	X	X	X	X	X	X	X	X	X	X	X		
20090066	293	End Cover Handle/Arm Support, grey	X	X	X	X	X	X	X	X	X	X	X	X	X		
20090067	294	End Cover Armrest, grey	X	X	X	X	X	X	X	X	X	X	X	X	X		
31590014	311	Latch, composite	X	X	X	X	X	X	X	X	X	X	X	X	X		
20090107	323	Easy Strapper G7/G8	X	X	X	X	X	X	X	X	X	X	X	X	X		
20090106	324	Easy Strapper G77/G88 from S/N 7723630 / G9	X	X	X	X	X	X	X	X	X	X	X	X	X		
20090071	330	Low base kit Golvo 7007/8008	X	X	X	X	X	X	X	X	X	X	X	X	X		
20490031	333	Wireless hand control HBL20						X	X	X	X	X	X	X	X		
20490039	336	Control Unit CBL, by S/N						X	X	X	X	X	X	X	X		
20490054	336C	Control Unit CBL by S/N, cable							X	X	X	X	X	X	X		
21090018	348	Extension cable CBL20						X	X	X	X	X	X	X	X		
21090019	349	Extension cable CBL20, US						X	X	X	X	X	X	X	X		
2006107	350	Extra Battery Box						X	X	X	X	X	X	X	X		
21090020	351	Beltclip HBL20						X	X	X	X	X	X	X	X		
21090021	352	Clips with hook HBL20						X	X	X	X	X	X	X	X		
21090022	353	Charge cable hook						X	X	X	X	X	X	X	X		
21090023	354	Hand control hanger						X	X	X	X	X	X	X	X		
31590005	397	Q-link	X	X	X	X	X	X	X	X	X	X	X	X	X		

Golvo™ mobile lift - Spare Parts Matrix

SPARE PART NO.	POS. NO.	NAME	2000009	2000010	2000050	2000051	2000061	2000019	2000039	2000014	2000034	2000015	2000035	2000024	2000025	2000045 2000045CN	2000049
31590025	398	Lift strap joint	x	x	x	x	x	x	x	x	x	x	x	x			
20090072	404	Friction plates 2,8-3,4 mm	x	x	x	x	x	x	x	x	x	x	x	x			
20090073	411	Control Unit CBL, Golvo															
20090088	411C	Control Unit CBL, Golvo Cable															
2006010	419	Adapter Golvo (for Scale)	x	x	x	x	x	x	x	x	x	x	x	x			
20090074	420	Limit Switch Box															
20090076	421	Limit Switch PCBA															
20090077	422	Locking device CBL20,															
20090078	423	Actuator G8 / G88 / G9															
21090028	425	Grease SHC460 30g	x	x	x	x	x	x	x	x	x	x	x	x			
20090080	485	Fork locking attachment	x	x	x	x	x	x	x	x	x	x	x	x			
20090081	486	Wheel Fork	x	x	x	x	x	x	x	x	x	x	x	x			
20090082	487	Parking bracket	x	x	x	x	x	x	x	x	x	x	x	x			
20490052	490	SlingBar Assembly set	x	x	x	x	x	x	x	x	x	x	x	x			
20090085	492	Lift strap Cover Strip 7007/8008															
20090087	496	Lift strap Cover Strip 7000/8000	x		x									x			
20490057	511	Locking Ring CBL															
20090092	534	Front Wheel LowBase															
20090091	535	Cable for Base Motor	x	x	x	x	x										
20090097	549	Control Box Jumbo Care															
20090098	550	Hand Control 6 Buttons, LED															
20090099	551	Limit switch box Jumbo care															
20090100	552	Limit switch PCBA Jumbo Care															
20090101	553	Mast attachment	x	x	x	x	x	x	x	x	x	x	x	x			
20090102	554	Snap On Handle	x	x	x	x	x	x	x	x	x	x	x	x			

Spare Parts Positions Liko™ M220/ Liko™ M230 mobile lift

PROD. NO.	PRODUCT
2050010	Liko M220
2050015	Liko M230



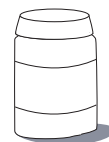
443





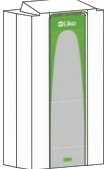



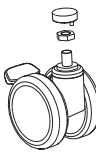
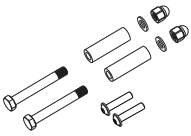
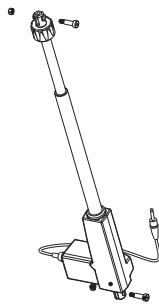
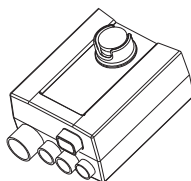
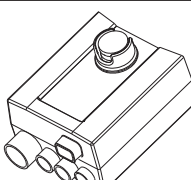
444



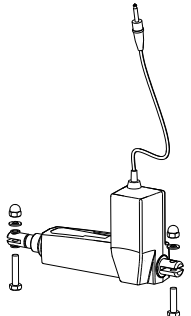
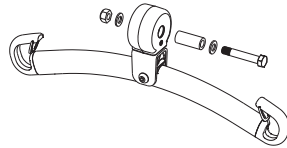
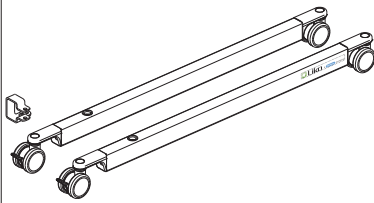
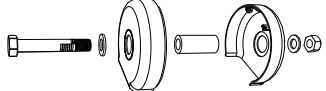
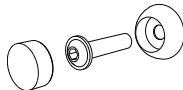






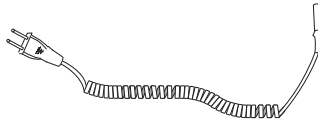
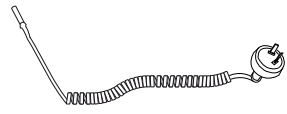
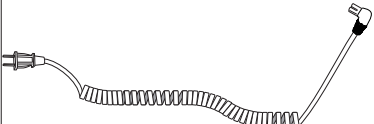
448



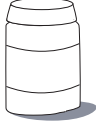



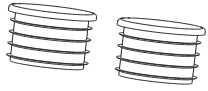
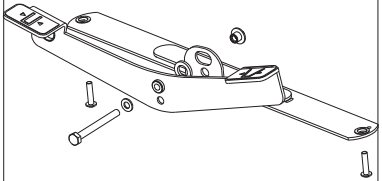
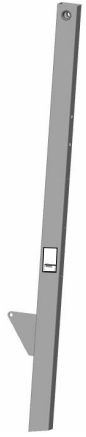
Liko™ M220 / Liko™ M230 mobile lift

PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20090023	19	Attachment for Control Box	pce		3EN630238
20090027	23	Cable for charging UK	pce		3EN630237
2006106	139	Battery Lead Acid	pce		3EN630224
31590014	311	Latch, composite <i>Fits Universal SlingBar from S/N 1299511</i>	pair		3SE640242
21090022	353	Charge cable hook , 10 pce	set		3SE650219
20190065	363	Front wheel 75m	pce		3SE650221
20190066	364	Rear wheel 75mm	pce		3SE650222
20590001	432	Base screw assembly	set		3EN650275
20590002	433	Actuator	pce		3SE650276
20590003	434	Control Unit M220	pce		3SE650277
20590004	435	Control Unit M230	pce		3SE650277

Liko™ M220 / Liko™ M230 mobile lift

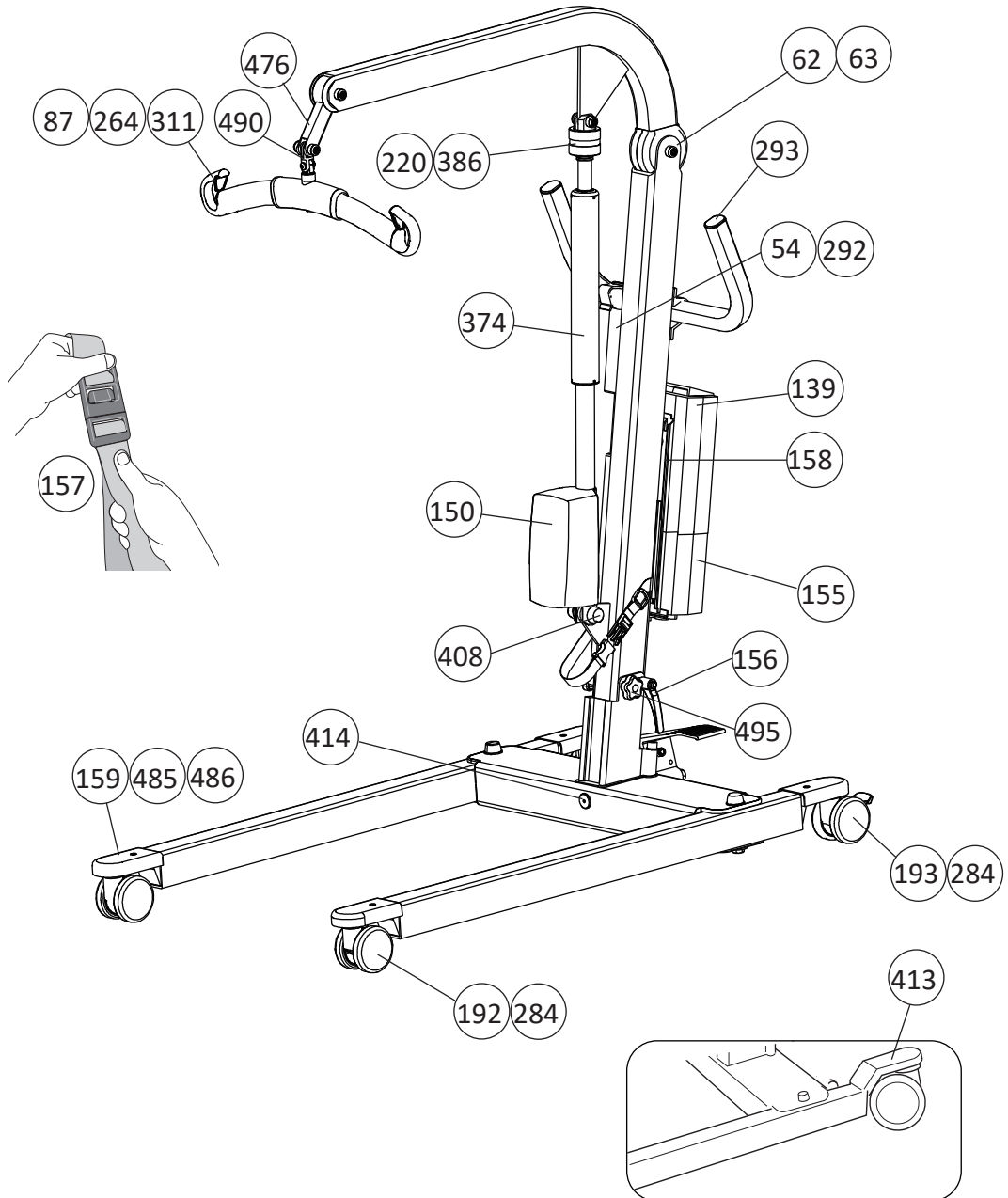
PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20590005	436	Base Motor	pce		3SE650278
20590006	437	SlingBar Liko 450	pce		3EN650279
20590007	438	Liko M220/230 Legs	pair		3SE650280
20590008	440	Lift arm plastic cover	set		3SE650281
20590009	441	Pedal stop, 10 pce (3 parts/pce)	set		3SE650282
20590012	443	Labels Liko M220	set	2 x  1 x  1 x  2 x 	3SE650283
20590013	444	Labels Liko M230	set	2 x  2 x 	3SE650283
20590014	445	Cable for Charging EU	pce		3SE650284
20590015	446	Cable for Charging AU / NZ	pce		
20590016	447	Cable for Charging US	pce		

Liko™ M220 / Liko™ M230 mobile lift

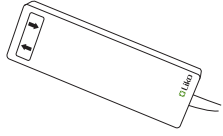
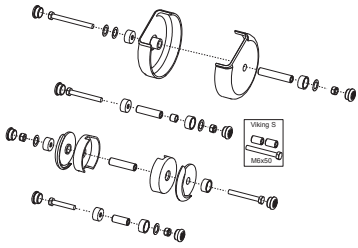
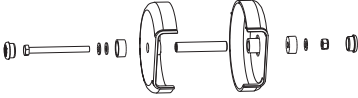
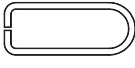
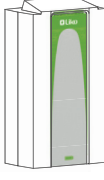
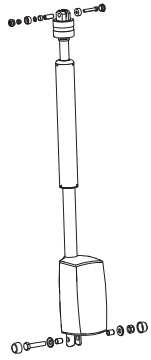
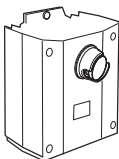
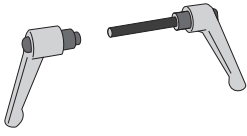

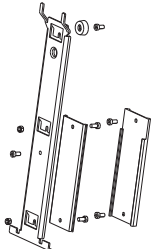
PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20190048	448	Lacquer Top Coat for Improvement, Grey	pce		-
20590010	449	Hand Control 2-button M220	pce		3SE650285
20590011	450	Hand Control 4-button M230	pce		
20590018	457	Emergency guard, grey. 10 pce	set		3SE650286
20590019	458	End cap	pair		3SE650292
20590020	462	Pedal assembly	set		3SE650296
20590022	507	Mast M220/M230	pce		3SE690213

Spare Parts Positions LikoLight™ mobile lift

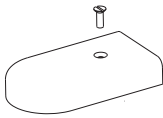

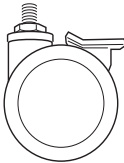
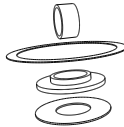



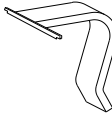


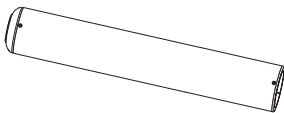
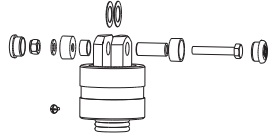

PROD. NO.	PRODUCT
2030001	LikoLight (Discontinued 2017)
2030001H	LikoLight, large wheels: 100/125 (Discontinued 2013)



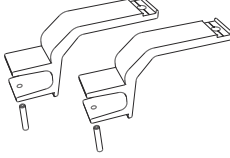
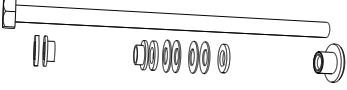
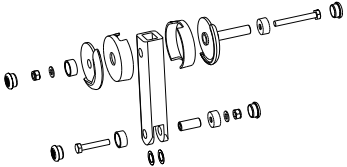
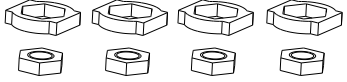
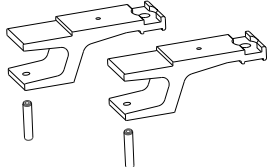
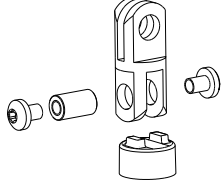
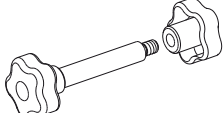
LikoLight™ mobile lift

PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20190016	54	Hand control EM 2-Button	pce		3SE630204
20190044	62	Overhead Beam Restoration set	set		3SE630202
20190045	63	Lift Arm Plastic Cover	set		3SE630211
31190018	87	Latch	pair		3EN640271
2006106	139	Battery Lead Acid	pce		3EN630224
20390008	150	Actuator LikoLight	pce		3SE630291
20390007	155	Control Box LikoLight	pce		3SE630223
20390003	156	Locking Handle	set		3SE630288
20390004	157	Conveyor Strap	pce		3SE630289
20390005	158	Attachment for Control Box LikoLight	set		3SE630290

LikoLight™ mobile lift

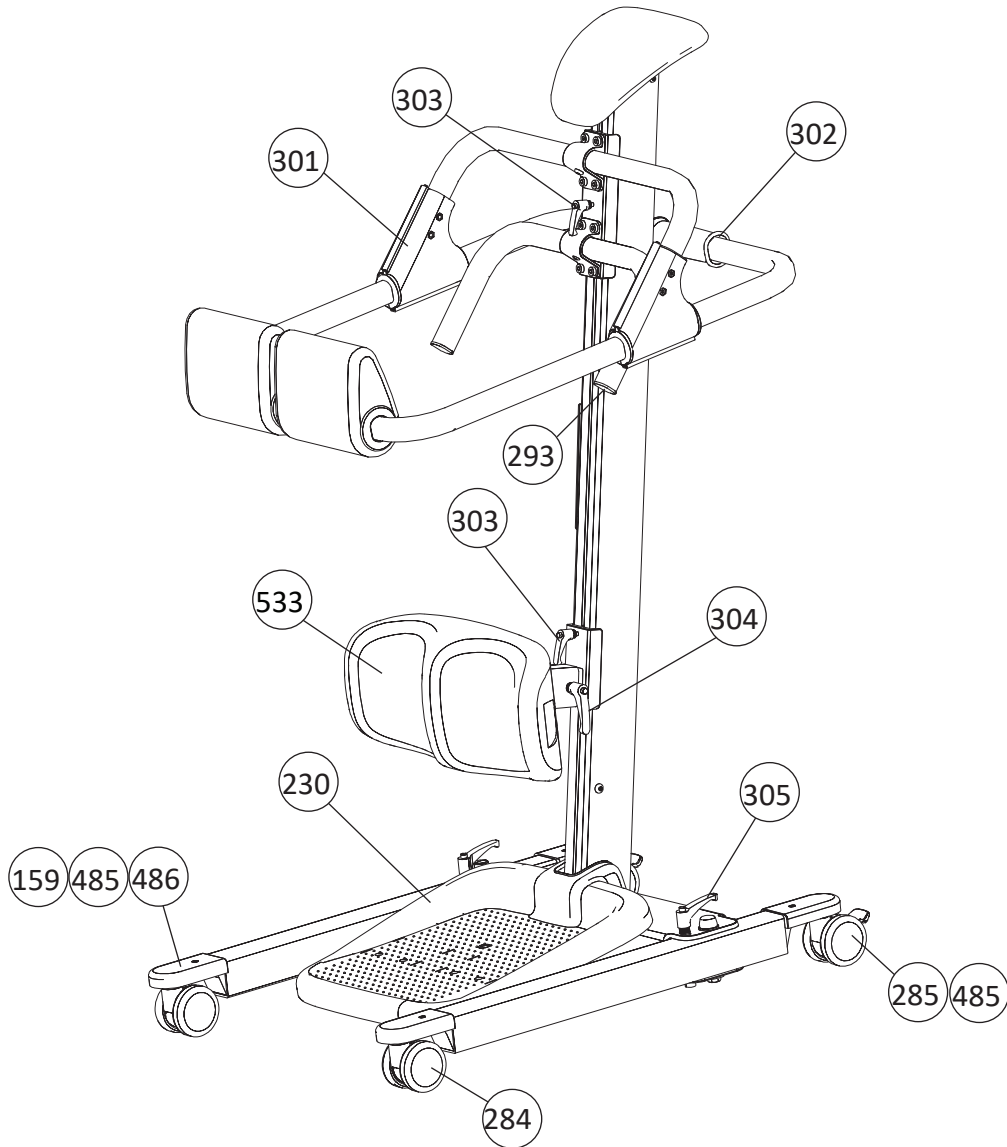
PROD. NO.	POS. NO.	NAME	UNIT	ASSEMBLY INSTR.
20390006	159	Fork Cover	pair	 183916
20490003	192	Front Wheel 100 mm <i>Fits 2030001H, LikoLight™ large wheels</i> <i>We recommend replacing both wheels, as there are two different versions that can vary in appearance. (dimension is the same)</i>	pce	 3SE630293
20490004	193	Rear Wheel 125 mm <i>Fits 2030001H, LikoLight™ large wheels</i> <i>We recommend replacing both wheels, as there are two different versions that can vary in appearance. (dimension is the same)</i>	pce	 3SE630294
20190051	220	Restoration set Emergency Lowering	set	 3SE630222
31590014	311	Latch, composite <i>Fits Universal SlingBar from S/N 1299511</i>	pair	 3SE640242
20490014	284	Front Wheel 75 mm Standard <i>From S/N 2503102</i>	pce	 182904
20490015	285	Rear Wheel 75 mm Standard <i>From S/N 2503102</i>	pce	 182903
20090058	292	Hook for Hand control 2,4,6	pce	 3SE630210
20090066	293	End Cover Handle/Arm Support grey, 4 pce	set	 3SE630271
31590014	311	Latch, composite <i>Fits Universal SlingBar from S/N 1299511</i>	pair	 3SE640242
20390017	374	Outer Tube LA32, 310 mm	pce	 3SE650237
20390018	386	Emergency lowering	set	 3SE650245
20190072	408	Lower Actuator attachment	set	 3SE650257

LikoLight™ mobile lift

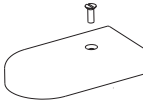
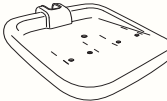




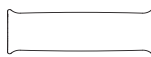
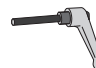


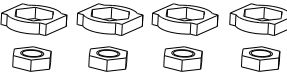
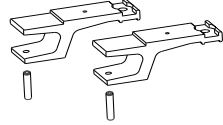


PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20490037	413	Rear Wheel fork	pair		3SE650264
21090036	414	Attachment base adjustment	set		3SE650266
20190074	476	Flexlink Alu	set		3SE650297
20090080	485	Fork locking attachment, 4 pair	set		3SE690208
20090081	486	Wheel Fork	pair		3SE690209
20490052	490	SlingBar Assembly set	set		3SE690210
20390019	495	Guide Screw	set		3SE690216

Spare Parts Positions RollOn™ support for raising

PROD. NO.	PRODUCT
6001001	RollOn (Discontinued 2020)

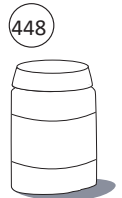
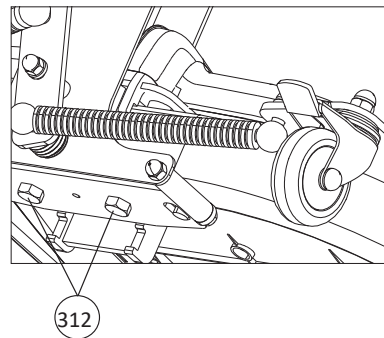
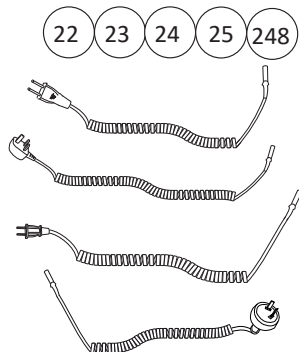
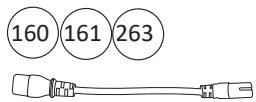
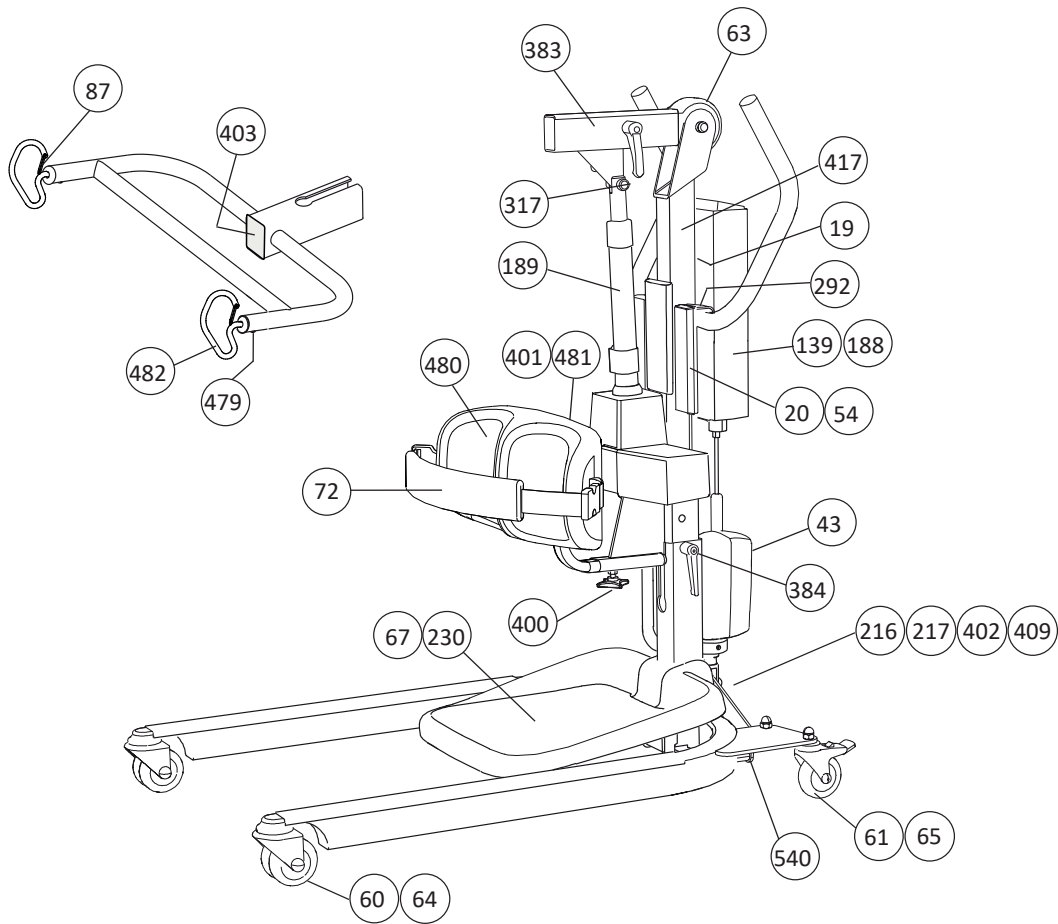


RollOn™ support for raising


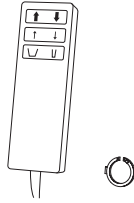
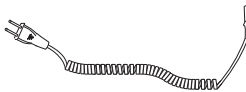

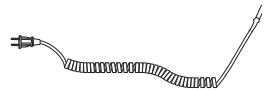
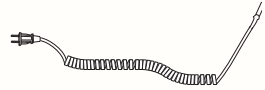
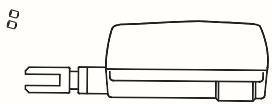
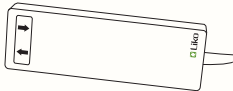
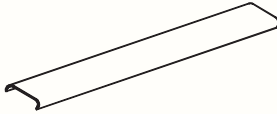
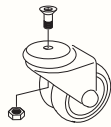
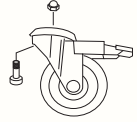
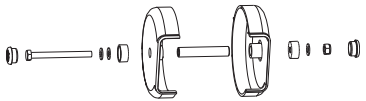
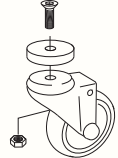
PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20390006	159	Fork Cover	pair		183916
20290026	230	Foot Tray cover , Sa II	pce		3SE630245
20490014	284	Front Wheel 75 mm Standard	pce		182904
20490015	285	Rear Wheel 75 mm Standard	pce		182903
20090066	293	End Cover Handle/Arm Support grey, 4 pce	set		3SE630271
60090001	301	Plastic cover X-cross	pce		3EN630231
60090002	302	Safety lock	pce		3EN630232
60090003	303	Locking Handle Mast	pce		3EN630233
60090004	304	Locking Handle Leg Support	set		183012
60090005	305	Locking Handle Base	pce		3EN630235
20090080	485	Fork locking attachment, 4 pair	set		3EN690208
20090081	486	Wheel Fork	pair		3EN690209
183010	533	Leg Support	pce		182213
6001100SVFI 6001100ENES 6001100DEFR 6001100FRNL		Quick Reference Guide RollOn Swedish/Finnish English/Spanish German/French French/Dutch	pce pce pce pce		-

Spare Parts Positions Sabina™ II mobile lift

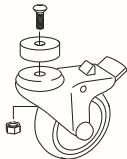
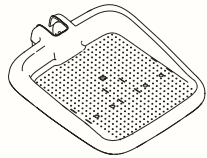
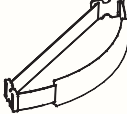
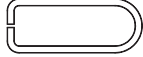
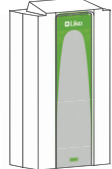


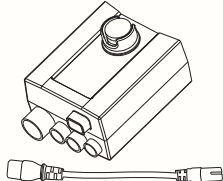
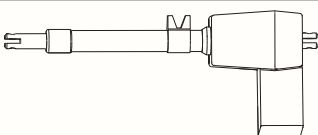
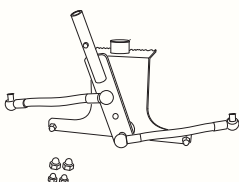
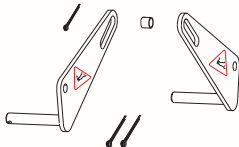
PROD. NO.	PRODUCT
2020003	Sabina II EE
2020004	Sabina II EM, (Discontinued 2013)



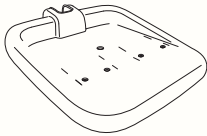

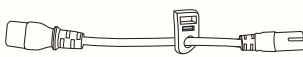
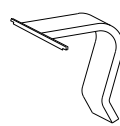
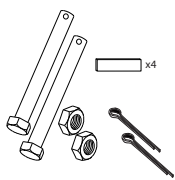
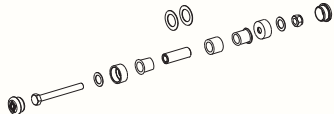



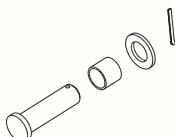

Sabina™ II EE, EM mobile lift

PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20090023	19	Attachment for Control Box	pce		3EN630238
20090089	20	Hand control 6-Button <i>fits Sabina EE / Sabina II EE</i>	pce		3SE690232
20090026	22	Cable for charging EU	pce		3EN630237
20090027	23	Cable for charging UK	pce		
20090028	24	Cable for charging US	pce		
20090029	25	Cable for charging JP	pce		3EN690279
20190003	43	Actuator for Base Widening <i>fits Sabina EE / Sabina II EE</i>	pce		3SE640284
20190016	54	Hand control EM 2-Button <i>fits Sabina EE / Sabina II EE</i>	pce		3SE630204
20190029	58	Leg Protector	pair		183073
20190042	60	Front Wheel 75 mm	pce		3SE640283
20190043	61	Rear Wheel 75 mm	pce		3SE640282
20190045	63	Lift Arm Plastic Cover	set		3SE630211
20190046	64	Front Wheel 100 mm	pce		3SE640281

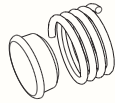
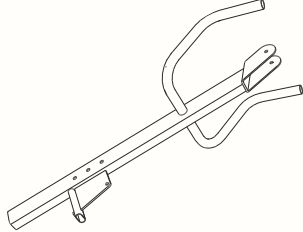
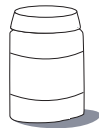
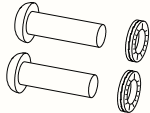
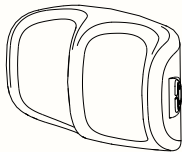
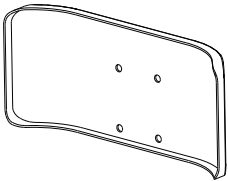
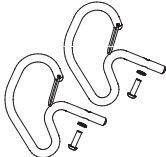

Sabina™ II EE, EM mobile lift

PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20190047	65	Rear Wheel 100 mm	pce		3SE640285
20290006	67	Foot Tray complete	pce		3SE630245
20290022	72	Calf Strap <i>(Registered as an accessory from 2011)</i>	pce		-
31190018	87	Latch	pair		3EN640271
2006106	139	Battery Lead Acid	pce		3EN630224
21090001	160	Extension Cable	pce		3EN630225
21090000	161	Extension Cable JP	pce		3EN690280
20290024	188	Control Box Sa II	pce		3SE630223
20290024US	188	Control Box Sa II, US <i>only for the US market</i>	pce		
20290024JP	188	Control Box Sa II, JP <i>only for the Japanese market</i>	pce		
20290023	189	Actuator Sa II	pce		3SE640247
20190049	216	Restoration set Manual Base	set		3SE630226
20190050	217	Restoration set Electrical Base	set		3SE630227

Sabina™ II EE, EM mobile lift

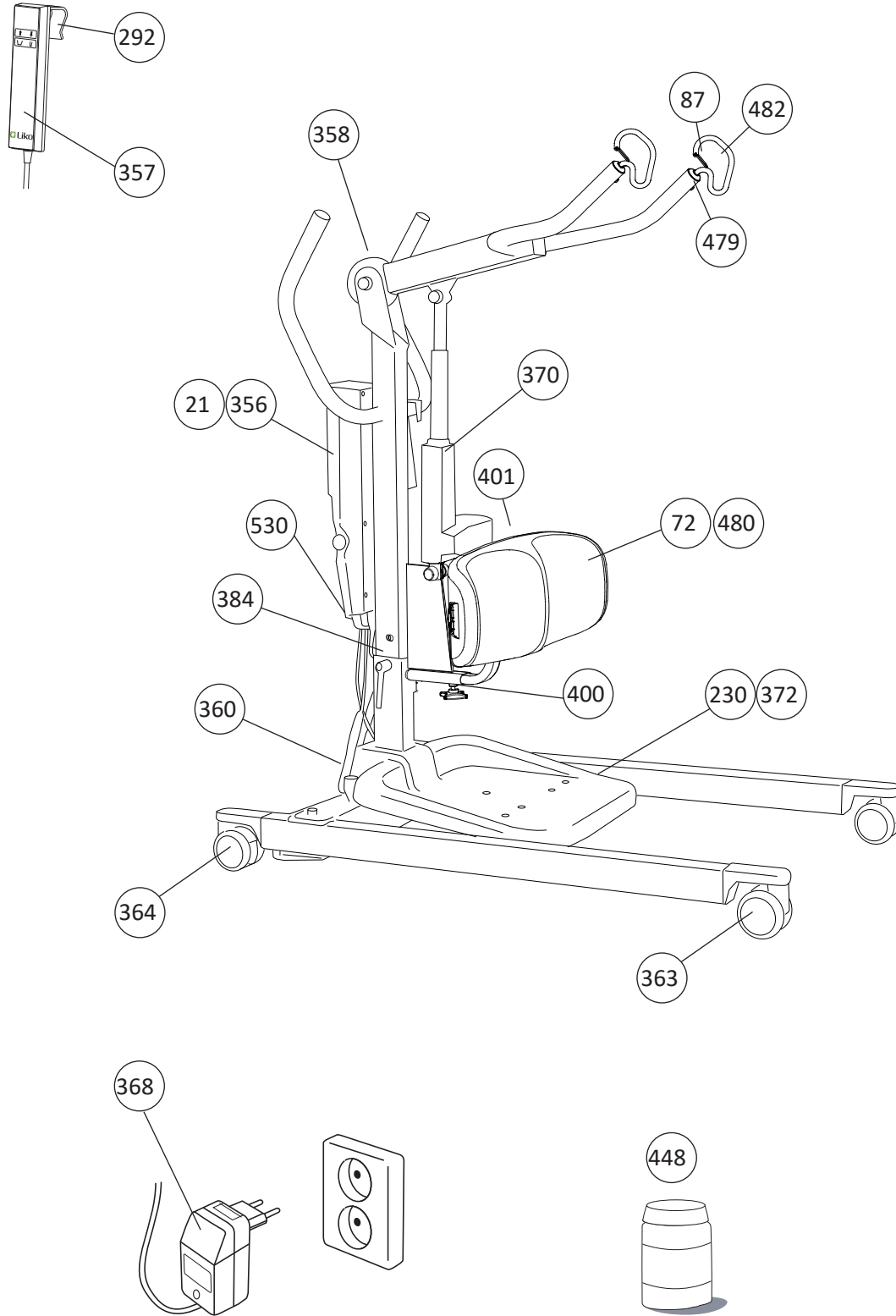
PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20290026	230	Foot Tray cover , Sa II	pce		3SE630245
20090050	248	Cable for Charging AU/NZ	pce		3EN630237
21090011	263	Extension Cable US	pce		3EN630218
20090058	292	Hook for Hand control 2,4,6	pce		3SE630210
20190057	312	Screw M12x105 U1,U2,Sa	pce		3SE640245
20290029	317	Pivot point actuator Sa II	set		3SE640246
2027010	383	Sabina Lift Arm incl. Locking Handles	set		3SE630264
2107101	384	Locking handles	pair		3SE650248
20290033	400	Star wheel M8	pce		3SE650265
20290034	401	Locking rail & Star wheel M6	set		3SE650251
20190070	402	Bolt Connection	set		3SE650252
20290035	403	Plastic cover	pce		3SE650253

Sabina™ II EE, EM mobile lift

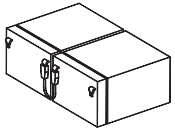
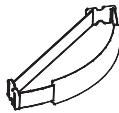
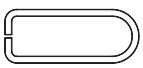
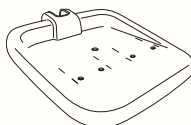

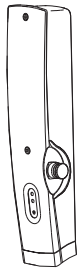
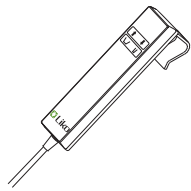
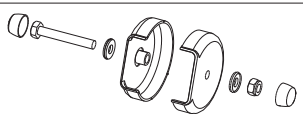



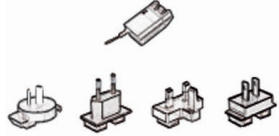
PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20190073	409	Pressure spring	set		3SE650258
20290038	417	Mast Sabina II	pce		3SE650268
20190048	448	Lacquer Top Coat for Improvement, Grey	pce		-
20290042	479	Hook assembly	set		3SE690202
20290043	480	Leg Support II	pce		3SE690205
20290044	481	Leg Support cover, Sabina <i>Sa II to S/N 460799</i>	pce		3SE690203
20290046	482	Sabina Hook Retrofit kit II	set		3SE680204
184010	540	Center Assembly Set <i>Sa II EM from S/N 414443</i>	set		3SE670224

Spare Parts Positions Sabina™ 200 mobile lift

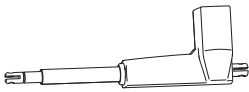
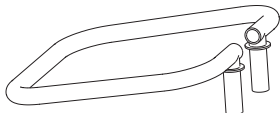



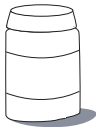
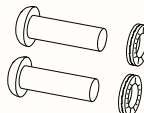
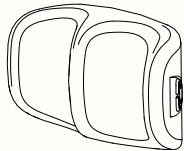
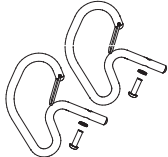
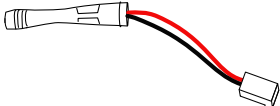

PROD. NO.	PRODUCT
2020020	Sabina 200 (Discontinued 2018)



Sabina™ 200 mobile lift

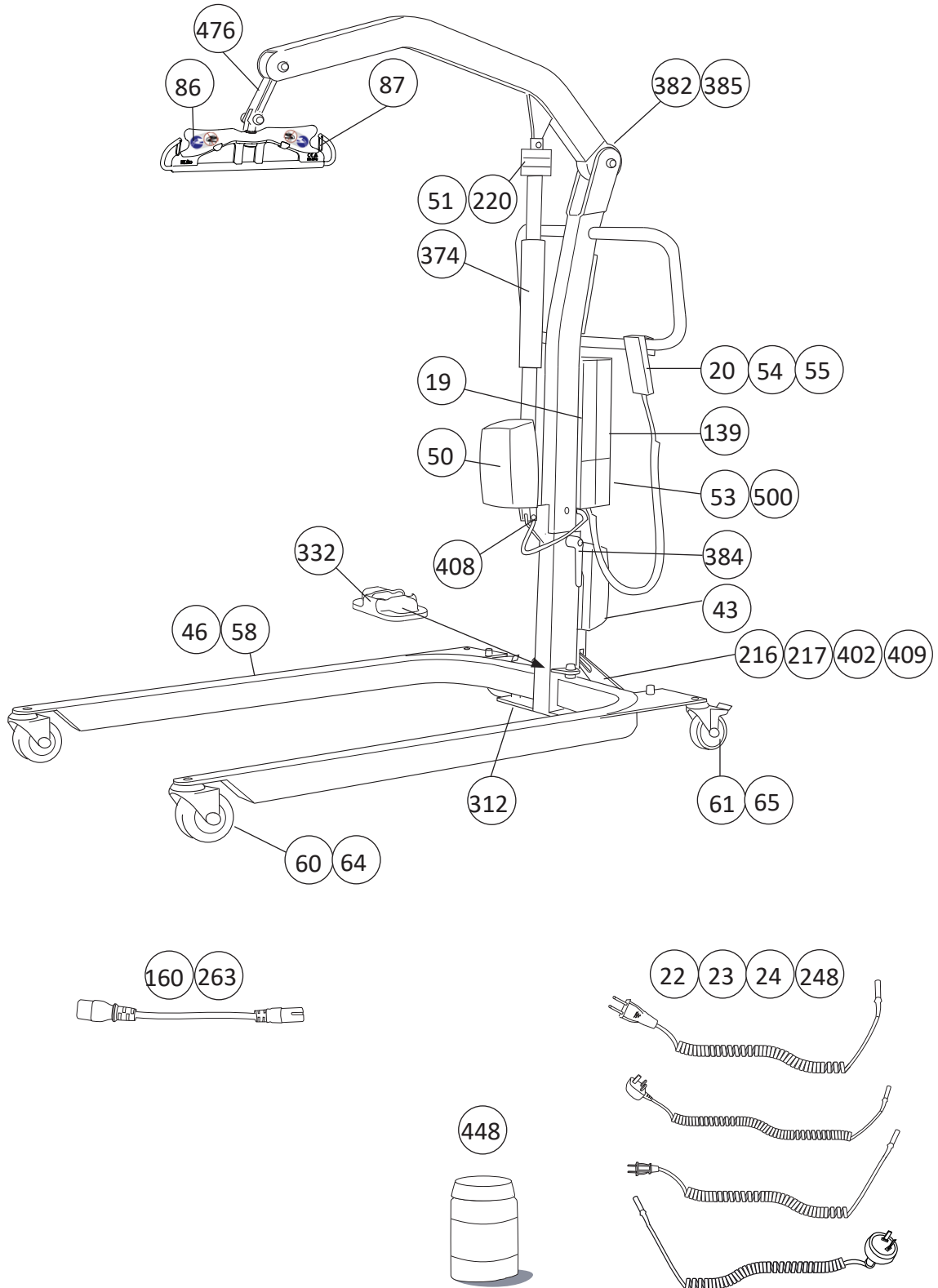
PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20090025	21	Battery 2,9 Ah /12V	pair		3SE380201
20290022	72	Calf Strap <i>From S/N 3501600</i>	pce		-
31190018	87	Latch	pair		3EN640271
20290026	230	Foot Tray cover , Sa II	pce		3SE630245
20090058	292	Hook for Hand control 2,4,6	pce		3SE630210
20190060	356	Control Box Uno 200 <i>batteries included</i>	pce		3SE650223
20190061	357	Hand Control Uno 200	pce		3SE650224
20190062	358	Lift arm, plastic cover	set		3SE650225
20190064	360	Base motor Uno 200	pce		3SE650227
20190065	363	Front wheel 75m	pce		3SE650221
20190066	364	Rear wheel 75mm	pce		3SE650222
21090025	368	Charger CH01	set		3EN650232

Sabina™ 200 mobile lift

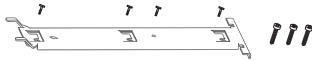
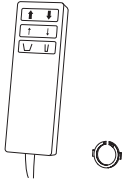


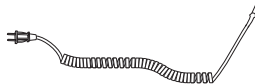
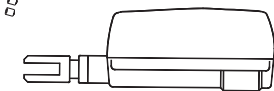
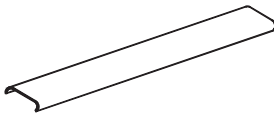
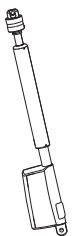
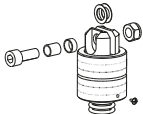
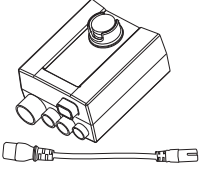
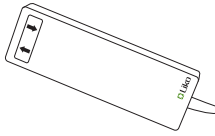

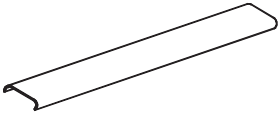
PROD. NO.	POS. NO.	NAME	UNIT	ASSEMBLY INSTR.
20290030	370	Actuator LA31-300	pce	 3SE650234
20290032	372	Foot Frame Sabina 200	pce	 3SE650236
2107101	384	Locking handles	pair	 3SE650248
20290033	400	Star wheel M8	pce	 3SE650265
20290034	401	Locking rail & Star wheel M6	set	 3SE650251
20190048	448	Lacquer Top Coat for Improvement, Grey	pce	 -
20290042	479	Hook assembly	set	 3SE690202
20290043	480	Leg Support II	pce	 3SE690205
20290046	482	Sabina Hook Retrofit kit II	set	 3SE680204
20190079	530	Charger Socket Connection	pce	 3SE690237
2020200SVFI 2020200NODK 2020200ENES 2020200DEFR 2020200FRNL 2020200ITPT		Quick Reference Guide Sabina 200 Swedish/Finnish Norwegian/Danish English/Spanish German/French French/Dutch Italian/Portuguese	pce pce pce pce pce	 -

Spare Parts Positions Uno™ 100/102 mobile lift


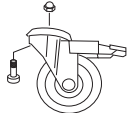
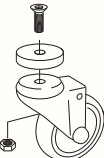
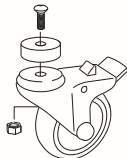

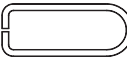
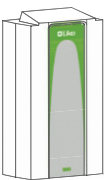

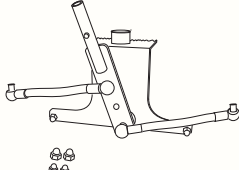
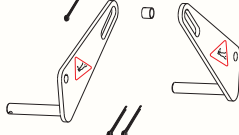
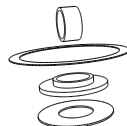
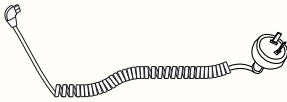

PROD. NO.	PRODUCT	PROD. NO.	PRODUCT
2010011	Uno 100 EE (Discontinued 2013)	2010004	Uno 102 EE up to S/N 51 999 (Discontinued 2013)
2010012	Uno 100 EM (Discontinued 2011)	2010003	Uno 102 EM (Discontinued 2011)
		2010005	Uno 102 ES (Discontinued 2011)



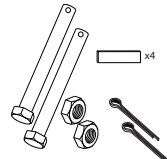

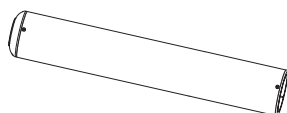
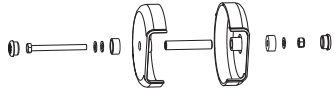

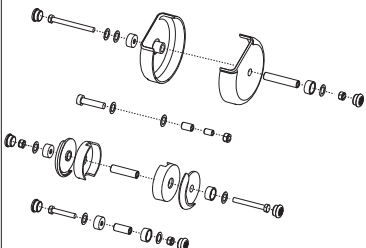
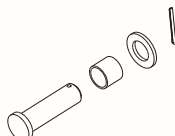

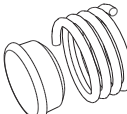

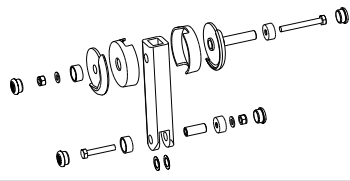
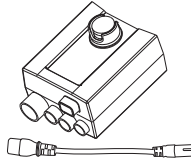
Uno™ 100 EM, EE / 102 EM, EE, ES mobile lifts (up to S/N 51 999)

PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20090023	19	Attachment for Control Box	pce		3EN630238
20090089	20	Hand control 6-Button <i>fits Uno 102 ES</i>	pce		3SE690232
20090026	22	Cable for charging EU	pce		3EN630237
20090027	23	Cable for charging UK	pce		
20090028	24	Cable for charging US	pce		
20190003	43	Actuator for Base Widening	pce		3SE640284
20190006	46	Leg Protector Uno 102	pair		183073
20190010	50	Actuator JKSM 350 G Uno 100/102	pce		3SE630236
20190011	51	Emergency Lowering Device	pce		3SE630230
20190014	53	Control Box Uno 102 ES	pce		3SE630223
20190014US	53	Control Box Uno 102 ES <i>only for the North American market</i>	pce		
20190016	54	Hand control EM 2-Button	pce		3SE630204
20190017	55	Hand Control EE 4-Button	pce		3SE630204
20190029	58	Leg Protector <i>fits Uno 100 and Uno 102 from S/N 52 000</i>	pair		183073

Uno™ 100 EM, EE / 102 EM, EE, ES mobile lifts (up to S/N 51 999)

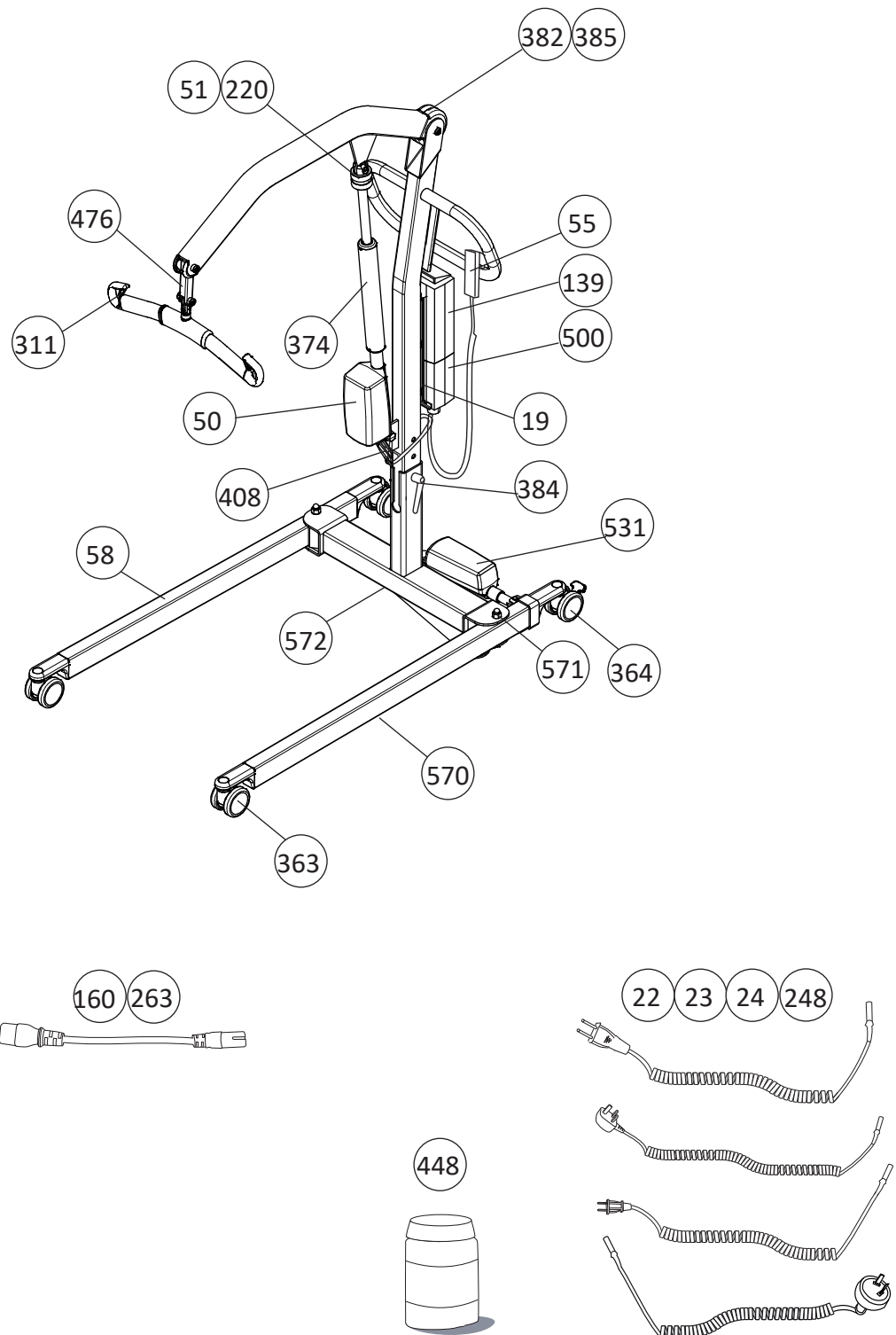
PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20190042	60	Front Wheel 75 mm	pce		3SE640283
20190043	61	Rear Wheel 75 mm	pce		3SE640282
20190046	64	Front Wheel 100 mm	pce		3SE640281
20190047	65	Rear Wheel 100 mm	pce		3SE640285
3156075	86	Universal SlingBar 450	pce		7EN160185
31190018	87	Latch	pair		3EN640271
2006106	139	Battery Lead Acid	pce		3EN630224
21090001	160	Extension Cable	pce		3EN630225
20190049	216	Restoration set Manual Base	set		3SE630226
20190050	217	Restoration set Electrical Base	set		3SE630227
20190051	220	Restoration set Emergency Lowering	set		3SE630222
20090050	248	Cable for Charging AU/NZ	pce		3EN630237
21090011	263	Extension Cable US	pce		3EN630218

Uno™ 100 EM, EE / 102 EM, EE, ES mobile lifts (up to S/N 51 999)

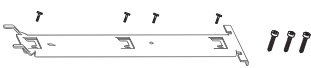
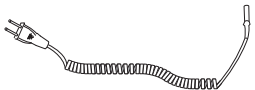

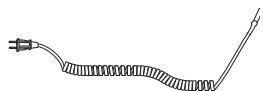
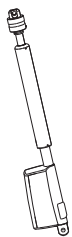
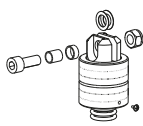

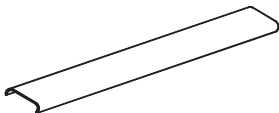
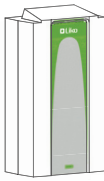

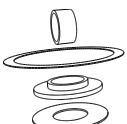

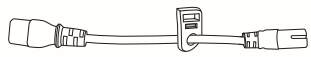
PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20190057	312	Screw M12x105 U1,U2,Sa	pce		3SE640245
20190058	332	Center hood	pce		3SE640261
20390017	374	Outer Tube LA32, 310 mm	pce		3SE650237
20190068	382	Lift Arm Plastic Cover set	set		3SE650239
2107101	384	Locking handles	pair		3SE650248
20190069	385	Overhead Beam Restoration	set		3SE650240
20190070	402	Bolt Connection	set		3SE650252
20190072	408	Lower Actuator attachment	set		3SE650257
20190073	409	Pressure spring	pce		3SE650258
20190048	448	Lacquer Top Coat for Improvement, Grey	pce		-
20190074	476	Flexlink Alu	set		3SE650297
20190076	500	Control Box Uno 100/102 EE, EM	pce		3SE630223
20190076US	500	Control Box Uno 100/102 EE, EM <i>only for the North American market</i>	pce		

Spare Parts Positions Uno™ 102 mobile lift


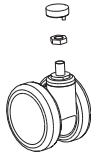

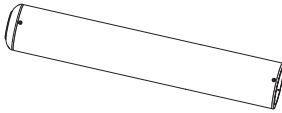
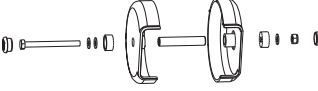

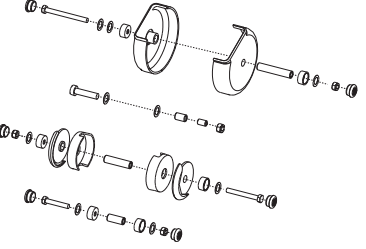
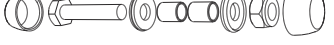
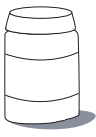
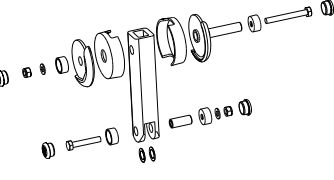
PROD. NO.	PRODUCT
2010004	Uno 102 EE, Rev 01 (from S/N 52 000)



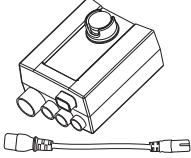
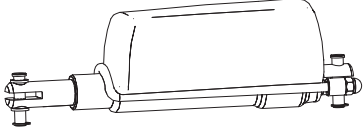
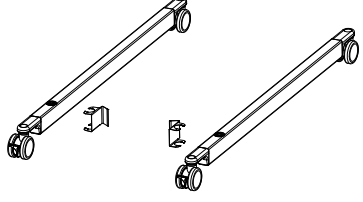
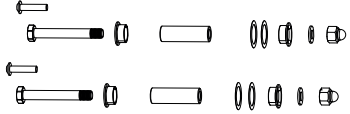

Uno™ 102 EE, Rev 01 mobile lift (from S/N 52 000)

PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20090023	19	Attachment for Control Box	pce		3EN630238
20090026	22	Cable for charging EU	pce		3EN630237
20090027	23	Cable for charging UK	pce		
20090028	24	Cable for charging US	pce		
20190010	50	Actuator JKSM 350 G U1-2	pce		3SE630236
20190011	51	Emergency Lowering Device	pce		3SE630206
20190017	55	Hand Control EE 4-Button	pce		3SE630204
20190029	58	Leg Protector <i>fits Uno 100 and Uno 102 from S/N 52 000</i>	pair		183073
2006106	139	Battery Lead Acid	pce		3EN630224
21090001	160	Extension Cable	pce		3EN630225
20190051	220	Restoration set Emergency Lowering	set		3SE630222
20090050	248	Cable for Charging AU/NZ	pce		3EN630237
21090011	263	Extension Cable US	pce		3EN630218

Uno™ 102 EE, Rev 01 mobile lift (from S/N 52 000)

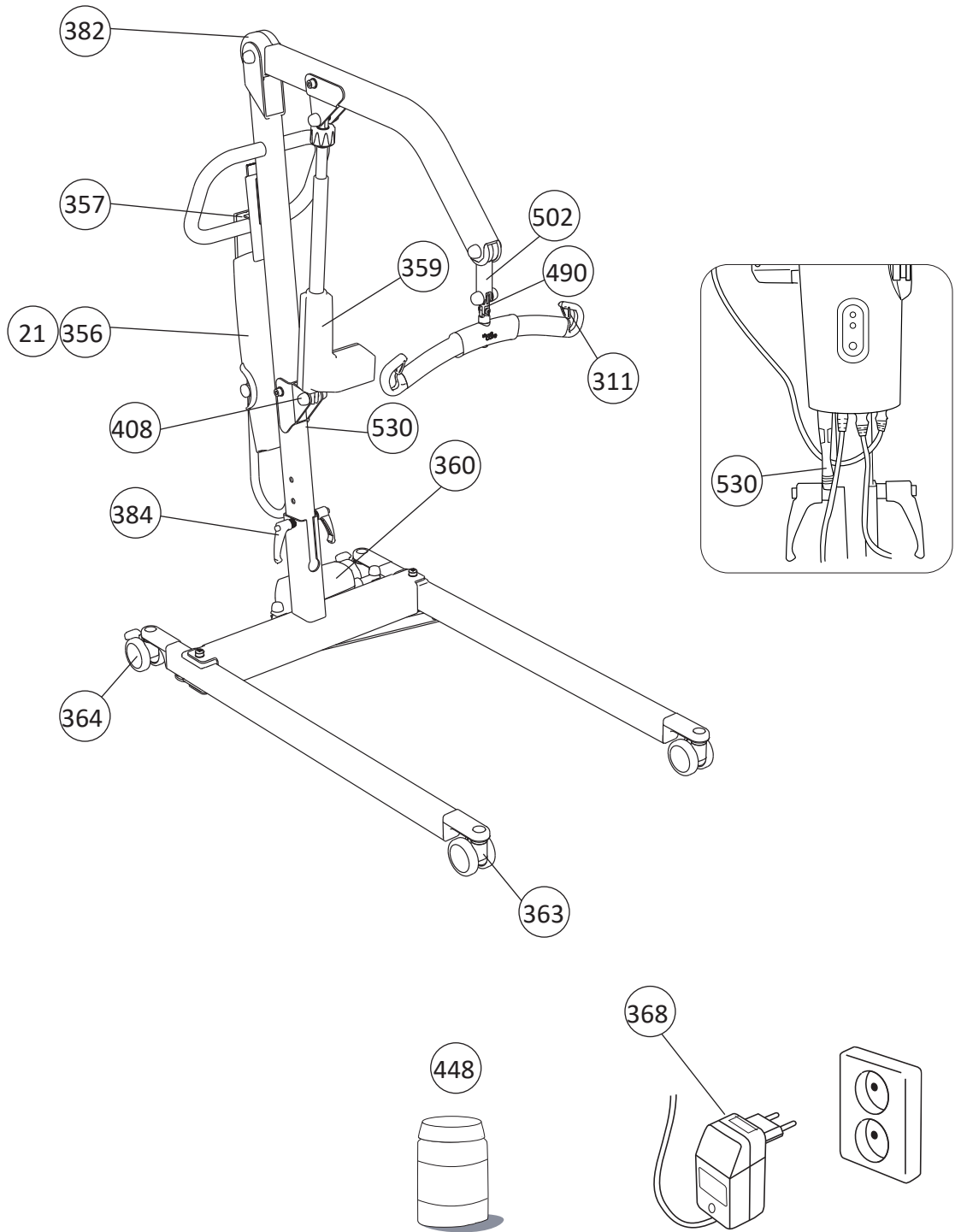
PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
31590014	311	Latch, composite <i>Fits Universal SlingBar from S/N 1299511</i>	pair		3SE640242
20190065	363	Front wheel 75m	pce		3SE650221
20190066	364	Rear wheel 75mm	pce		3SE650222
20390017	374	Outer Tube LA32, 310 mm	pce		3SE650237
20190068	382	Lift Arm Plastic Cover set	set		3SE650239
2107101	384	Locking handles	pair		3SE650248
20190069	385	Overhead Beam Restoration	set		3SE650240
20190072	408	Lower Actuator attachment	set		3SE650257
20190048	448	Lacquer Top Coat for Improvement, Grey	pce		-
20190074	476	Flexlink Alu	set		3SE650297

Uno™ 102 EE, Rev 01 mobile lift (from S/N 52 000)

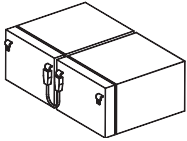

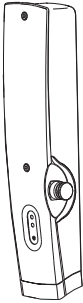
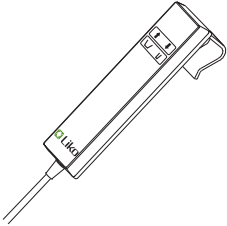
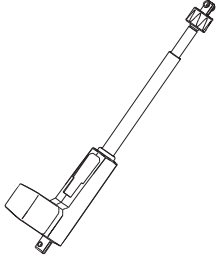
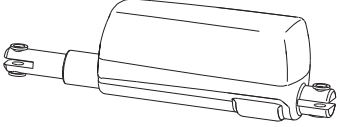
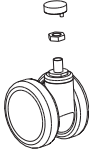
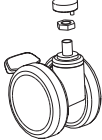
PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20190076	500	Control Box Uno 100/102 EE, EM	pce		3SE630223
20190076US	500	Control Box Uno 100/102 EE, EM <i>only for the North American market</i>	pce		
20190080	531	Base Motor II	pce		182224
20190081	570	Legs Uno 102	pair		3EN690293
20190082	571	Base Screw Assembly Uno 102	set		3EN690294
20190083	572	Diagonal Stopper	set		3EN690295

Spare Parts Positions Uno™ 200 mobile lift

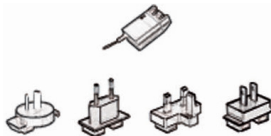
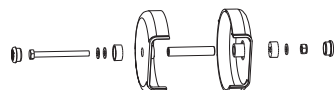


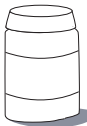
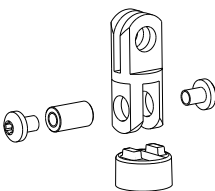
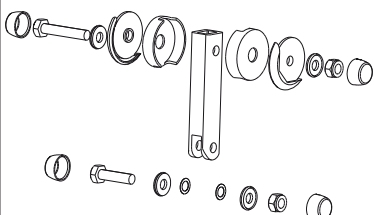
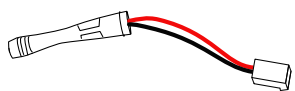
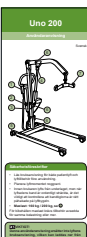
PROD. NO.	PRODUCT
2010020	Uno 200 (Discontinued 2014)



Uno™ 200 mobile lift


PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20090025	21	Battery 2,9 Ah /12V	pair		3SE380201
31590014	311	Latch, composite <i>Fits Universal SlingBar from S/N 1299511</i>	pair		3SE640242
20190060	356	Control Box Uno 200 <i>batteries included</i>	pce		3SE650223
20190061	357	Hand Control Uno 200	pce		3SE650224
20190063	359	Actuator LA31	pce		3SE650226
20190064	360	Base motor Uno 200	pce		3SE650227
20190065	363	Front wheel 75m	pce		3SE650221
20190066	364	Rear wheel 75mm	pce		3SE650222

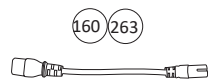
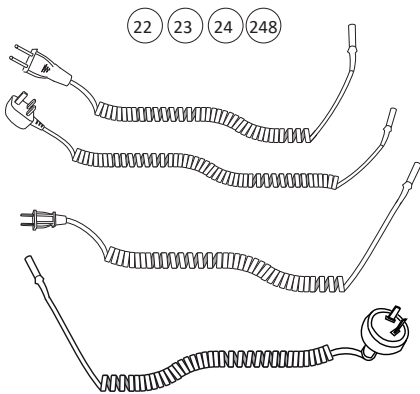
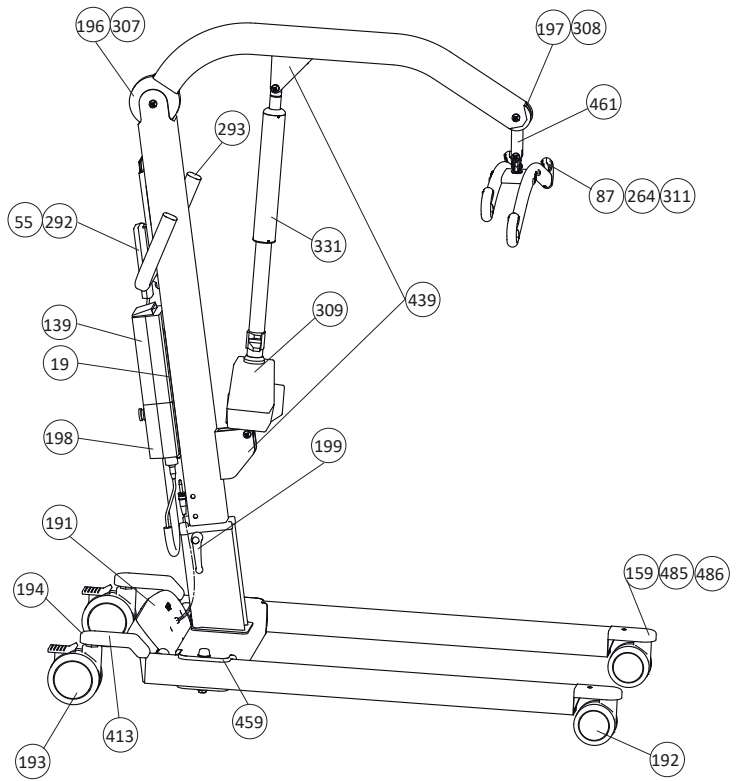
Uno™ 200 mobile lift

PROD. NO.	POS. NO.	NAME	UNIT	ASSEMBLY INSTR.
21090025	368	Charger CH01	set	 3EN650232
20190068	382	Lift Arm Plastic Cover set	set	 3SE650239
2107101	384	Locking handles	pair	 3SE650248
20190072	408	Lower Actuator attachment	set	 3SE650257
20190048	448	Lacquer Top Coat for Improvement , Grey	pce	 -
20490052	490	SlingBar Assembly set	set	 3SE690210
20190071	502	Flexlink Uno 200	set	 3SE690211
20190079	530	Charger Socket Connection	pce	 3SE690237
2010200SVFI 2010200NODK 2010200ENES 2010200DEFR 2010200FRNL 2010200ITPT		Quick Reference Guide Uno 200 Swedish/Finnish Norwegian/Danish English/Spanish German/French French/Dutch Italian/Portuguese	pce pce pce pce pce pce	

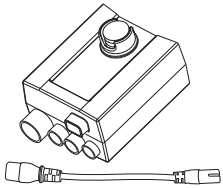
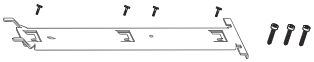


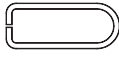
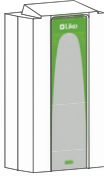
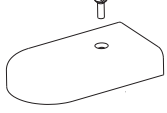

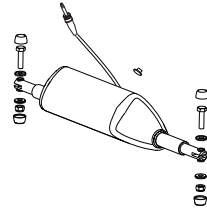
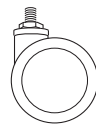

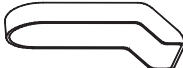
Spare Parts Positions Viking™ XL mobile lift

PROD. NO.	PRODUCT
2040003	Viking XL (Discontinued 2012) S/N: 800 001- 890 000

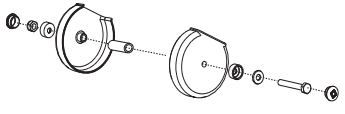
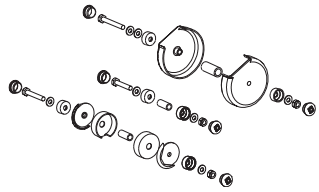
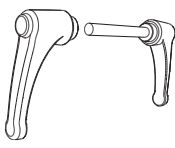

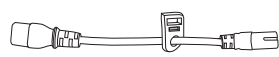
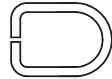


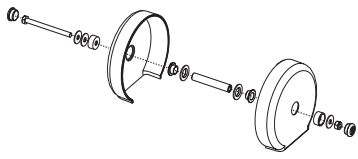
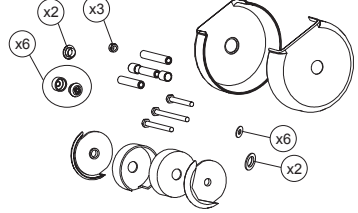
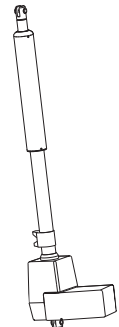
 There are four models of Viking™ XL mobile lift; Prod. No. 2040003, 2040013, 2040033, 2040043. One spare parts list for each model. Make sure to order correct spare part.




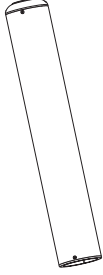
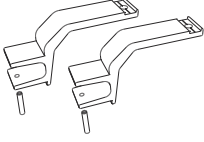
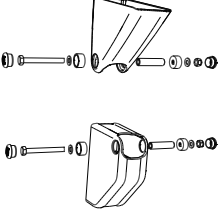
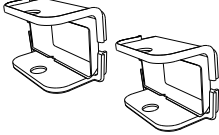
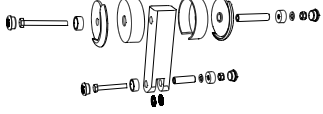
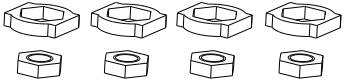
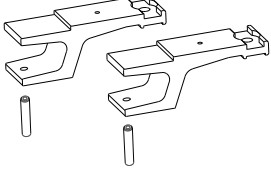
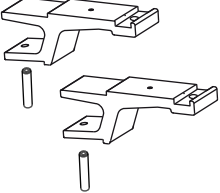
Viking™ XL mobile lift, 2040003

PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20490009	198	Control Box Viking L/XL	pce		3SE630223
20490009US	198	Control Box Viking L/XL, US <i>only for the US market</i>	pce		
20090023	19	Attachment for Control Box	pce		3EN630238
20090026	22	Cable for Charging, EU	pce		3EN630237
20090027	23	Cable for Charging, UK	pce		
20090028	24	Cable for Charging, US	pce		
20190017	55	Hand Control EE 4-Button	pce		3SE630204
31190018	87	Latch	pair		3EN640271
2006106	139	Battery Lead Acid	pce		3EN630224
20390006	159	Fork Cover	pair		183916
21090001	160	Extension Cable	pce		3EN630225
20490002	191	Base Motor Viking XL complete	pce		3SE640291
20490003	192	Front Wheel 100 mm <i>We recommend replacing both wheels, as there are two different versions that can vary in appearance. (dimension is the same)</i>	pce		3SE630293
20490004	193	Rear Wheel 125 mm <i>We recommend replacing both wheels, as there are two different versions that can vary in appearance. (dimension is the same)</i>	pce		3SE630294
20490005	194	Rear Fork Cover	pair		3SE630295

Viking™ XL mobile lift, 2040003


PROD. NO.	POS. NO.	NAME	UNIT	ASSEMBLY INSTR.	
20490007	196	Lift Arm Plastic Cover <i>S/N 801196-801999</i>	set		3SE630240
20490008	197	Overhead Beam Restoration set <i>S/N 801196-801999</i>	set		3SE630228
20490010	199	Locking Handle Viking XL <i>from S/N 800987</i>	pair		3SE630244
20090050	248	Cable for Charging AU/NZ	pce		3EN630237
21090011	263	Extension Cable US	pce		3EN630218
31590011	264	Latch Universal SlingBar	pair		3SE630229
20090058	292	Hook for Hand control 2,4,6	pce		3SE630210
20090066	293	End Cover Handle/Arm Support grey, 4 pce	set		3SE630271
20490023	307	Lift arm plastic cover <i>Viking XL 2040003: fr. S/N 802000</i>	set		3EN630250
20490024	308	Overhead beam restoration set <i>Viking XL 2040003: fr. S/N 802000</i>	set		3EN630251
20490025	309	Actuator Viking XL	pce		3SE640241

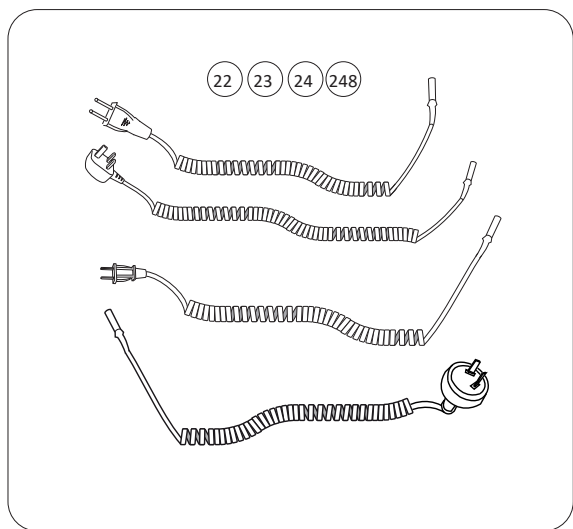
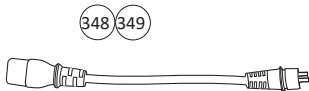
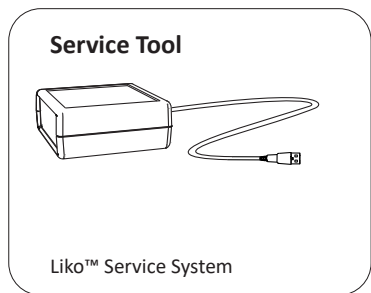
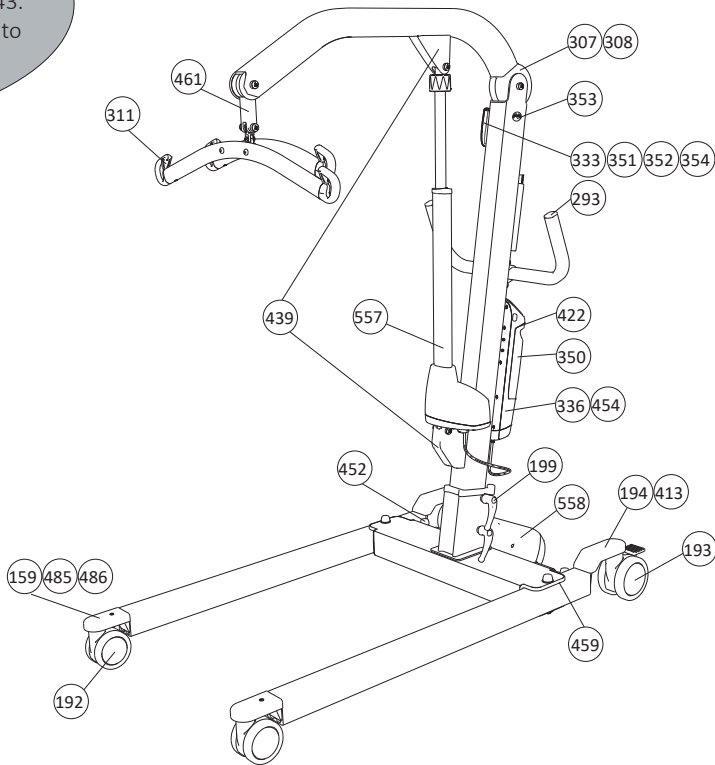
Viking™ XL mobile lift, 2040003

PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
31590014	311	Latch, composite <i>Fits Universal SlingBar from S/N 1299511</i>	pair		3SE640242
20490030	331	Outer Tube	pce		3SE640260
20490037	413	Rear Wheel fork	pair		3SE650264
20490040	439	Bracket Cover	set		3SE650273
20490047	459	Leg Cover L&XL	set		3EN650294
20490049	461	Flexlink	set		3SE650295
20090080	485	Fork locking attachment, 4 pair	set		3SE690208
20090081	486	Wheel Fork	pair		3SE690209
20490060	512	Front Wheel Fork <i>Fits Viking XL 2040003L</i>	pair		3SE690221



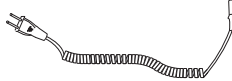


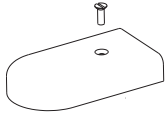



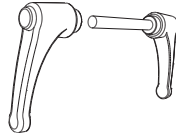
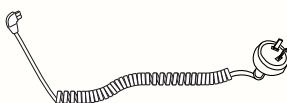

Spare Parts Positions Viking™ XL mobile lift

PROD. NO.	PRODUCT
2040013	Viking XL intelligent control box (Discontinued 2017) S/N: 7 900 001- 8 000 000

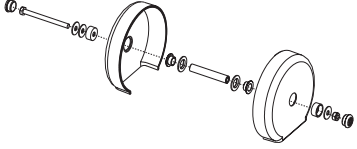
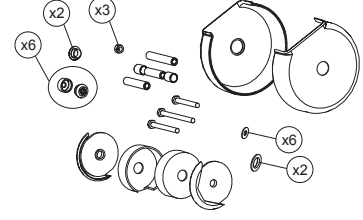









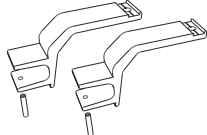
 There are four models of Viking™ XL mobile lift; Prod. No. 2040003, 2040013, 2040033, 2040043. One spare parts list for each model. Make sure to order correct spare part.




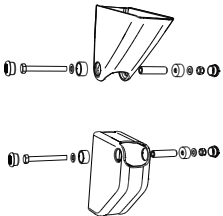
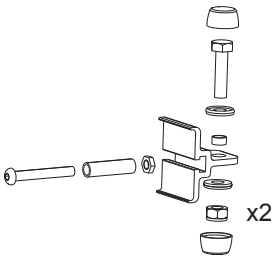
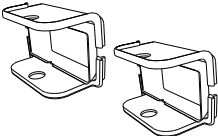
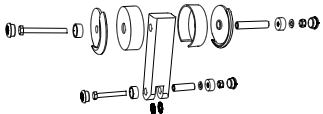
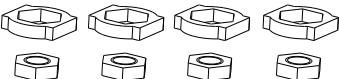
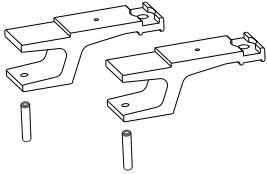
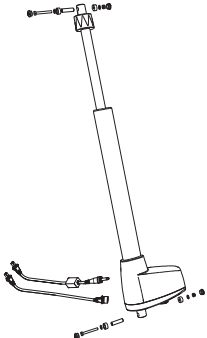
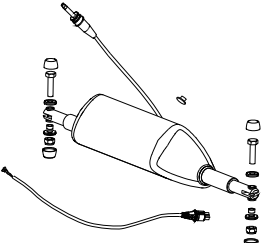
Viking™ XL mobile lift, 2040013

PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20490039	336	Control Unit CBL, by S/N <i>Can only be ordered if S/N (serial number) can be provided to Liko™ Customer Service. Will be programmed at Liko™ before delivery.</i> <i>2 weeks delivery time from Hill-Rom/Liko™ Alvik, Sweden</i>	pce		3SE650270
20490044	454	Control Unit CBL Viking XL <i>Before use; requires Software and, needs to be programmed with Liko™ Service System, Prod. No. 21090024</i>	pce		3SE650290
20090026	22	Cable for Charging, EU	pce		3EN630237
20090027	23	Cable for Charging, UK	pce		
20090028	24	Cable for Charging, US	pce		
20390006	159	Fork Cover	pair		183916
20490003	192	Front Wheel 100 mm <i>We recommend replacing both wheels, as there are two different versions that can vary in appearance. (dimension is the same)</i>	pce		3SE630293
20490004	193	Rear Wheel 125 mm <i>We recommend replacing both wheels, as there are two different versions that can vary in appearance. (dimension is the same)</i>	pce		3SE630294
20490005	194	Rear Fork Cover	pair		3SE630295
20490010	199	Locking Handle Viking XL	pair		3SE630244
20090050	248	Cable for Charging AU/NZ	pce		3EN630237
20090066	293	End Cover Handle/Arm Support grey, 4 pce	set		3SE630271

Viking™ XL mobile lift, 2040013


PROD. NO.	POS. NO.	NAME	UNIT	ASSEMBLY INSTR.
20490023	307	Lift arm plastic cover	set	 3EN630250
20490024	308	Overhead beam restoration set	set	 3EN630251
31590014	311	Latch, composite <i>Fits Universal SlingBar from S/N 1299511</i>	pair	 3SE640242
20490031	333	Wireless hand control HBL20	set	 3SE640298
21090018	348	Extension cable CBL20	pce	 3SE650216
21090019	349	Extension cable CBL 20 US,	pce	 3SE650216
2006107	350	Extra battery box	pce	 -
21090020	351	Beltclip HBL20	pair	 3SE650217
21090021	352	Clips with hook HBL20	pair	 3SE650218
21090022	353	Charge cable hook , 10 pce	set	 3SE650219
21090023	354	Hand control hanger, 10 pce	set	 3SE650220
20490037	413	Rear Wheel fork	pair	 3SE690221

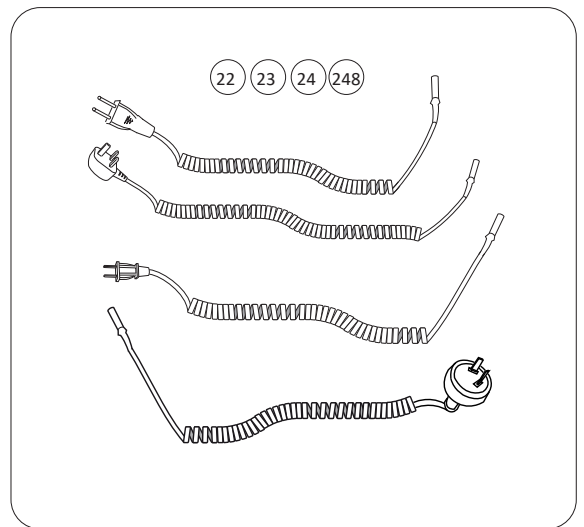
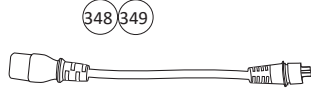
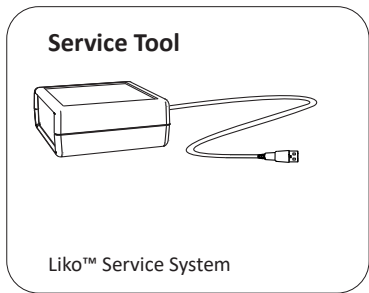
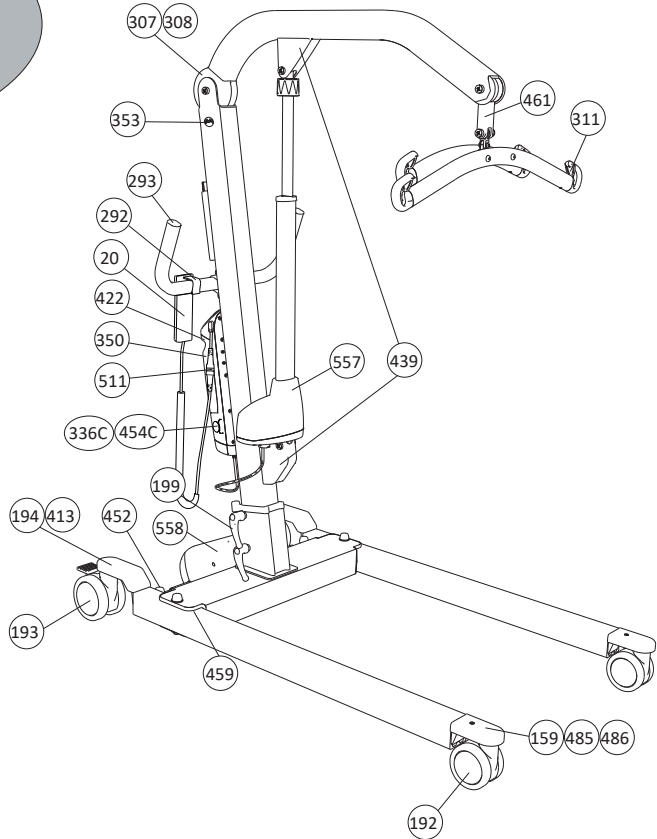
Viking™ XL mobile lift, 2040013

PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20090077	422	Locking device CBL20, 10 pce	set		3SE640217
20490040	439	Bracket Cover	set		3SE650273
20490042	452	Restoration set motor attachment	set		3SE650288
20490047	459	Leg Cover L&XL	set		3EN650294
20490049	461	Flexlink	set		3SE650295
20090080	485	Fork locking attachment, 4 pair	set		3SE690208
20090081	486	Wheel Fork	pair		3SE690209
20490065	557	Actuator LA44	set		3EN690287
20490066	558	Base Motor, wide	pce		3SE650293

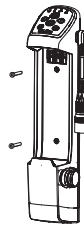
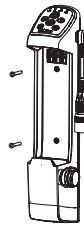
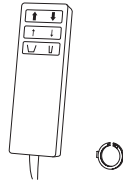
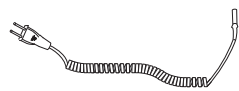


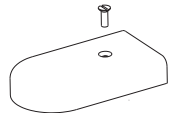


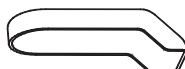
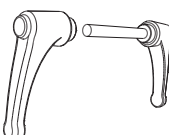
Spare Parts Positions Viking™ XL mobile lift

PROD. NO.	PRODUCT
2040033	Viking XL intelligent control box and cable hand control (<i>Discontinued 2017</i>) S/N: 9 100 001- 9 200 000


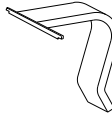

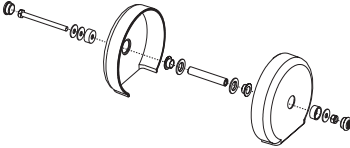
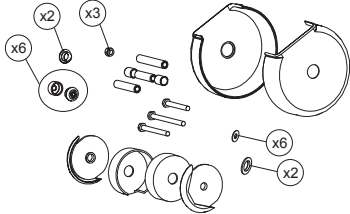



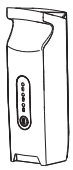

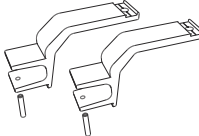

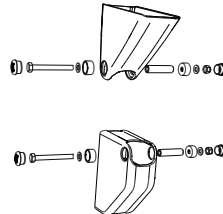
 There are four models of Viking™ XL mobile lift; Prod. No. 2040003, 2040013, 2040033, 2040043. One spare parts list for each model. Make sure to order correct spare part.



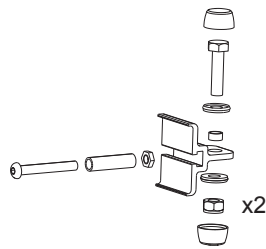
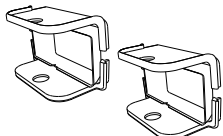
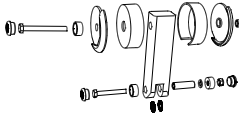
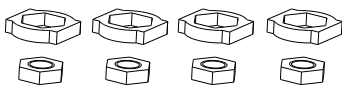
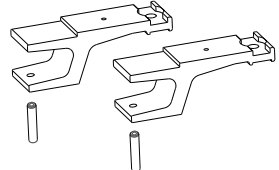
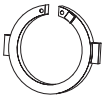
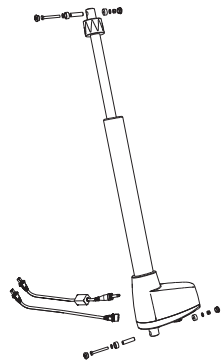
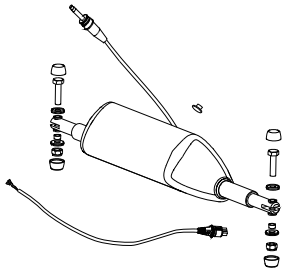
Viking™ XL mobile lift, 2040033

PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20490054	336C	Control Unit CBL by S/N, Cable <i>Can only be ordered if S/N (serial number) can be provided to Liko™ Customer Service. Will be programmed at Liko™ before delivery. 2 weeks delivery time from Hill-Rom/Liko™ Alvik, Sweden</i>	pce		3SE690227
20490056	454C	Control Unit CBL Viking XL, Cable <i>Before use; requires Software and, needs to be programmed with Liko™ Service System, Prod. No. 21090024</i>	pce		3SE690229
20090089	20	Hand control 6-Button	pce		3SE690232
20090026	22	Cable for Charging, EU	pce		3EN630237
20090027	23	Cable for Charging, UK	pce		
20090028	24	Cable for Charging, US	pce		
20390006	159	Fork Cover	pair		3EN380202
20490003	192	Front Wheel 100 mm <i>We recommend replacing both wheels, as there are two different versions that can vary in appearance. (dimension is the same)</i>	pce		3SE630293
20490004	193	Rear Wheel 125 mm <i>We recommend replacing both wheels, as there are two different versions that can vary in appearance. (dimension is the same)</i>	pce		3SE630294
20490005	194	Rear Fork Cover	pair		3SE630295
20490010	199	Locking Handle Viking XL	pair		3SE630244


Viking™ XL mobile lift, 2040033

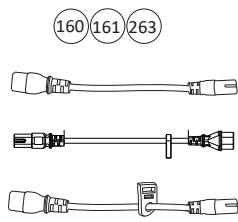
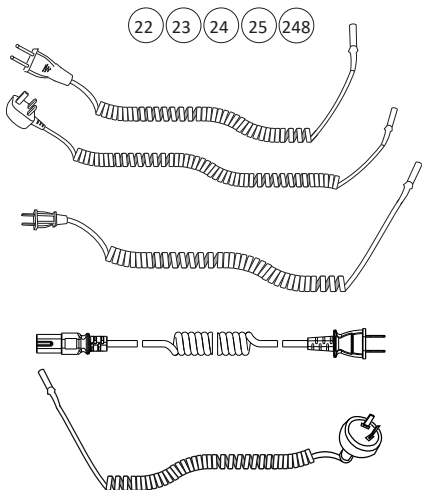
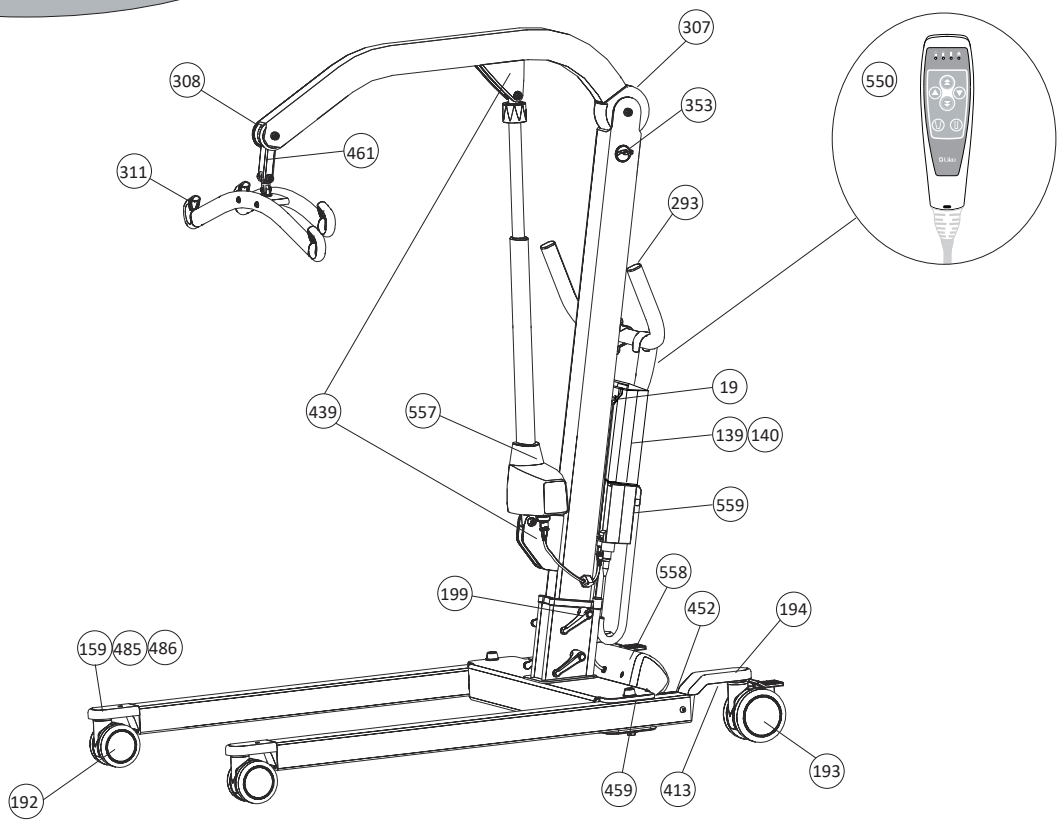
PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20090050	248	Cable for Charging AU/NZ	pce		3EN630237
20090058	292	Hook for Hand control 2,4,6	pce		3SE630210
20090066	293	End Cover Handle/Arm Support grey, 4 pce	set		3SE630271
20490023	307	Lift arm plastic cover	set		3EN630250
20490024	308	Overhead beam restoration set	set		3EN630251
31590014	311	Latch, composite <i>Fits Universal SlingBar from S/N 1299511</i>	pair		3SE640242
21090018	348	Extension cable CBL20	pce		3SE650216
21090019	349	Extension cable CBL 20 US,	pce		
2006107	350	Extra battery box	pce		-
21090022	353	Charge cable hook , 10 pce	set		3SE650219
20490037	413	Rear Wheel fork	pair		3SE690221
20090077	422	Locking device CBL20, 10 pce	set		3SE640217
20490040	439	Bracket Cover	set		3SE650273

Viking™ XL mobile lift, 2040033


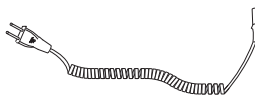



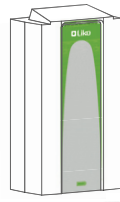
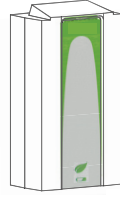
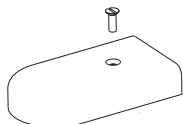

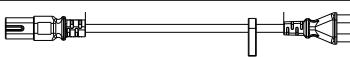
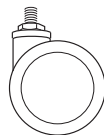

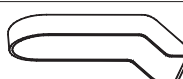
PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20490042	452	Restoration set motor attachment	set		3SE650288
20490047	459	Leg Cover L&XL	set		3EN650294
20490049	461	Flexlink	set		3SE650295
20090080	485	Fork locking attachment, 4 pair	set		3SE690208
20090081	486	Wheel Fork	pair		3SE690209
20490057	511	Locking Ring CBL, 10 pce	set		3SE690231
20490065	557	Actuator LA44 <i>Replaces 20490045</i>	set		3EN690287
20490066	558	Base Motor, wide <i>Replaces 20490046</i>	pce		3SE650293

PROD. NO.	PRODUCT
2040043	Viking XL Jumbo care control box and LED hand control S/N: 9 600 001- 9 700 000

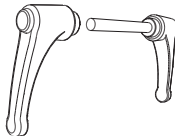
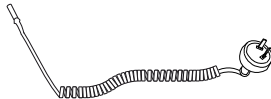
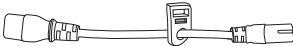

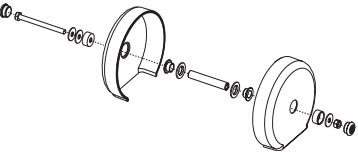
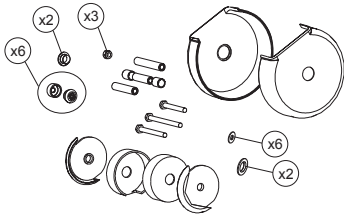


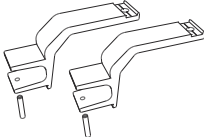
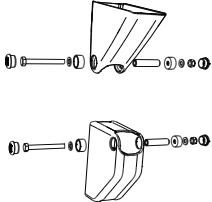
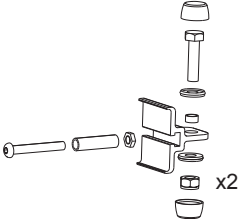
 There are four models of Viking™ XL mobile lift; Prod. No. 2040003, 2040013, 2040033, 2040043. One spare parts list for each model. Make sure to order correct spare part.



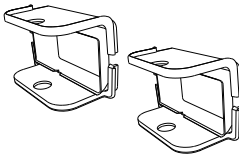
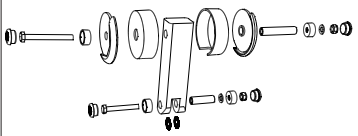
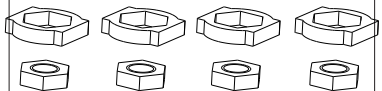
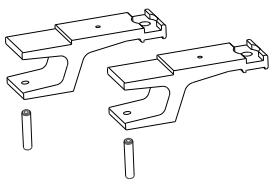

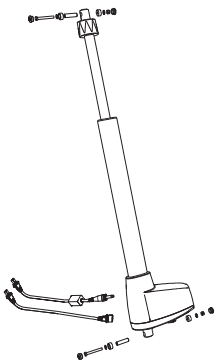
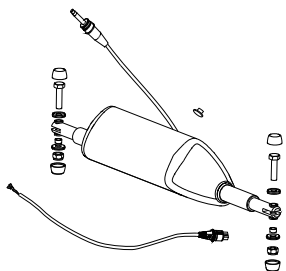
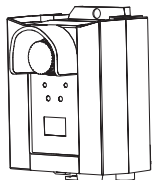
Viking™ XL mobile lift, 2040043

PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20090023	19	Attachment for Control Box	pce		3EN630238
20090026	22	Cable for Charging, EU	pce		3SE630237
20090027	23	Cable for Charging, UK	pce		
20090028	24	Cable for Charging, US	pce		
20090029	25	Cable for charging JP	pce		
2006106	139	Battery Lead Acid	pce		3EN630224
2006110	140	Battery Lithium Ion	pce		3EN690296
20390006	159	Fork Cover	pair		3SE380202
21090001	160	Extension Cable	pce		3EN630225
21090000	161	Extension Cable JP	pce		3EN690280
20490003	192	Front Wheel 100 mm <i>We recommend replacing both wheels, as there are two different versions that can vary in appearance. (dimensions are the same)</i>	pce		3SE630293
20490004	193	Rear Wheel 125 mm <i>We recommend replacing both wheels, as there are two different versions that can vary in appearance. (dimensions are the same)</i>	pce		3SE630294
20490005	194	Rear Fork Cover	pair		3SE630295

Viking™ XL mobile lift, 2040043

PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20490010	199	Locking Handle Viking XL	pair		3SE630244
20090050	248	Cable for Charging, AU/NZ	pce		3SE630237
21090011	263	Extension Cable US	pce		3SE650216
20090066	293	End Cover Handle/Arm Support grey, 4 pce	set		3SE630271
20490023	307	Lift arm plastic cover	set		3EN630250
20490024	308	Overhead beam restoration set	set		3EN630251
31590014	311	Latch, composite <i>Fits Universal SlingBar from S/N 1299511</i>	pair		3SE640242
21090022	353	Charge cable hook , 10 pce	set		3SE650219
20490037	413	Rear Wheel fork	pair		3SE690221
20490040	439	Bracket Cover	set		3SE650273
20490042	452	Restoration set motor attachment	set		3SE650288

Viking™ XL mobile lift, 2040043

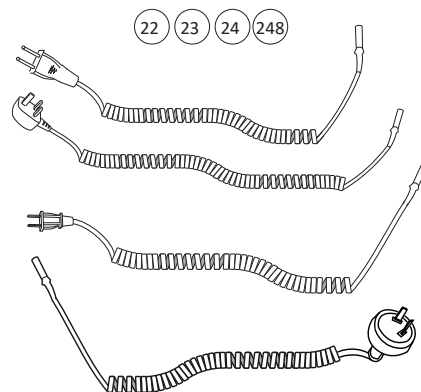
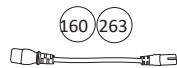
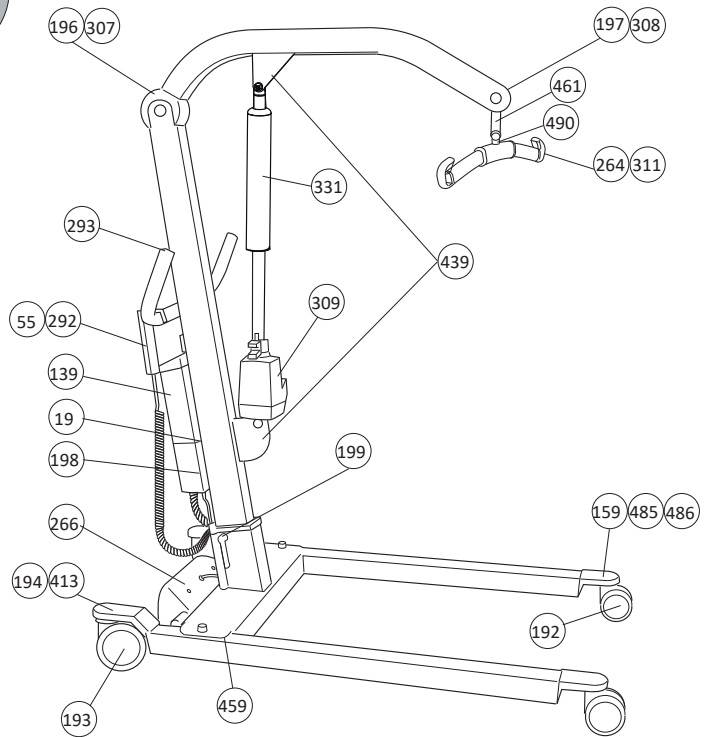
PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20490047	459	Leg Cover L&XL	set		3EN650294
20490049	461	Flexlink	set		3SE650295
20090080	485	Fork locking attachment, 4 pair	set		3SE690208
20090081	486	Wheel Fork	pair		3SE690209
20090098	550	Hand Control 6 button, LED	pce		3EN690283
20490065	557	Actuator LA44 <i>Replaces 20490045</i>	set		3EN690287
20490066	558	Base Motor, wide <i>Replaces 20490046</i>	pce		3SE650293
20490067	559	Control Box XL Jumbo Care	pce		3EN690289

Spare Parts Positions Viking™ L mobile lift

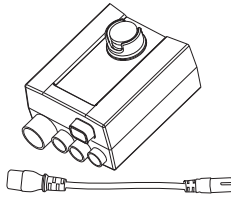

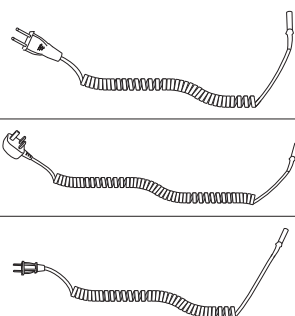

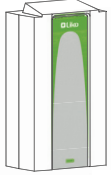
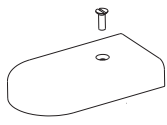

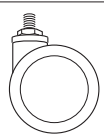
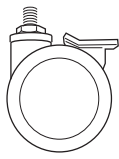

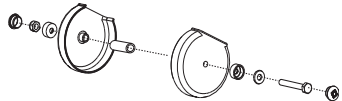
PROD. NO.	PRODUCT
2040004	Viking L (Discontinued 2012) S/N 7 200 201- 7 300 300



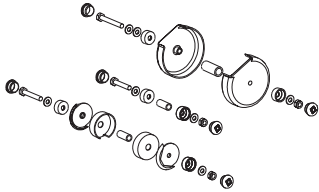
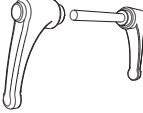



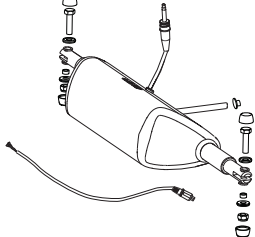


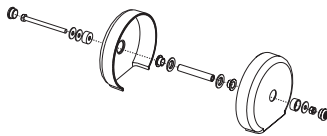
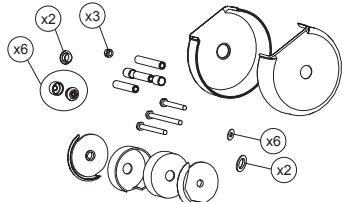
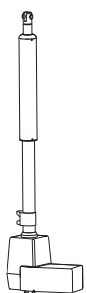
There are four models of Viking™ L mobile lift; Prod. No. 2040004, 2040014, 2040034, 2040044. One spare parts list for each model. Make sure to order correct spare part.





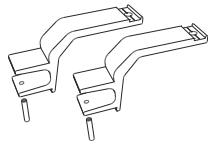
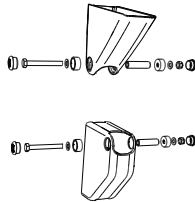
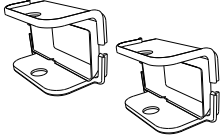
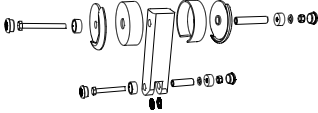
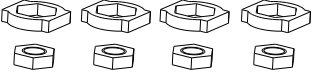
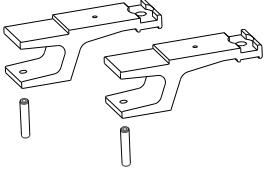
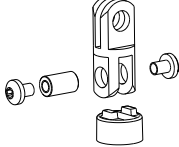
Viking™ L mobile lift, 2040004

PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20490009	198	Control Box Viking L/XL	pce		3SE630223
20490009US	198	Control Box Viking L/XL, US <i>only for the US-market</i>	pce		
20090023	19	Attachment for Control Box	pce		3EN630238
20090026	22	Cable for Charging, EU	pce		3EN630237
20090027	23	Cable for Charging, UK	pce		
20090028	24	Cable for Charging, US	pce		
20190017	55	Hand Control EE 4-Button	pce		3SE630204
2006106	139	Battery Lead Acid	pce		3EN630224
20390006	159	Fork Cover	pair		183916
21090001	160	Extension Cable	pce		3EN630225
20490003	192	Front Wheel 100 mm <i>We recommend replacing both wheels, as there are two different versions that can vary in appearance. (dimension is the same)</i>	pce		3SE630293
20490004	193	Rear Wheel 125 mm <i>We recommend replacing both wheels, as there are two different versions that can vary in appearance. (dimension is the same)</i>	pce		3SE630294
20490005	194	Rear Fork Cover	pair		3SE630295
20490007	196	Lift Arm Plastic Cover Vi <i>To S/N 7200488.</i>	set		3SE630240

Viking™ L mobile lift, 2040004

PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20490008	197	Overhead Beam Restoration set <i>To S/N 7200488</i>	set		3SE630228
20490010	199	Locking Handle Viking XL <i>from S/N 7200431</i>	pair		3SE630244
20090050	248	Cable for Charging AU/NZ	pce		3EN630237
21090011	263	Extension Cable US	pce		3EN630218
31590011	264	Latch Universal SlingBar	pair		3SE630229
20490011	266	Base Motor Viking M, L	pce		3EN630296
20090058	292	Hook for Hand control 2, 4, 6	pce		3SE630210
20090066	293	End Cover Handle/Arm Support grey, 4 pce	set		3SE630271
20490023	307	Lift arm plastic cover <i>Prod. No. 2040004, from S/N 7200795</i>	set		3EN630250
20490024	308	Overhead beam restoration set <i>Prod. No. 2040004, from S/N 7200795</i>	set		3EN630251
20490025	309	Actuator Viking L, XL	pce		3SE640241

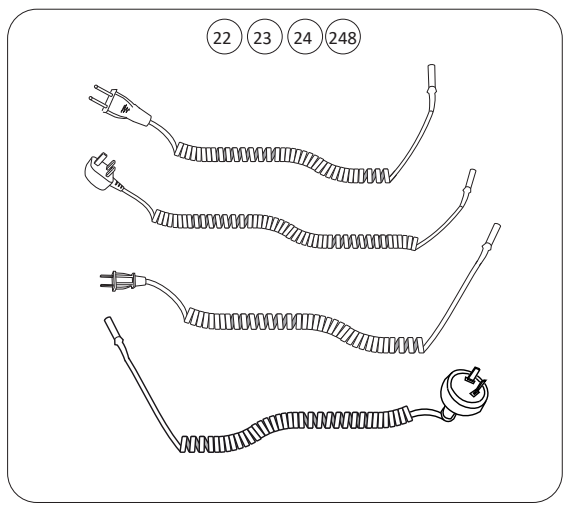
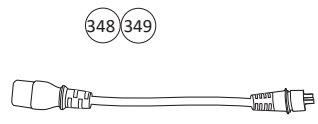
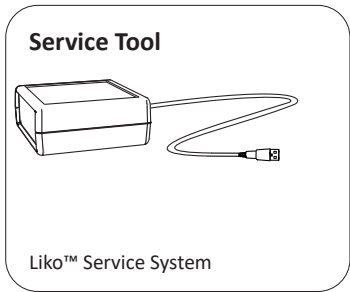
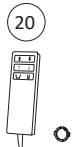
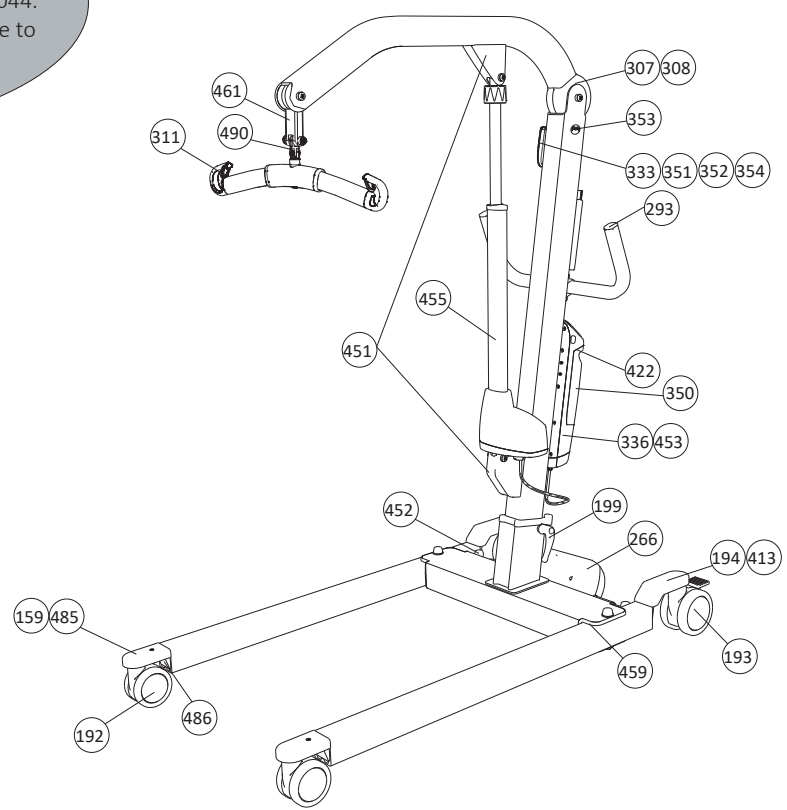
Viking™ L mobile lift, 2040004

PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
31590014	311	Latch, composite <i>Fits Universal SlingBar from S/N 1299511</i>	pair		3SE640242
20490030	331	Outer Tube	pce		3SE640260
20490037	413	Rear Wheel fork	pair		3SE690221
20490040	439	Bracket Cover	set		3SE650273
20490047	459	Leg Cover L&XL	set		3EN650294
20490049	461	Flexlink	set		3SE650295
20090080	485	Fork locking attachment, 4 pair	set		3SE690208
20090081	486	Wheel Fork	pair		3SE690209
20490052	490	SlingBar Assembly set	set		3SE690210



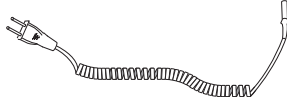
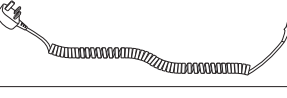
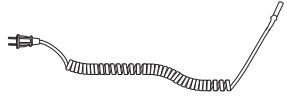
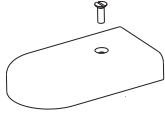

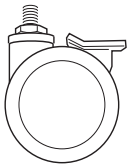

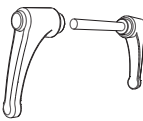

Spare Parts Positions Viking™ L mobile lift

PROD. NO.	PRODUCT
2040014	Viking L (with intelligent control box) <i>(Discontinued 2017)</i> (S/N 7 800 001- 7 900 000)

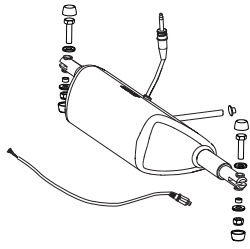

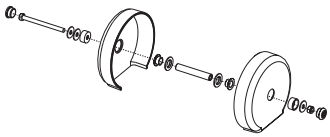
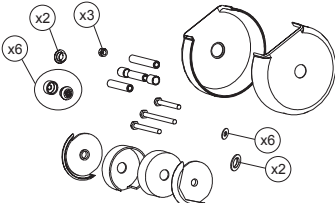









! There are four models of Viking™ L mobile lift; Prod. No. 2040004, 2040014, 2040034, 2040044. One spare parts list for each model. Make sure to order correct spare part.



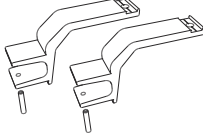

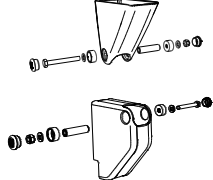
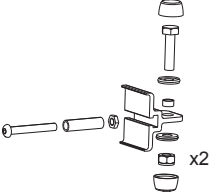
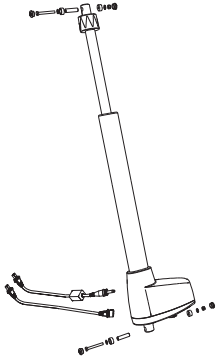
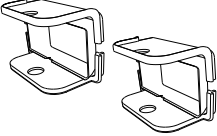
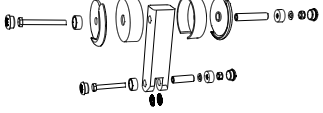
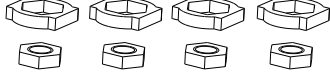
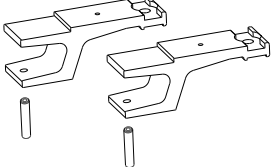
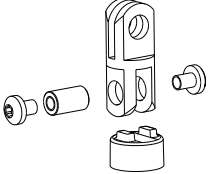
Viking™ L mobile lift, 2040014

PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20490039	336	Control Unit CBL, by S/N <i>Can only be ordered if S/N (serial number) can be provided to Liko™ Customer Service.</i> <i>Will be programmed at Liko™ before delivery.</i> <i>2 weeks delivery time from Hill-Rom/Liko™ Alvik, Sweden</i>	pce		3SE650270
20490043	453	Control Unit CBL, Viking L <i>Before use; requires Software and, needs to be programmed with Liko™ Service System, Prod. No. 21090024</i>	pce		3SE650290
20090026	22	Cable for Charging, EU	pce		3EN630237
20090027	23	Cable for Charging, UK	pce		
20090028	24	Cable for Charging, US	pce		
20390006	159	Fork Cover	pair		3EN380202
20490003	192	Front Wheel 100 mm <i>We recommend replacing both wheels, as there are two different versions that can vary in appearance. (dimension is the same)</i>	pce		3SE630293
20490004	193	Rear Wheel 125 mm <i>We recommend replacing both wheels, as there are two different versions that can vary in appearance. (dimension is the same)</i>	pce		3SE630294
20490005	194	Rear Fork Cover	pair		3SE630295
20490010	199	Locking Handle Viking XL	pair		3SE630244
20090050	248	Cable for Charging AU/NZ	pce		3EN630237

Viking™ L mobile lift, 2040014

PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20490011	266	Base Motor Viking M, L	pce		3SE630271
20090066	293	End Cover Handle/Arm Support grey, 4 pce	set		3SE630271
20490023	307	Lift arm plastic cover	set		3EN630250
20490024	308	Overhead beam restoration set	set		3EN630251
31590014	311	Latch, composite <i>Fits Universal SlingBar from S/N 1299511</i>	pair		3SE640242
20490031	333	Wireless hand control HBL20	set		3SE640298
21090018	348	Extension cable CBL20	pce		3SE650216
21090019	349	Extension cable CBL 20 US,	pce		
2006107	350	Extra battery box	pce		-
21090020	351	Beltclip HBL20	pair		3SE650217
21090021	352	Clips with hook HBL20	pair		3SE650218
21090022	353	Charge cable hook , 10 pce	set		3SE650219
21090023	354	Hand control hanger, 10 pce	set		3SE650220

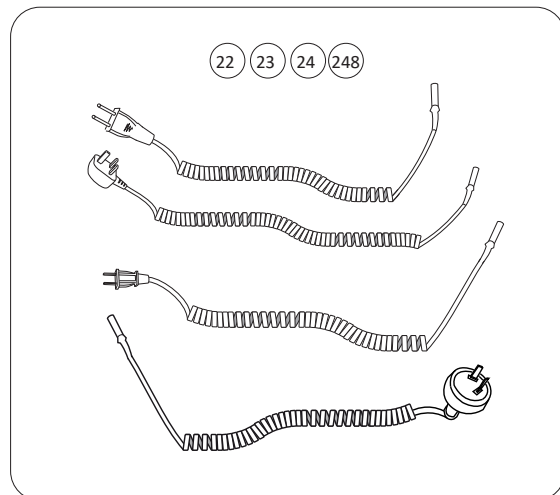
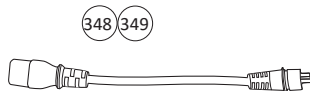
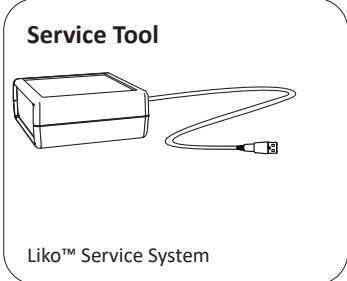
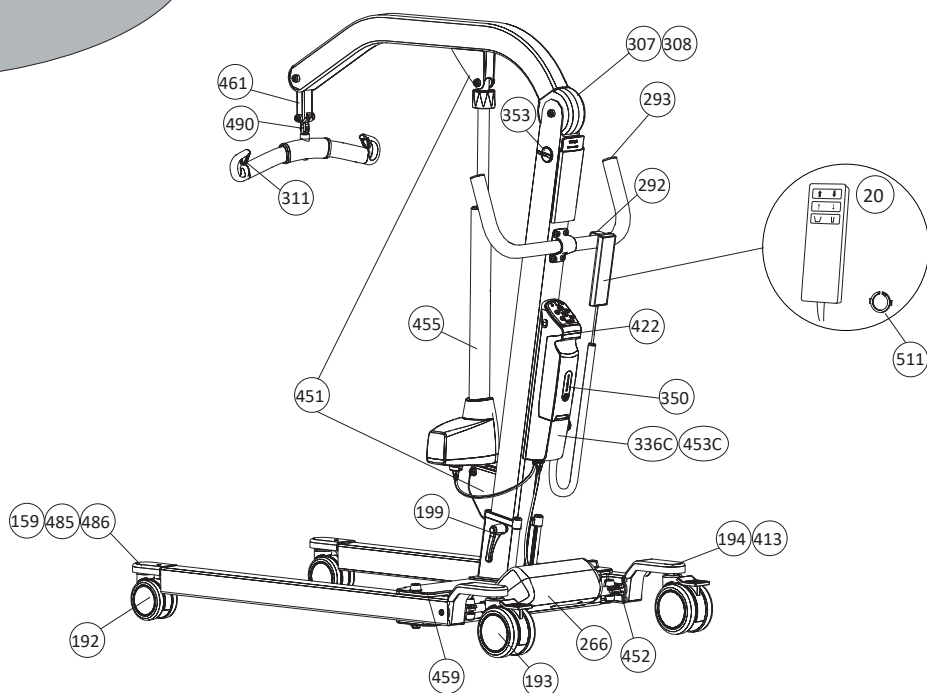
Viking™ L mobile lift, 2040014

PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20490037	413	Rear Wheel fork	pair		3SE690221
20090077	422	Locking device CBL20, 10 pce	set		3SE640217
20490041	451	Bracket Cover II	set		3SE650287
20490042	452	Restoration set motor attachment	set		3SE650288
20490065	557	Actuator LA44 <i>Replaces 20490045</i>	set		3EN690287
20490047	459	Leg Cover L&XL	set		3EN650294
20490049	461	Flexlink	set		3SE650295
20090080	485	Fork locking attachment, 4 pair	set		3SE690208
20090081	486	Wheel Fork	pair		3SE690209
20490052	490	SlingBar Assembly set	set		3SE690210


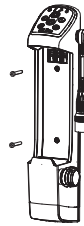
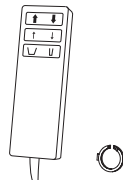
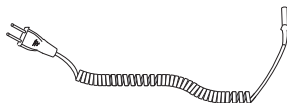

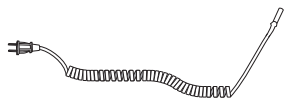
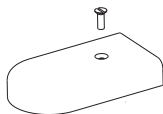
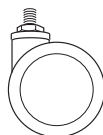
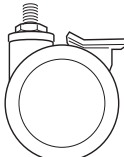

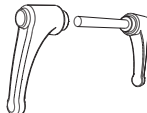
Spare Parts Positions Viking™ L mobile lift

PROD. NO.	PRODUCT
2040034	Viking L (with intelligent control box and cable hand control) (Discontinued 2017) (S/N 9 300 001- 9 400 000)


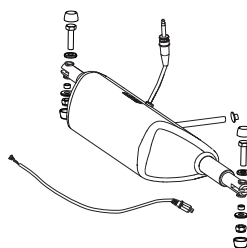
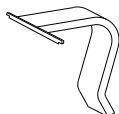

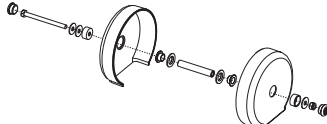
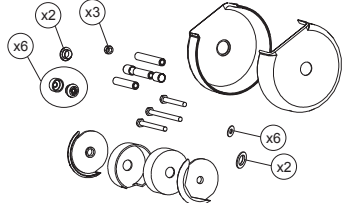





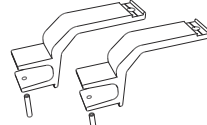

! There are four models of Viking™ L mobile lift; Prod. No. 2040004, 2040014, 2040034, 2040044. One spare parts list for each model. Make sure to order correct spare part.



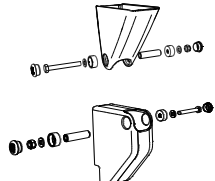
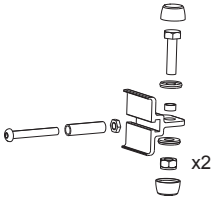
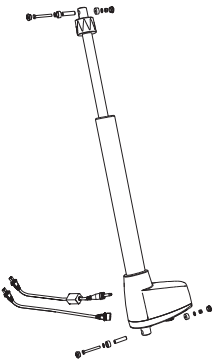
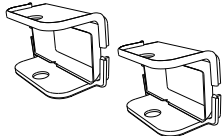
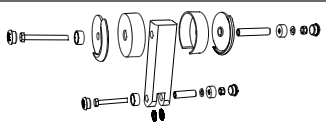
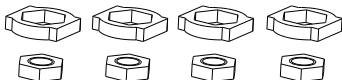
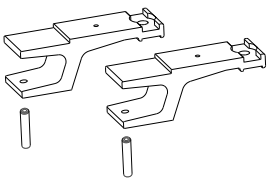
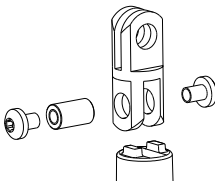
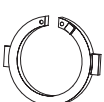
Viking™ L mobile lift, 2040034

PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20490054	336C	Control Unit CBL by S/N, Cable <i>Can only be ordered if S/N (serial number) can be provided to Liko™ Customer Service. Will be programmed at Liko™ before delivery.</i> <i>2 weeks delivery time from Hill-Rom/Liko™ Alvik, Sweden</i>	pce		3SE690227
20490055	453C	Control Unit CBL Viking L, Cable <i>Before use; requires Software and, needs to be programmed with Liko™ Service System, Prod. No. 21090024</i>	pce		3SE690228
20090089	20	Hand control 6-Button	pce		3SE690232
20090026	22	Cable for Charging, EU	pce		3EN630237
20090027	23	Cable for Charging, UK	pce		
20090028	24	Cable for Charging, US	pce		
20390006	159	Fork Cover	pair		183916
20490003	192	Front Wheel 100 mm <i>We recommend replacing both wheels, as there are two different versions that can vary in appearance. (dimension is the same)</i>	pce		3SE630293
20490004	193	Rear Wheel 125 mm <i>We recommend replacing both wheels, as there are two different versions that can vary in appearance. (dimension is the same)</i>	pce		3SE630294
20490005	194	Rear Fork Cover	pair		3SE630295
20490010	199	Locking Handle Viking XL	pair		3SE630244


Viking™ L mobile lift, 2040034

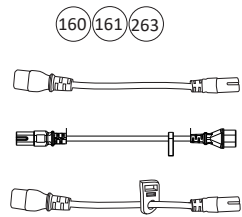
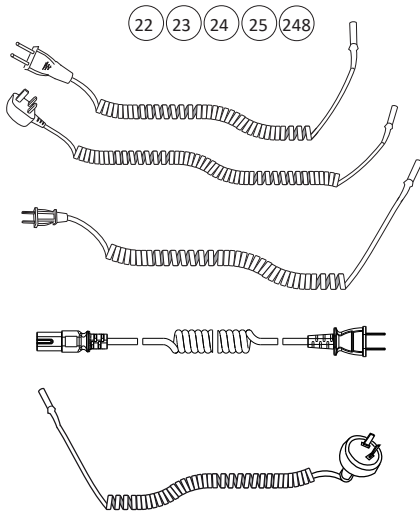
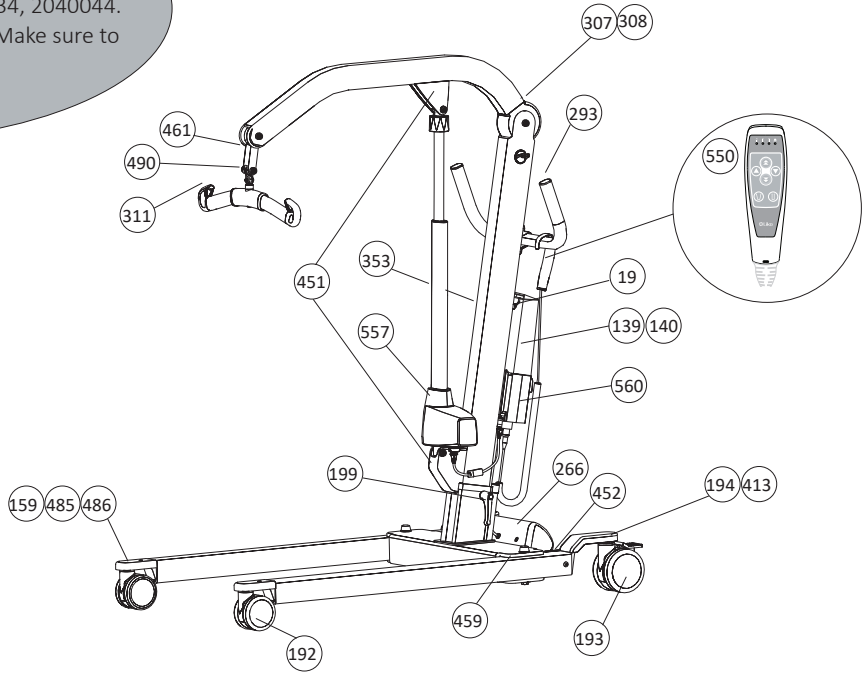
PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20090050	248	Cable for Charging AU/NZ	pce		3EN630237
20490011	266	Base Motor Viking M, L	pce		3SE630271
20090058	292	Hook for Hand control 2,4,6	pce		3SE630210
20090066	293	End Cover Handle/Arm Support grey, 4 pce	set		3SE630271
20490023	307	Lift arm plastic cover <i>Prod. No. 2040004, from S/N 7200795</i>	set		3EN630250
20490024	308	Overhead beam restoration set <i>Prod. No. 2040004, from S/N 7200795</i>	set		3EN630251
31590014	311	Latch, composite <i>Fits Universal SlingBar from S/N 1299511</i>	pair		3SE640242
21090018	348	Extension cable CBL20	pce		3SE650216
21090019	349	Extension cable CBL 20 US,	pce		
2006107	350	Extra battery box	pce		-
21090022	353	Charge cable hook , 10 pce	set		3SE650219
20490037	413	Rear Wheel fork	pair		3SE650264
20090077	422	Locking device CBL20, 10 pce	set		3SE640217

Viking™ L mobile lift, 2040034

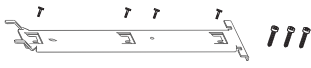

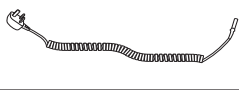
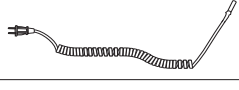

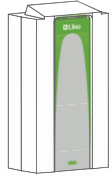
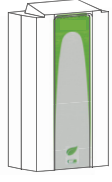
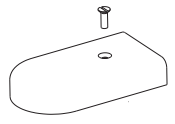

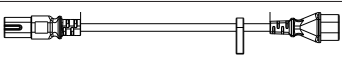



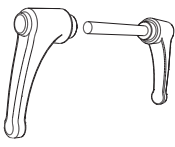
PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20490041	451	Bracket Cover II	set		3SE650287
20490042	452	Restoration set motor attachment	set		3SE650288
20490065	557	Actuator LA44 <i>Replaces 20490045</i>	set		3EN690287
20490047	459	Leg Cover L&XL	set		3EN650294
20490049	461	Flexlink	set		3SE650295
20090080	485	Fork locking attachment, 4 pair	set		3SE690208
20090081	486	Wheel Fork	pair		3SE690209
20490052	490	SlingBar Assembly set	set		3SE690210
20490057	511	Locking Ring CBL, 10 pce	set		3SE690231

PROD. NO.	PRODUCT
2040044	Viking L jumbo care control box and LED hand control (S/N: 9 500 001- 9 600 000)

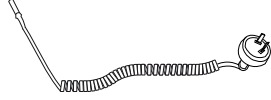
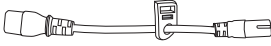
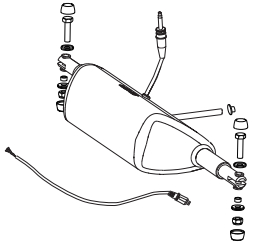

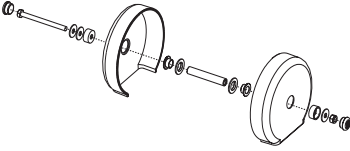
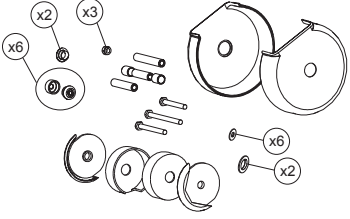


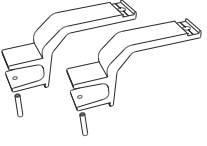
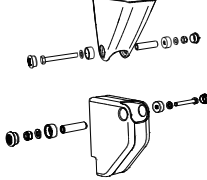
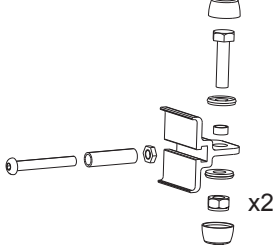
 There are four models of Viking™ L mobile lift; Prod. No. 2040004, 2040014, 2040034, 2040044. One spare parts list for each model. Make sure to order correct spare part.



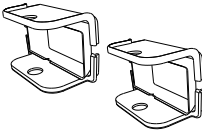
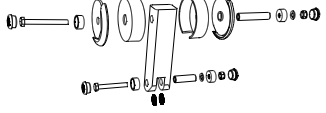
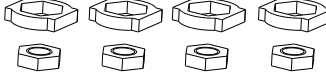
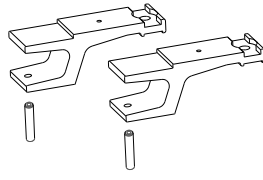
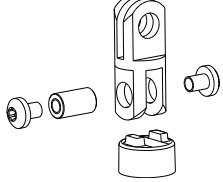

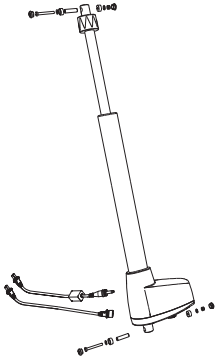
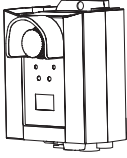
Viking™ L mobile lift, 2040044

PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20090023	19	Attachment for Control Box	pce		3EN630238
20090026	22	Cable for Charging, EU	pce		3SE630237
20090027	23	Cable for Charging, UK	pce		
20090028	24	Cable for Charging, US	pce		
20090029	25	Cable for charging JP	pce		
2006106	139	Battery Lead Acid	pce		3EN630224
2006110	140	Battery Lithium Ion	pce		3EN690296
20390006	159	Fork Cover	pair		3SE380202
21090001	160	Extension Cable	pce		3EN630225
21090000	161	Extension Cable JP	pce		3EN690280
20490003	192	Front Wheel 100 mm <i>We recommend replacing both wheels, as there are two different versions that can vary in appearance. (dimension is the same)</i>	pce		3SE630293
20490004	193	Rear Wheel 125 mm <i>We recommend replacing both wheels, as there are two different versions that can vary in appearance. (dimension is the same)</i>	pce		3SE630294
20490005	194	Rear Fork Cover	pair		3SE630295
20490010	199	Locking Handle Viking XL	pair		3SE630244

Viking™ L mobile lift, 2040044


PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20090050	248	Cable for Charging, AU/NZ	pce		3SE630237
21090011	263	Extension Cable US	pce		3EN630218
20490011	266	Base Motor Viking M, L	pce		3SE630271
20090066	293	End Cover Handle/Arm Support grey, 4 pce	set		3SE630271
20490023	307	Lift arm plastic cover	set		3EN630250
20490024	308	Overhead beam restoration set	set		3EN630251
31590014	311	Latch, composite <i>Fits Universal SlingBar from S/N 1299511</i>	pair		3SE640242
21090022	353	Charge cable hook , 10 pce	set		3SE650219
20490037	413	Rear Wheel fork	pair		3SE650264
20490041	451	Bracket Cover II	set		3SE650287
20490042	452	Restoration set motor attachment	set		3SE650288

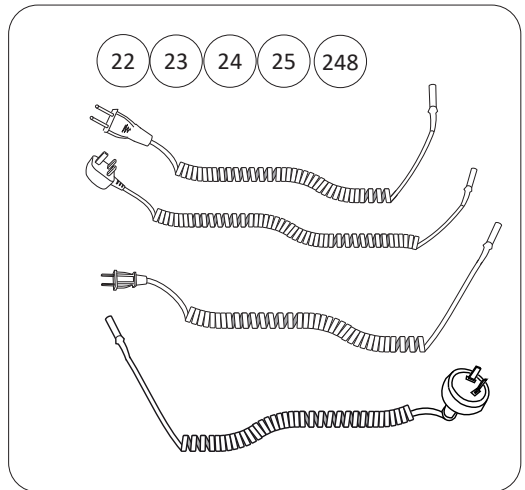
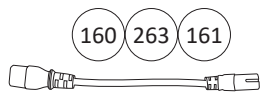
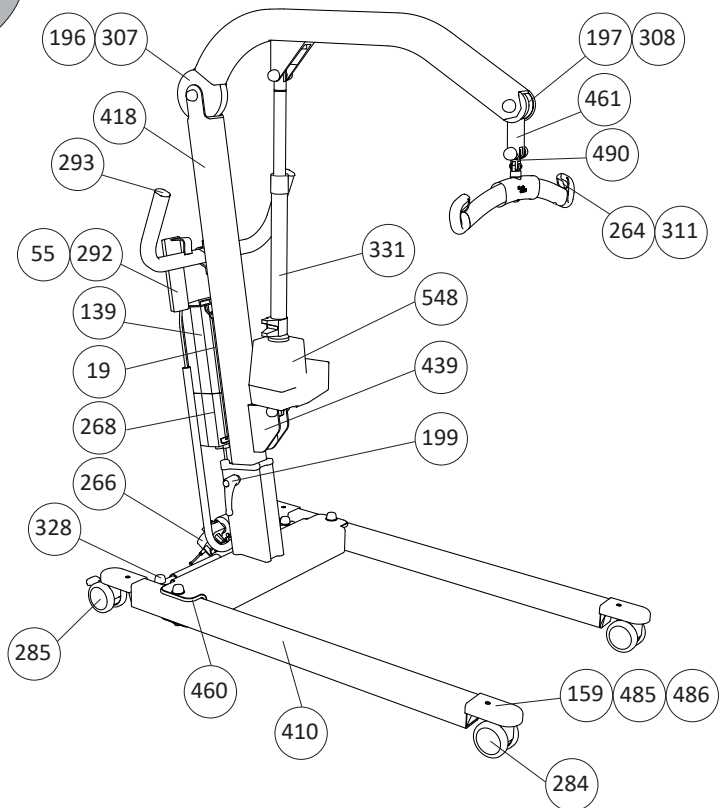
Viking™ L mobile lift, 2040044

PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20490047	459	Leg Cover L&XL	set		3EN650294
20490049	461	Flexlink	set		3SE650295
20090080	485	Fork locking attachment, 4 pair	set		3SE690208
20090081	486	Wheel Fork	pair		3SE690209
20490052	490	SlingBar Assembly set	set		3SE690210
20090098	550	Hand Control 6 button, LED	pce		3EN690283
20490065	557	Actuator LA44	set		3EN690287
20490068	560	Control Box L Jumbo Care	pce		3EN690282

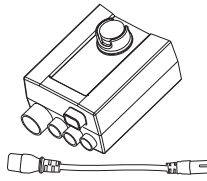

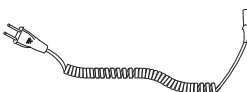


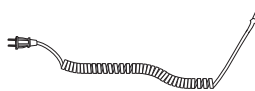

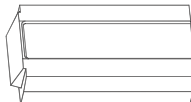
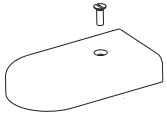


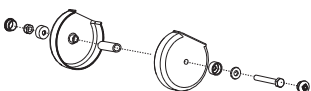
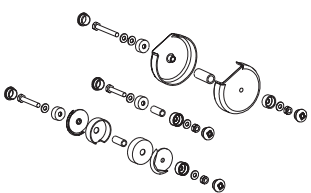
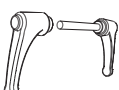
Spare Parts Positions Viking™ M mobile lift

PROD. NO.	PRODUCT
2040005	Viking M (Discontinued 2013) (S/N 7 100 101 - 7 200 200)




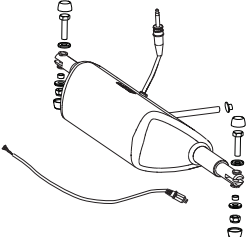
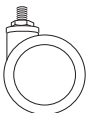

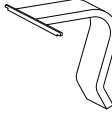

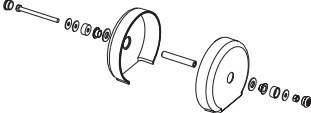
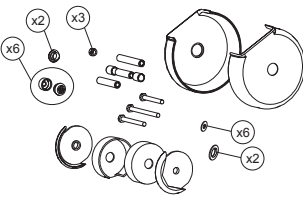

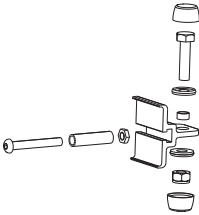
 There are four models of Viking™ M mobile lift; Prod. No. 2040005, 2040015, 2040035 and 2040045. One spare parts list for each model. Make sure to order correct spare part.



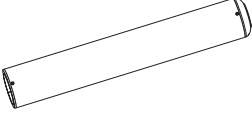
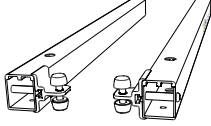
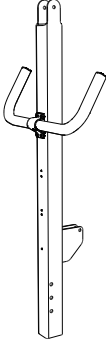
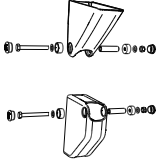
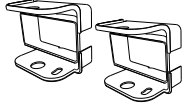
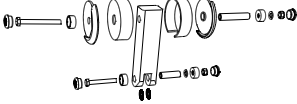
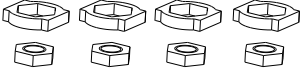
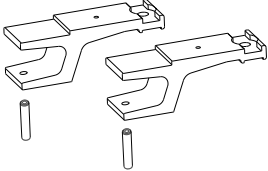
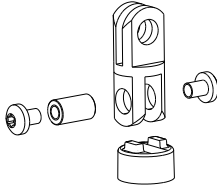
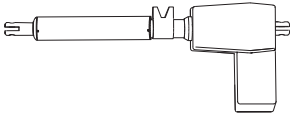
Viking™ M mobile lift, 2040005

PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20490013	268	Control Box Viking M S/N 7100101-7200200	pce		3SE630223
20490013US	268	Control Box Viking M , US S/N 7100101-7200200 only for the US-market			
20490013JP	268	Control Box Viking M, JP S/N 7100101-7200200 only for the Japanese market			
20090023	19	Attachment for Control Box	pce		3EN630238
20090026	22	Cable for charging EU	pce		3EN630237
20090027	23	Cable for charging UK	pce		
20090028	24	Cable for charging US	pce		
20090029	25	Cable for charging JP	pce		3EN690279
20190017	55	Hand control EE 4 buttons	pce		3SE630204
2006106	139	Extra Battery	pce		3EN630224
20390006	159	Fork Cover	pair		183916
21090001	160	Extension Cable	pce		3EN630225
21090000	161	Extension Cable JP			3EN690280
20490007	196	Lift Arm Plastic Cover to S/N 7106200	set		3SE630240
20490008	197	Restoration set Overhead beam to S/N 7106200	set		3SE630228
20490010	199	Locking handle Viking XL	pair		3SE630244

Viking™ M mobile lift, 2040005

PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20090050	248	Cable for Charging AU/NZ	pce		3EN630237
21090011	263	Extension Cable US	pce		3EN630218
31590011	264	Latch Universal SlingBar	pair		3SE630229
20490011	266	Base Motor Viking M, L <i>(Viking M 2040005 up to S/N 7107499 also need spare part 20490028, Pos 328)</i>	pce		3EN630296
20490014	284	Front Wheel 75 mm Standard from S/N 7102064	pce		182904
20490015	285	Rear Wheel 75 mm Standard from S/N 7102064	pce		182903
20090058	292	Hook for Hand control 2,4,6	pce		3SE630210
20090066	293	End Cover Handle/Arm Support grey, 4 pce	set		3SE630271
20490023	307	Lift Arm Plastic Cover from S/N 7106201	set		3EN630250
20490024	308	Overhead Beam Rest. set from S/N 7106201	set		3EN630251
31590014	311	Latch, composite <i>Fits Universal SlingBar from S/N 1299511</i>	pair		3SE640242
20490028	328	Restoration set motor attachment	set		3SE640258

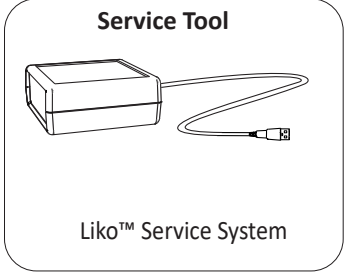
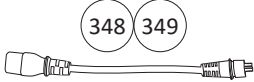
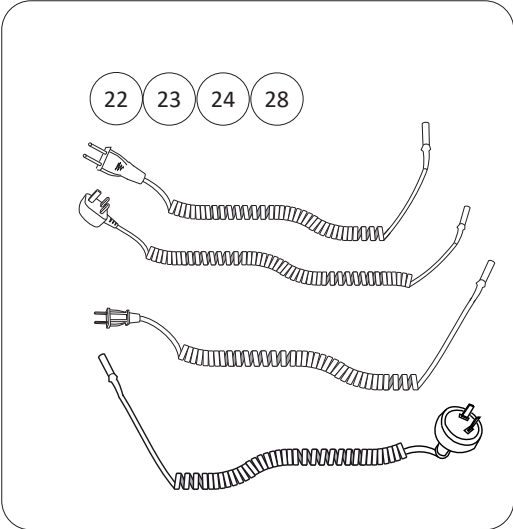
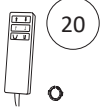
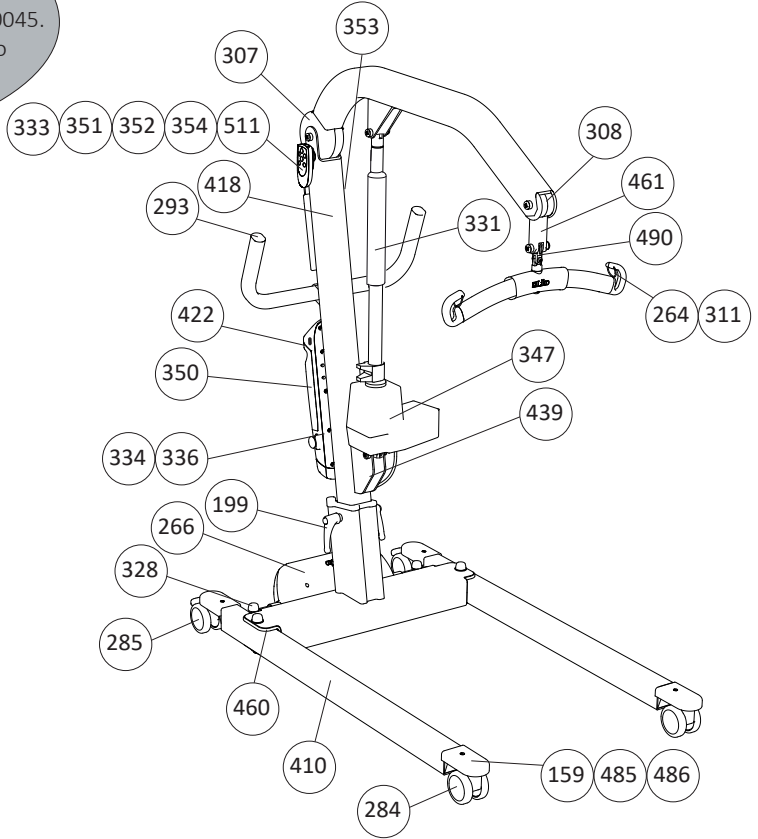
Viking™ M mobile lift, 2040005

PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20490030	331	Outer Tube	set		3SE640260
20490036	410	Viking M Legs <i>from S/N 7100308</i>	pair		3SE650259
20490038	418	Mast Viking M <i>including handle and motor attachment</i>	set		3EN650269
20490040	439	Bracket Cover	set		3SE650273
20490048	460	Leg Cover	set		3EN650294
20490049	461	Flexlink	set		3SE650295
20090080	485	Fork locking attachment, 4 pair	set		3SE690208
20090081	486	Wheel Fork	pair		3SE690209
20490052	490	SlingBar Assembly set	set		3SE690210
20490064	548	Actuator LA34 CBJ	pce		3EN690277



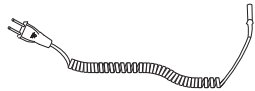

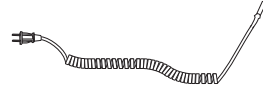
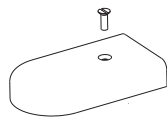
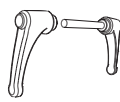

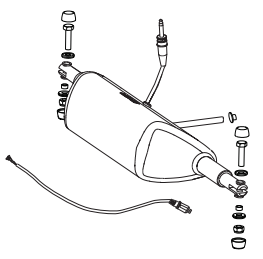


Spare Parts Positions Viking™ M mobile lift

PROD. NO.	PRODUCT
2040015	Viking M (CBL control box) <i>(Discontinued 2016)</i> (S/N 7 500 401 - 7 600 500)


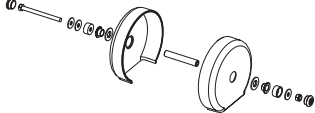
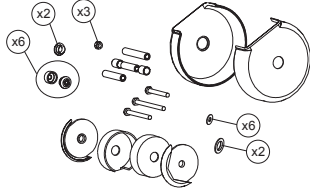

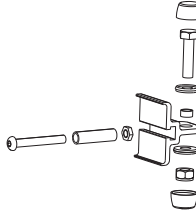
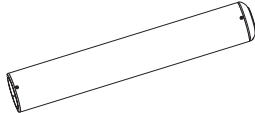

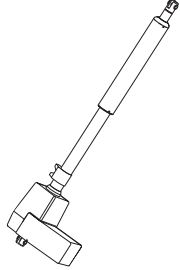



! There are four models of Viking™ M mobile lift; Prod. No. 2040005, 2040015, 2040035 and 2040045. One spare parts list for each model. Make sure to order correct spare part.







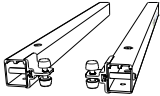
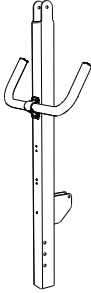

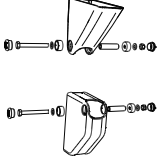

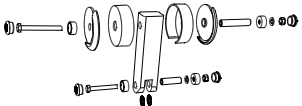
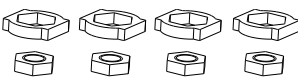
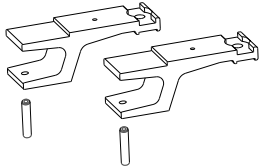
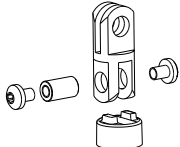
Viking™ M mobile lift, 2040015

PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20490032	334	Control Unit CBL, Viking M <i>Before use; requires Software and, needs to be programmed with Liko™ Service System, Prod. No. 21090024</i>	pce		3SE640299
20490039	336	Control Unit CBL, by S/N <i>Can only be ordered if S/N (serial number) can be provided to Liko™ Customer Service. Will be programmed at Liko™ before delivery. 2 weeks delivery time from Hill-Rom/Liko™ Alvik, Sweden</i>	pce		3SE650270
20090026	22	Cable for charging EU	pce		3EN630237
20090027	23	Cable for charging UK	pce		
20090028	24	Cable for charging US	pce		
20390006	159	Fork Cover	pair		183916
20490010	199	Locking handle Viking XL	pair		3SE630244
20090050	248	Cable for Charging AU/NZ	pce		3EN630237
20490011	266	Base Motor Viking M, L <i>(Viking M 2040005 up to S/N 7107499 also need spare part 20490028, Pos 328)</i>	pce		3EN630296
20490014	284	Front Wheel 75 mm Standard <i>from S/N 7102064</i>	pce		182904
20490015	285	Rear Wheel 75 mm Standard <i>from S/N 7102064</i>	pce		182903

Viking™ M mobile lift, 2040015

PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20090066	293	End Cover Handle/Arm Support grey, 4 pce	set		3SE630271
20490023	307	Lift Arm Plastic Cover <i>from S/N 7106201</i>	set		3EN630250
20490024	308	Overhead Beam Rest. set <i>from S/N 7106201</i>	set		3EN630251
31590014	311	Latch, composite <i>Fits Universal SlingBar from S/N 1299511</i>	pair		3SE640242
20490028	328	Restoration set motor attachment	set		3SE640258
20490030	331	Outer Tube	set		3SE640260
20490031	333	Wireless hand control HBL20	set		3SE640298
20490033	347	Actuator Vi M	pce		3EN650214
21090018	348	Extension cable CBL20	pce		3SE650216
21090019	349	Extension cable CBL 20 US,	pce		3SE650216
2006107	350	Extra battery box	pce		7EN137106 Instruction Guide

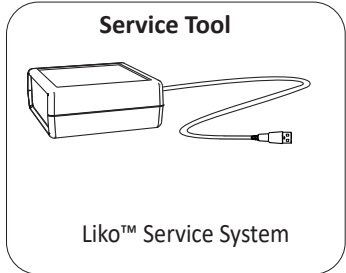
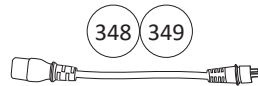
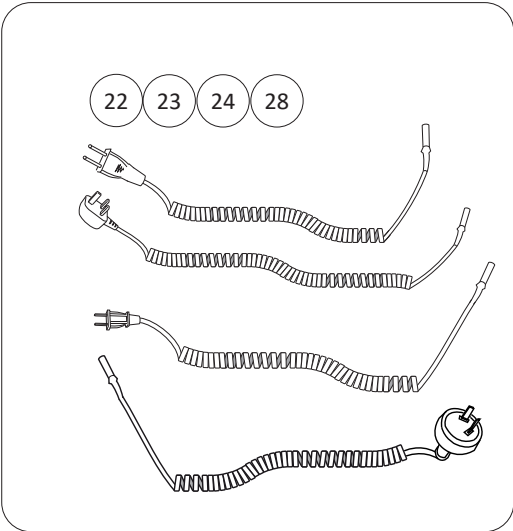
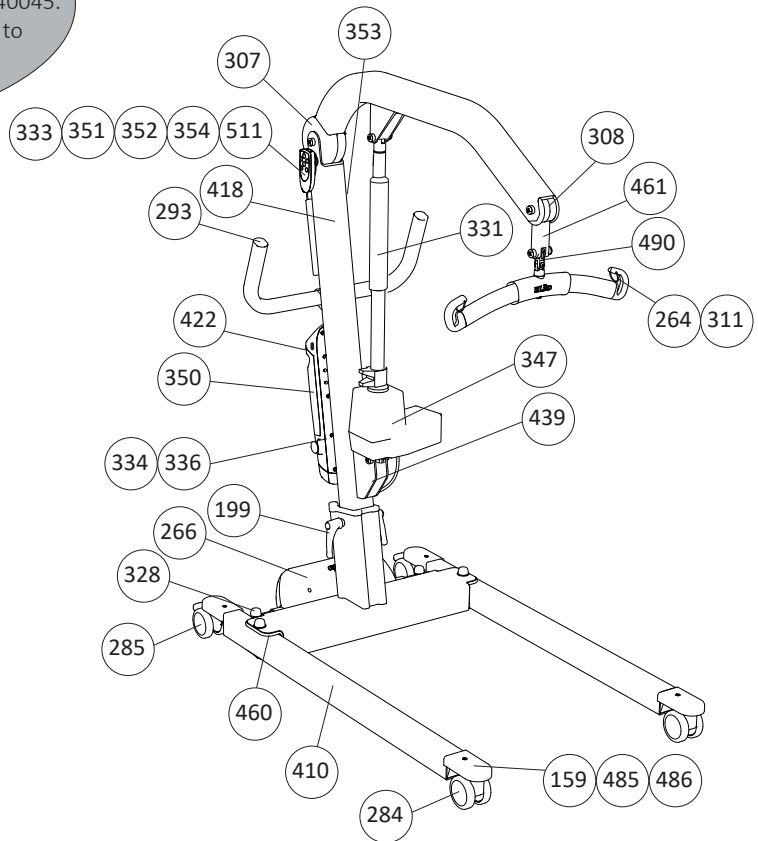
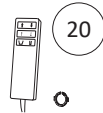
Viking™ M mobile lift, 2040015

PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
21090020	351	Beltclip HBL20	pair		3SE650217
21090021	352	Clips with hook HBL20	pair		3SE650218
21090022	353	Charge cable hook , 10 pce	set		3SE650219
21090023	354	Hand control hanger, 10 pce	set		3SE650220
20490036	410	Viking M Legs <i>from S/N 7100308</i>	pair		3SE650259
20490038	418	Mast Viking M <i>including handle and motor attachment</i>	set		3EN650269
20090077	422	Locking device CBL20, 10 pce	set		3SE640217
20490040	439	Bracket Cover	set		3SE650273
20490048	460	Leg Cover	set		3EN650294
20490049	461	Flexlink	set		3SE650295
20090080	485	Fork locking attachment, 4 pair	set		3SE690208
20090081	486	Wheel Fork	pair		3SE690209
20490052	490	SlingBar Assembly set	set		3SE690210

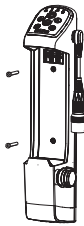
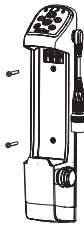
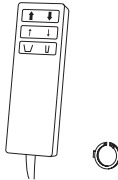
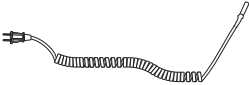
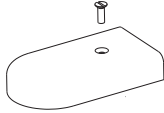
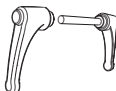

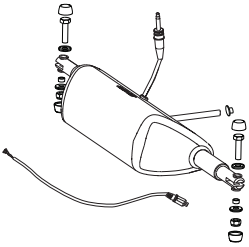


Spare Parts Positions Viking™ M mobile lift

PROD. NO.	PRODUCT
2040035	Viking M (CBL control box and cable hand control) <i>(Discontinued 2016)</i> (S/N 9 200 001 - 9 300 000)

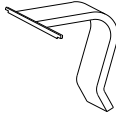

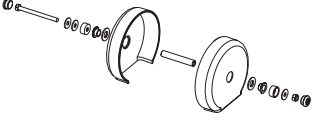
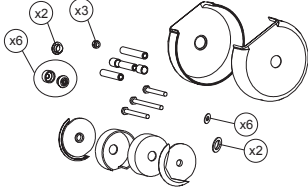

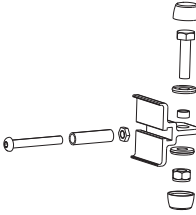
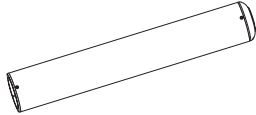
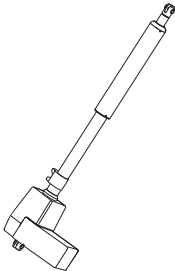



! There are four models of Viking™ M mobile lift; Prod. No. 2040005, 2040015, 2040035 and 2040045. One spare parts list for each model. Make sure to order correct spare part.




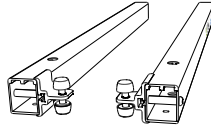
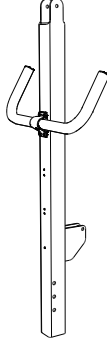

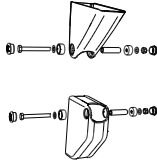
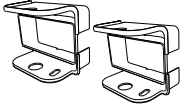
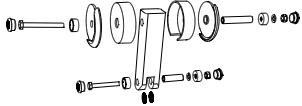
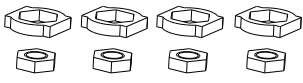
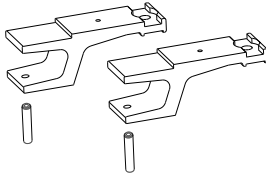
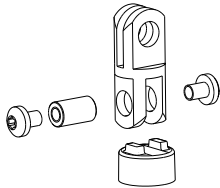
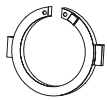
Viking™ M mobile lift, 2040035

PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20490053	334C	Control Unit CBL Viking M, Cable <i>Before use; requires Software and, needs to be programmed with Liko™ Service System, Prod. No. 21090024</i>	pce		3SE690226
20490054	336C	Control Unit CBL by S/N, Cable <i>Can only be ordered if S/N (serial number) can be provided to Liko™ Customer Service. Will be programmed at Liko™ before delivery. 2 weeks delivery time from Hill-Rom/Liko™ Alvik, Sweden</i>	pce		3SE690227
20090089	20	Hand control 6-Button	pce		3SE690232
20090026	22	Cable for charging EU	pce		3SE630237
20090027	23	Cable for charging UK			
20090028	24	Cable for charging US			
20390006	159	Fork Cover	pair		183916
20490010	199	Locking handle Viking XL	pair		3SE630244
20090050	248	Cable for Charging AU/NZ	pce		3EN630237
20490011	266	Base Motor Viking M, L <i>(Viking M 2040005 up to S/N 7107499 also need spare part 20490028, Pos 328)</i>	pce		3EN630296
20490014	284	Front Wheel 75 mm Standard <i>from S/N 7102064</i>	pce		182904
20490015	285	Rear Wheel 75 mm Standard <i>from S/N 7102064</i>	pce		182903

Viking™ M mobile lift, 2040035

PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20090058	292	Hook for Hand control 2,4,6	pce		3SE630210
20090066	293	End Cover Handle/Arm Support grey, 4 pce	set		3SE630271
20490023	307	Lift Arm Plastic Cover <i>from S/N 7106201</i>	set		3EN630250
20490024	308	Overhead Beam Rest. set <i>from S/N 7106201</i>	set		3EN630251
31590014	311	Latch, composite <i>Fits Universal SlingBar from S/N 1299511</i>	pair		3SE640242
20490028	328	Restoration set motor attachment	set		3SE640258
20490030	331	Outer Tube	set		3SE640260
20490033	347	Actuator Vi M	pce		3EN650214
21090018	348	Extension cable CBL20	pce		3SE650216
21090019	349	Extension cable CBL 20 US,	pce		3SE650216
2006107	350	Extra battery box	pce		7EN137106 Instruction Guide

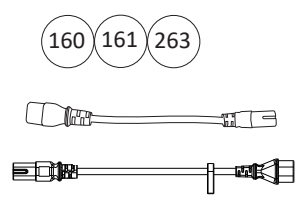
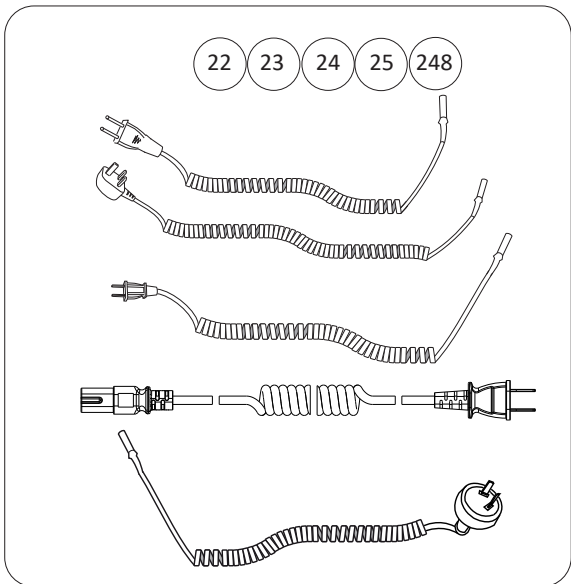
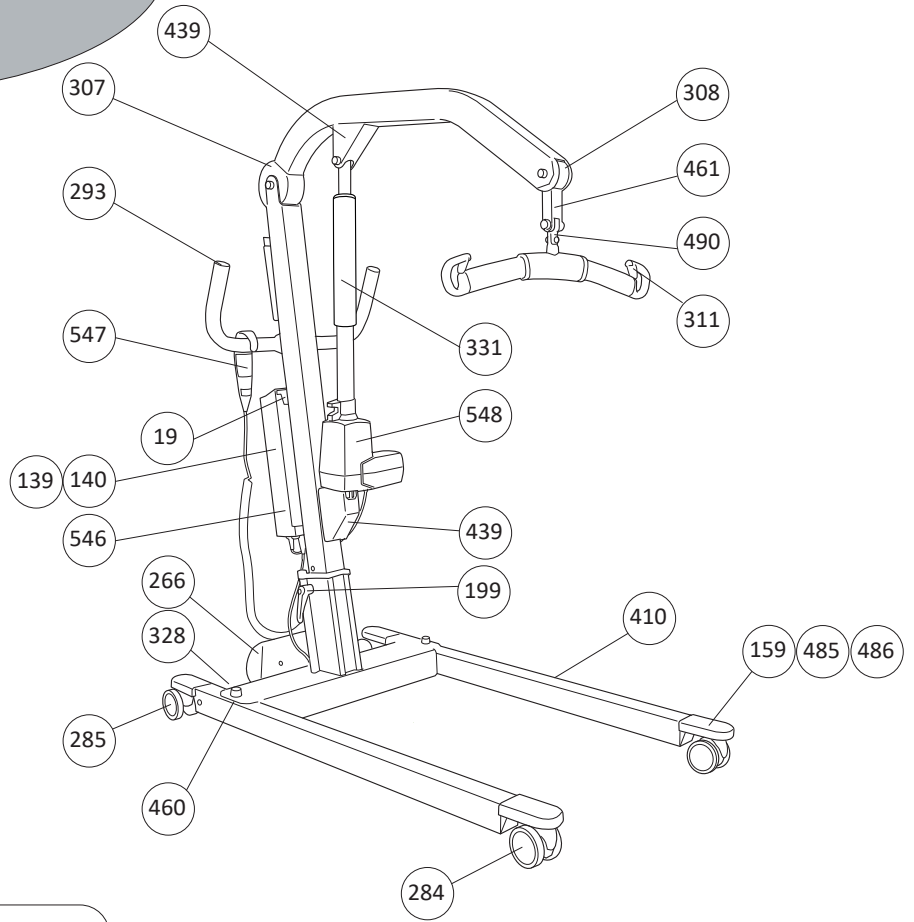
Viking™ M mobile lift, 2040035

PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
21090022	353	Charge cable hook , 10 pce	set		3SE650219
20490036	410	Viking M Legs <i>from S/N 7100308</i>	pair		3SE650259
20490038	418	Mast Viking M <i>including handle and motor attachment</i>	set		3EN650269
20090077	422	Locking device CBL20, 10 pce	set		3SE640217
20490040	439	Bracket Cover	set		3SE650273
20490048	460	Leg Cover	set		3EN650294
20490049	461	Flexlink	set		3SE650295
20090080	485	Fork locking attachment, 4 pair	set		3SE690208
20090081	486	Wheel Fork	pair		3SE690209
20490052	490	SlingBar Assembly set	set		3SE690210
20490057	511	Locking Ring CBL, 10 pce	set		3SE690231

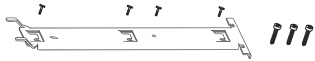
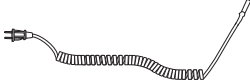
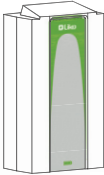
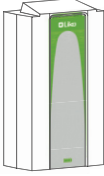
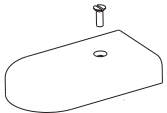

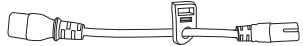
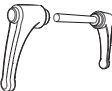


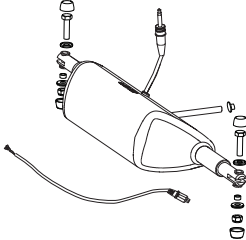


Spare Parts Positions Viking™ M mobile lift

PROD. NO.	PRODUCT
2040045	Viking M (Jumbo care control box and LED hand control) from S/N 9 400 001


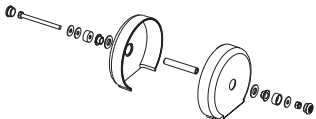
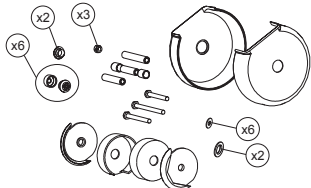

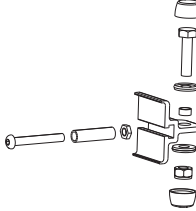
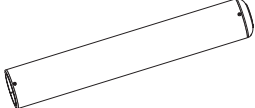

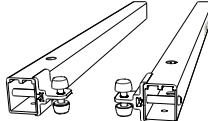
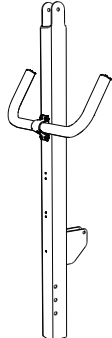
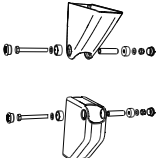
! There are four models of Viking™ M mobile lift; Prod. No. 2040005, 2040015, 2040035 and 2040045. One spare parts list for each model. Make sure to order correct spare part.



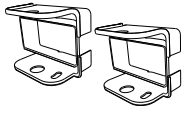
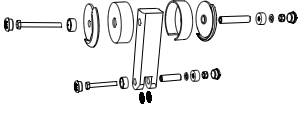
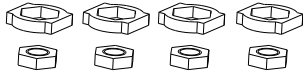
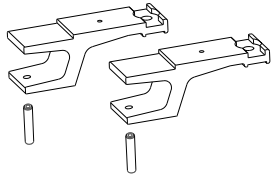
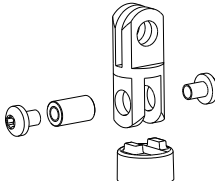
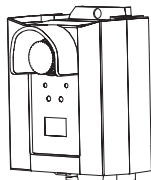

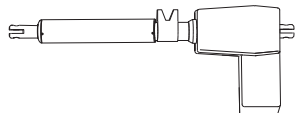
Viking™ M mobile lift, 2040045

PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20090023	19	Attachment for Control Box	pce		3EN630238
20090026	22	Cable for charging EU	pce		3SE630237
20090027	23	Cable for charging UK	pce		
20090028	24	Cable for charging US	pce		
20090029	25	Cable for charging JP	pce		
2006106	139	Battery Lead Acid, 24 V, 2.9 Ah	pce		3EN630224
2006109	140	Battery Lithium Ion, 25.6 V, 2.25 Ah	pce		3EN690296
20390006	159	Fork Cover	pair		183916
21090001	160	Extension Cable	pce		3EN630225
21090000	161	Extension cable JP	pce		3EN690280
20490010	199	Locking handle Viking XL	pair		3SE630244
20090050	248	Cable for Charging AU/NZ	pce		3EN630237
21090011	263	Extension Cable US	pce		3EN630218
20490011	266	Base Motor Viking M, L (Viking M 2040005 up to S/N 7107499 also need spare part 20490028, Pos 328)	pce		3EN630296
20490014	284	Front Wheel 75 mm Standard from S/N 7102064	pce		182904
20490015	285	Rear Wheel 75 mm Standard from S/N 7102064	pce		182903

Viking™ M mobile lift, 2040045

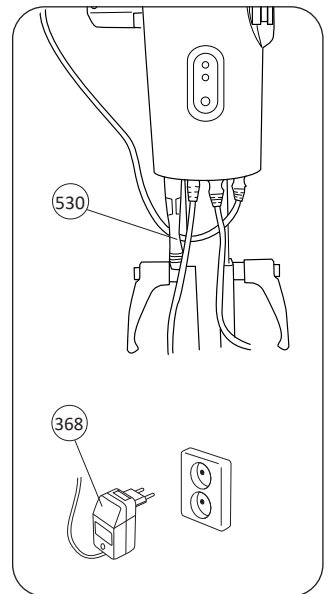
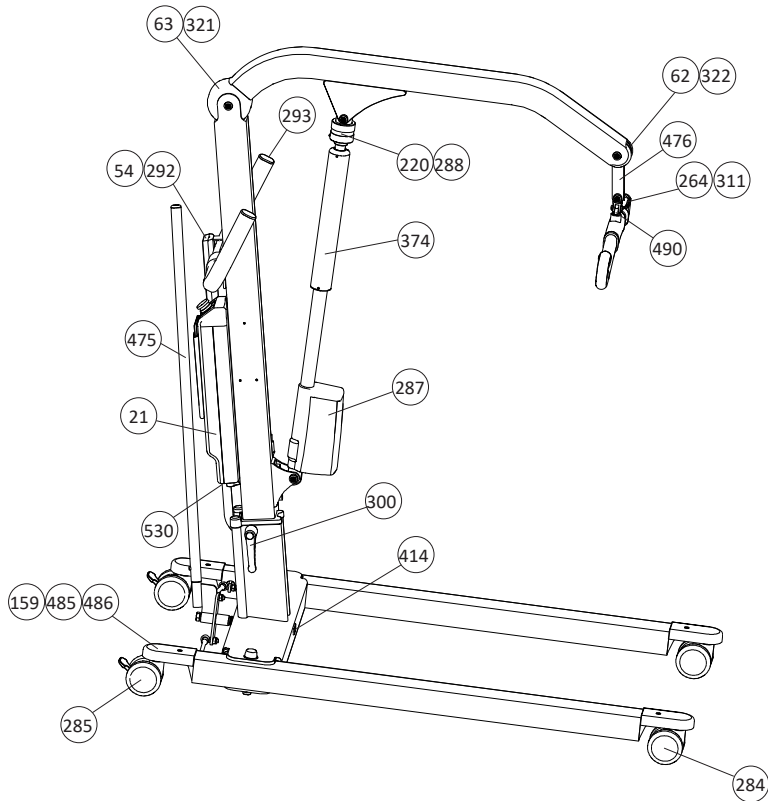
PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20090066	293	End Cover Handle/Arm Support grey, 4 pce	set		3SE630271
20490023	307	Lift Arm Plastic Cover <i>from S/N 7106201</i>	set		3EN630250
20490024	308	Overhead Beam Rest. set <i>from S/N 7106201</i>	set		3EN630251
31590014	311	Latch, composite <i>Fits Universal SlingBar from S/N 1299511</i>	pair		3SE640242
20490028	328	Restoration set motor attachment	set		3SE640258
20490030	331	Outer Tube	set		3SE640260
21090022	353	Charge cable hook , 10 pce	set		3SE650219
20490036	410	Viking M Legs <i>from S/N 7100308</i>	pair		3SE650259
20490038	418	Mast Viking M <i>including handle and motor attachment</i>	set		3EN650269
20490040	439	Bracket Cover	set		3SE650273

Viking™ M mobile lift, 2040045

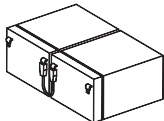
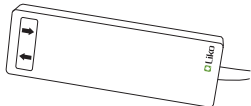
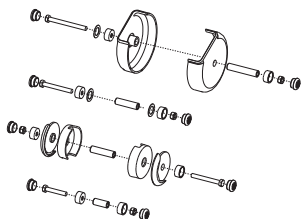
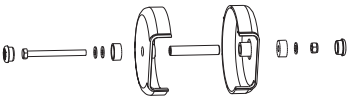
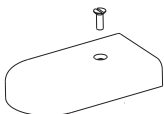
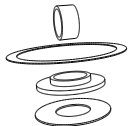

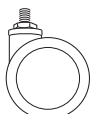
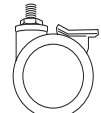
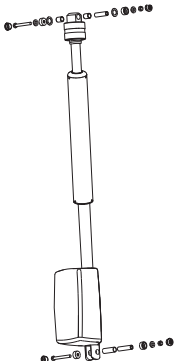
PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20490048	460	Leg Cover	set		3EN650294
20490049	461	Flexlink	set		3SE650295
20090080	485	Fork locking attachment, 4 pair	set		3SE690208
20090081	486	Wheel Fork	pair		3SE690209
20490052	490	SlingBar Assembly set	set		3SE690210
20490062	546	Control Box Jumbo Care	pce		3SE690231
20490063	547	Hand control, buttons LED	pce		3EN690276
20490064	548	Actuator LA34 CBJ	pce		3EN690277

Spare Parts Positions Viking™ S mobile lift

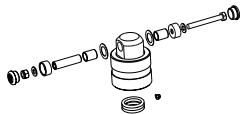
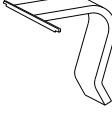

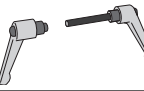

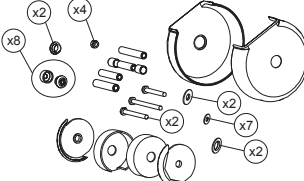
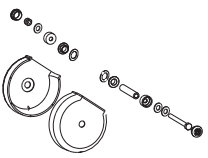
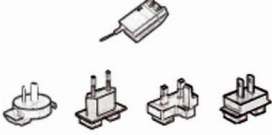
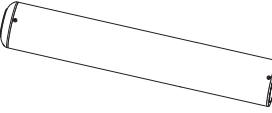
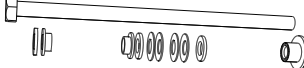
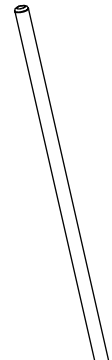
PROD. NO.	PRODUCT
2040006	Viking S (Discontinued 2018)



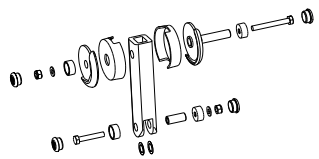
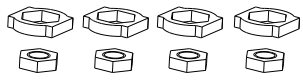
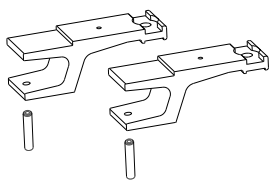
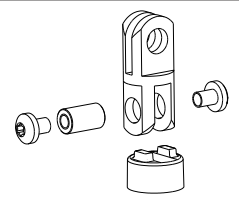
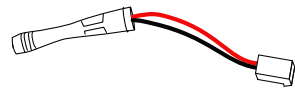
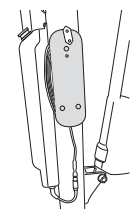

Viking™ S mobile lift

PROD. NO.	POS. NO.	NAME	UNIT	ASSEMBLY INSTR.
20090025	21	Battery 2,9/12V	pair	 3SE380201
20190016	54	Hand control EM 2-Button	pce	 3SE630204
20190044	62	Overhead Beam Restoration set <i>Up to S/N 7300867</i>	set	 3SE630202
20190045	63	Lift Arm Plastic Cover <i>Up to S/N 7300867</i>	set	 3SE630211
20390006	159	Fork Cover	pair	 183916
20190051	220	Restoration set Emergency Lowering	set	 3SE630222
31590011	264	Latch Universal SlingBar	pair	 3SE630229
20490014	284	Front Wheel 75 mm Standard	pce	 182904
20490015	285	Rear Wheel 75 mm Standard	pce	 182903
20490017	287	Actuator Viking S	pce	 3SE630205

Viking™ S mobile lift

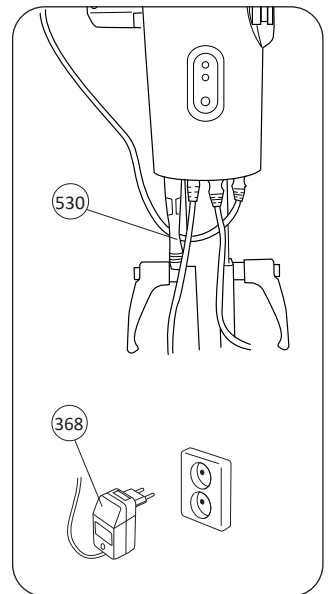
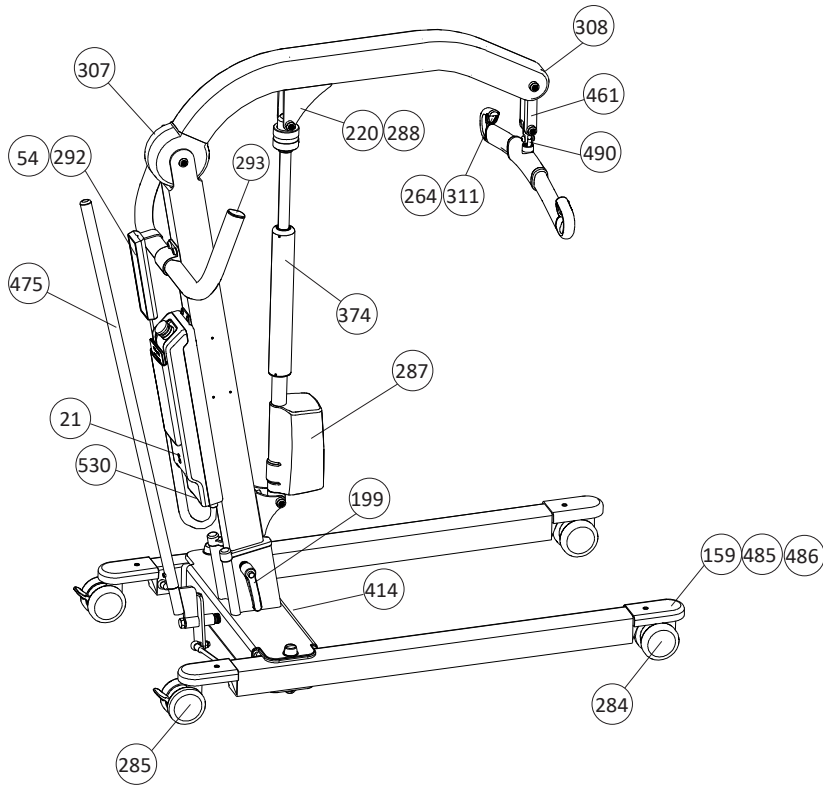
PROD. NO.	POS. NO.	NAME	UNIT	ASSEMBLY INSTR.
20490018	288	Emergency Lowering Device, Viking S, XS	pce	 3SE630206
20090058	292	Hook for Hand control 2,4,6	pce	 3SE630210
20090066	293	End Cover Handle/Arm Support grey, 4 pce	set	 3SE630271
20490022	300	Locking Handle Viking S	set	 3SE630219
31590014	311	Latch, composite <i>Fits Universal SlingBar from S/N 1299511</i>	pair	 3SE640242
20490026	321	Overhead Beam restoration set <i>From S/N 7300868</i>	set	 3SE640243
20490027	322	Lift arm Plastic cover <i>From S/N 7300868</i>	set	 3SE640244
21090025	368	Charger CH01	set	 3EN650232
20390017	374	Outer Tube LA32, 310 mm	pce	 3SE650237
21090036	414	Attachment, base adjustment	set	 3SE650266
20490050	475	Lever for base width adjustment From S/N <i>Viking S: 7301950</i> <i>Viking XS: 7401150</i>	pce	 7EN131801 Instruction Guide

Viking™ S mobile lift

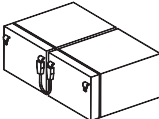
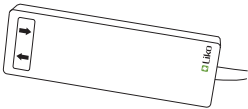
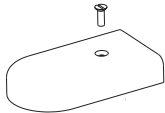
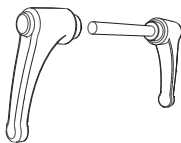
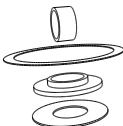
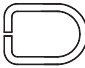
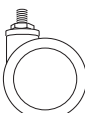

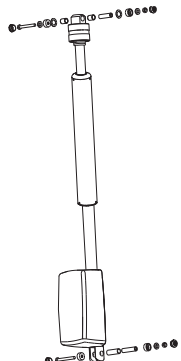
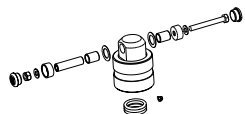

PROD. NO.	POS. NO.	NAME	UNIT	ASSEMBLY INSTR.
20190074	476	Flexlink Alu	set	 3SE650297
20090080	485	Fork locking attachment, 4 pair	set	 3SE690208
20090081	486	Wheel Fork	pair	 3SE690209
20490052	490	SlingBar Assembly set	set	 3SE690210
20190079	530	Charger Socket Connection	pce	 3SE690237
2107104 2107105 2107106		Cable Reel EU Cable Reel UK Cable Reel U / NZ	pce pce pce	 3NN160237
2040110SVFI 2040110NODK 2040110ENES 2040110DEFR 2040110FRNL 2040110ITPT		Quick Reference Guide Viking XS/S Swedish/Finnish Norwegian/Danish English/Spanish German/French French/Dutch Italian/Portuguese applies to: Viking S Prod. No. 2040006 from S/N 7301950	pce pce pce pce pce pce	 -

Spare Parts Positions Viking™ XS mobile lift


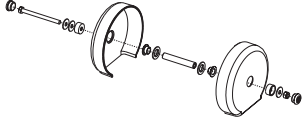
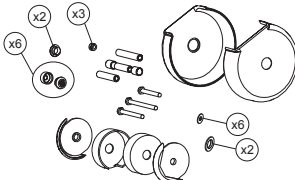

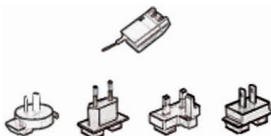
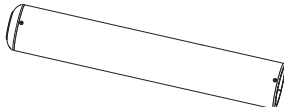
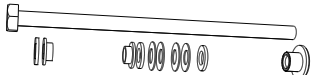
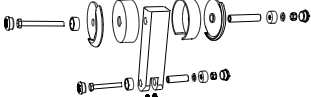
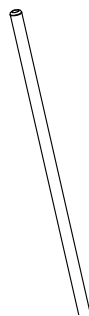
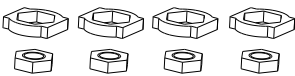
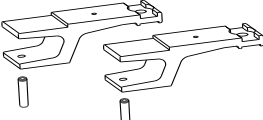
PROD. NO.	PRODUCT
2040007	Viking XS (Discontinued 2017)



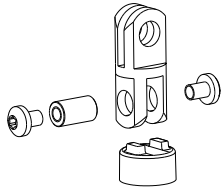
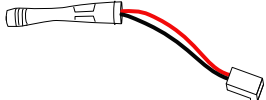
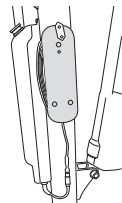

Viking™ XS mobile lift

PROD. NO.	POS. NO.	NAME	UNIT	ASSEMBLY INSTR.
20090025	21	Battery 2,9/12V	pair	 3SE380201
20190016	54	Hand control EM 2-Button	pce	 3SE630204
20390006	159	Fork Cover	pair	 183916
20490010	199	Locking Handle Viking XL	set	 3SE630244
20190051	220	Restoration set Emergency Lowering	set	 3SE630222
31590011	264	Latch Universal SlingBar	pair	 3SE630229
20490014	284	Front Wheel 75 mm Standard	pce	 182904
20490015	285	Rear Wheel 75 mm Standard	pce	 182903
20490017	287	Actuator Viking S	pce	 3SE630205
20490018	288	Emergency Lowering Device, Viking S, XS	pce	 3SE630206
20090058	292	Hook for Hand control 2,4,6	pce	 3SE630210








Viking™ XS mobile lift

PROD. NO.	POS. NO.	NAME	UNIT		ASSEMBLY INSTR.
20090066	293	End Cover Handle/Arm Support grey, 4 pce	set		3SE630271
20490023	307	Lift Arm Plastic Cover	set		3EN630250
20490024	308	Overhead Beam Restoration set	set		3EN630251
31590014	311	Latch, composite <i>Fits Universal SlingBar from S/N 1299511</i>	pair		3SE640242
21090025	368	Charger CH01	set		3EN650232
20390017	374	Outer Tube LA32, 310 mm	pce		3SE650237
21090036	414	Attachment, base adjustment	set		3SE650266
20490049	461	Flexlink	set		3SE650295
20490050	475	Lever for base width adjustment From S/N <i>Viking S: 7301950</i> <i>Viking XS: 7401150</i>	pce		7EN131801 Instruction Guide
20090080	485	Fork locking attachment, 4 pair	set		3SE690208
20090081	486	Wheel Fork	pair		3SE690209

Viking™ XS mobile lift

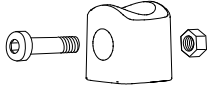

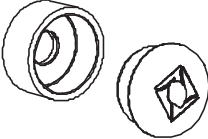








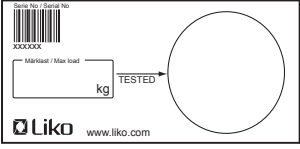

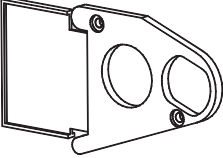
PROD. NO.	POS. NO.	NAME	UNIT	ASSEMBLY INSTR.
20490052	490	SlingBar Assembly set	set	 3SE690210
20190079	530	Charger Socket Connection	pce	 3SE690237
2107104 2107105 2107106		Cable Reel EU Cable Reel UK Cable Reel U / NZ	pce pce pce	 3NN160237
2040110SVFI 2040110NODK 2040110ENES 2040110DEFR 2040110FRNL 2040110ITPT		Quick Reference Guide Viking XS/S Swedish/Finnish Norwegian/Danish English/Spanish German/French French/Dutch Italian/Portuguese applies to: Viking XS Prod. No. 2040007 from S/N 7401150	pce pce pce pce pce pce	 -

Spare Parts, Slings

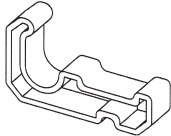
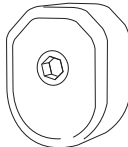
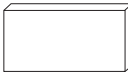
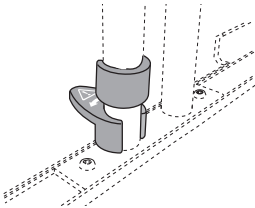


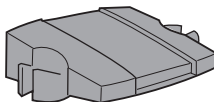
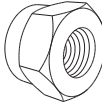






PROD. NO.	NAME	UNIT																																																																																																																					
31590016	Buckle SS40	pce																																																																																																																					
31590017	Light Weight Buckle	pce																																																																																																																					
31590018	Strap holder	pce																																																																																																																					
31590019	D-ring, plastic	pce																																																																																																																					
31590020	Lath 34,5 cm (13 in) Fits Liko HygieneVest Mod 55 and Liko Vest for Stand- ingShell Mod 67 in sizes XS, XXS.	pce																																																																																																																					
31590021	Lath 39 cm (15 in) Fits Liko HygieneVest Mod 55 and Liko Vest for Stand- ingShell Mod 67 in size S.	pce																																																																																																																					
31590006	Lath 43 cm (17 in) Fits Liko Original HighBack Sling Mod 200/210, Liko Soft Original HighBack Sling Mod 26, and Liko AmputeeSling Mod 75 in all sizes. Fits Liko HygieneVest Mod 55 and Liko Vest for StandingShell Mod 67 in sizes M, L, XL.	pce																																																																																																																					
31590023	Label Periodic Inspection, 10 pce	set	<p>900002571 rev 5 Periodic Inspection 80x80 2016</p> <p>Periodic Inspection</p> <table border="1"> <thead> <tr> <th rowspan="2">Year</th> <th colspan="12">Month</th> </tr> <tr> <th>1</th><th>2</th><th>3</th><th>4</th><th>5</th><th>6</th><th>7</th><th>8</th><th>9</th><th>10</th><th>11</th><th>12</th> </tr> </thead> <tbody> <tr><td>16</td><td></td><td></td><td></td><td></td><td></td><td></td><td></td><td></td><td></td><td></td><td></td><td></td></tr> <tr><td>17</td><td></td><td></td><td></td><td></td><td></td><td></td><td></td><td></td><td></td><td></td><td></td><td></td></tr> <tr><td>18</td><td></td><td></td><td></td><td></td><td></td><td></td><td></td><td></td><td></td><td></td><td></td><td></td></tr> <tr><td>19</td><td></td><td></td><td></td><td></td><td></td><td></td><td></td><td></td><td></td><td></td><td></td><td></td></tr> <tr><td>20</td><td></td><td></td><td></td><td></td><td></td><td></td><td></td><td></td><td></td><td></td><td></td><td></td></tr> <tr><td>21</td><td></td><td></td><td></td><td></td><td></td><td></td><td></td><td></td><td></td><td></td><td></td><td></td></tr> <tr><td>22</td><td></td><td></td><td></td><td></td><td></td><td></td><td></td><td></td><td></td><td></td><td></td><td></td></tr> </tbody> </table>	Year	Month												1	2	3	4	5	6	7	8	9	10	11	12	16													17													18													19													20													21													22												
Year	Month																																																																																																																						
	1	2	3	4	5	6	7	8	9	10	11	12																																																																																																											
16																																																																																																																							
17																																																																																																																							
18																																																																																																																							
19																																																																																																																							
20																																																																																																																							
21																																																																																																																							
22																																																																																																																							

△ Repairs and maintenance may only be performed using original Liko™ spare parts and only prescribed products according to assembly instruction to each spare part or this Spare Parts List.

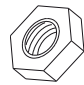
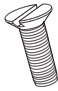

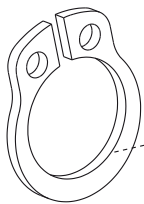
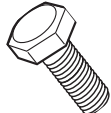


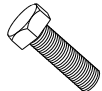
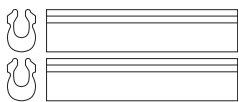

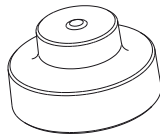
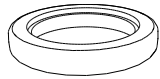
Spare Parts Accessories mobile lifts

PROD. NO.	NAME	UNIT		ASSEMBLY INSTR.
31590001	Assembly kit, LikoScale <i>required tools are included</i>	set	 x2	3SE650241
20090086	Brake pedal red, 2 pce	set		-
21090027	Cover hat and bottom, Liko, 10 pair	set		-
31090003	Cover Hat M10 Grey, 20 pce	set		-
31090004	Cover Hat M10 White, 20 pce	set		-
31090005	Cover Hat M8 Grey, 20 pce	set		-
31090006	Cover Hat M8 White, 20 pce	set		-
21090050	Cover hat M6, grey, 20 pce	set		-
21090005	Cover Hat Wheel M6 Grey, 10 pce	set		-
21090004	Cover Hat Wheel M8 Grey, 10 pce	set		-
20190067	Cover Hat M10 grey U102, 20 pce <i>Uno™ mobile lift, Sabina™ mobile lift</i>	set		-
31590013	Decal rail system max-load, 20 pce	set		-
21090012	Emergency guard, 10 pce <i>Golvo™ 7000/7007, Uno™ 100/102, Sabina™ II, LikoLight™, Viking™ XL/L/M mobile lifts</i>	set		3SE630201
20090067	End Cover Arm rest, grey <i>Golvo™ mobile lift, Viking™ M/L/XL mobile lift armrest</i>	set		3SE630272

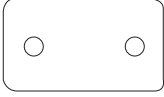


Spare Parts Accessories mobile lifts

PROD. NO.	NAME	UNIT		ASSEMBLY INSTR.
31590002	End Cover Liko Stretch Mod. 1800, 4 pce	set		-
31590004	End Protection to Side Rail	pce		-
31090007	Eraser Alu.	pce		-
31590003	Extra Catch on Stretcher	pce		7EN60201
31590012	Extra loop to Support Spring	pair		3SE640254
21090031	Fuse 10 A, 10 pce	set		-
2107100	Holder for charging cable	pce		7NN160252
20490061	Lock nut M12, 20 pck	set		-
21090002	Lock nut M10, 20 pce	set		-
21090003	Lock nut M8, 20 pce	set		-
21090046	Lock nut M8 nyloc A2, 20 pce	set		-
21090030	Lock nut M6, 20 pce	set		-
21090035	Lock nut M4, 20 pce	set		-
20190075	Lock nut M10, Domed, 20 pce	set		-

Spare Parts Accessories mobile lifts




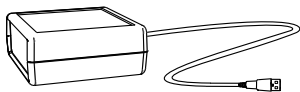




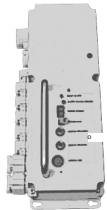

PROD. NO.	NAME	UNIT		ASSEMBLY INSTR.
21090044	Nut M12 low, 20 pce	set		-
21090034	Nylon screw M5x16, 20 pce <i>Fits fork cover; Golvo™, LikoLight™, RollOn™ and Viking™ mobile lifts.</i>	set		-
21090033	Plastic cover <i>Fits Universal SlingBar</i>	pair		3SE650249
21090043	Retaining ring SGA10, 10 pce	set	 Mount with sharp side out!	-
20490058	Screw M12x35, 20 pce	set		-
20490059	Screw M10x35 fzb, 20 pce	set		-
21090047	Screw K6S M5x8 Ezloc, 20 pce	set		-
21090029	Screw M6S M8X25 fzb, 20 pce	set		-
3166502 3166505	SideRails 200 SideRails 500 <i>Fits LikoStretch Mod. 1800.</i>	pair pair		-
20090083	Split Pin SP 3,2x20, 10 pce	set		-
21090045	Steering wheel Traverse Carrier, 10 pce	set		-
21090037	Stop ring, 10 pce <i>Fits Sling Bar Standard</i>	set		3SE650263

Spare Parts Accessories mobile lifts

PROD. NO.	NAME	UNIT	ASSEMBLY INSTR.	
31090002	Upright Support Distance, 10 pce	set		-
21090042	Washer PVC 10x16x2, 10 pce	set		-
21090032	Washer 12,5/20x0,42, 10 pce <i>Uno™, Sabina™, LikoLight™, Viking™ S/XS, Caroli™, Capella™ mobile lifts</i>	set		-


ServiceTools

Liko™ mobile lifts

PROD. NO.	NAME	UNIT	ASSEMBLY INSTR.
31090007	Eraser Alu.	pce	
21090028	Grease SHC460 30g <i>Golvo™ mobile lift, Likorall™ overhead lift</i>	pce	
20190048	Lacquer Top Coat for Improvement, Grey <i>Liko™ M220/M230, Sabina™, Uno™ mobile lifts</i>	pce	
21090024	Liko Service System <i>For programming the Control Unit CBL with service information. Record and store information on how the lift is used. Reset the LDS service interval after completed service/periodic inspection.</i> <i>Requires a computer and internet connection.</i>	set	
21090041	PTFE-Grease 150g	pce	
31590015	Seal colour Emergency Lowering, (50ml)	pce	
21090039	Service label Liko™ - Sweden, 100 pce	set	
21090040	Service label Sodexo - Sweden, 100 pce	set	
21090013 21090014 21090015	Test box for actuator LA 31/34 EU Test box for actuator LA 31/34 UK Test box for actuator LA 31/34 US	pce pce pce	
20090047	Test Equipment for Hand control	pce	



www.hillrom.com

 Liko AB
Nedre vägen 100
SE-975 92 Luleå, Sweden

Liko AB is a subsidiary of Hill-Rom Holdings inc.

Enhancing outcomes for
patients and their caregivers.

