

Z:\SW-Tek\Pelikaan lijn\4011 Electrisc\4011\

Signed	PS	Date	08-05-2006
Revised	2 /PS	Date	19-12-2008
Revised	3 /KA	Date	22-01-2015
Copyright reserved by law		Unit measurement: mm	


Overview parts Amfora

Lopital Nederland bv

Type nr: 62004011/4016

Amfora 62004011 / 62004016

Part nr.	QTY.	Art.nr.	Part
1	1	70401102	Carrier electric
2	1	70401004 70401501	Frame L=160 Frame L=190
3	2	67904120 67904125	Side rail complete L=160 Side rail complete L=190
4	1	67908070 67908068	Drip tray L=160 + Hose Drip tray L=190 + Hose
5	1	70410003 70410501	Inlay frame backrest L=160 Inlay frame backrest L=190
6	1	70410001	Adjusting backrest
7	1	70410004	Clamp high
8	1	70401012 70401512	Lying surface 160 Lying surface 190
9	2	70410002	Tube backrest adjusting
10	4	70519928	Castor Ø 125 mm braked
11	1	70401101	Actuator
12	1	70210510	Manual control
13	1	70404001	Clamp low
14	1	70210516	Charger
15	1	70210511	Mounting plate
16	1	70210518	Batterypack
17	1	70210515	Control box
18	4	70401002	Spacer
19	1	70210512	Mountingplate wall charger
20	8		Tube cap
21	2		Locknut
22	4		Slide bearing
23	2		Hex bolt
24	4		Slide bearing
25	3		Locknut
26	3		Round head screw
27	8		Hex bolt
28	3		Cylinder head screw
29	8		Locknut
30	24		Washer
31	4		Hex bolt
32	2		Hex nut
33	1		Countersunk screw
34	10		Washer
35	4		Lock nut
36	4		Nut cap
37	2		Wingbolt
38	2		Tube cap

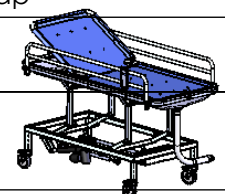

Signed	PS	Date	08-05-2006
Revised	2 /PS	Date	19-12-2008
Revised	3 /KA	Date	22-01-2015

Overview parts Amfora

Copyright reserved
by law

Unit measure-
ment: mm

Lopital Nederland bv

Type nr: 62004011/4016