

Z:\SW-Tek\Peilkaan lijn\4040\

Signed	JvdL	Date	01-06-2005
Revised	<b>5</b> /PS	Date	19-12-2008
Revised	<b>6</b> /KA	Date	23-01-2015

Copyright reserved by law	Unit of measurement: mm
---------------------------	-------------------------


# Overview parts Amfora

Lopital Nederland bv

Type nr: 63004040/4045

## Amfora 63004040 / 63004045

Part nr.	QTY.	Art.nr.	Part
1	1	70403001 70403501	Frame L=160 Frame L=190
2	2	70403004	Leg
3	1	70410001	Adjusting backrest
4	1	70410003 70410501	Inlay frame backrest L=160 Inlay frame backrest L=190
5	1	67908070 67908068	Drip tray L=160 + Hose Drip tray L=190 + Hose
6	1	70404001	Clamp low
7	1	67904120 67904125	Side rail complete L=160 Side rail complete L=190
8	1	70410002	Axis backrest
9	2	70412502	Hinge
10	1	70410004	Clamp high
11	1	70401012	Lying surface 160
12	2	70410002	Tube backrest adjusting
13	2	70407002	Wallmounting L=95
14	1	70403005	Locking axle
15	1	70407001	Locking support long
16	2		Buffer cap
17	2		Wingbolt
18	6		Lock nut
19	4		Washer
20	2		Hex bolt
21	8		Nut cap
22	4		Nut cap
23	2		Hex bolt
24	5		Washer
25	2		Locknut
26	2		Ball knob
27	10		Washer
28	2		Tube cap

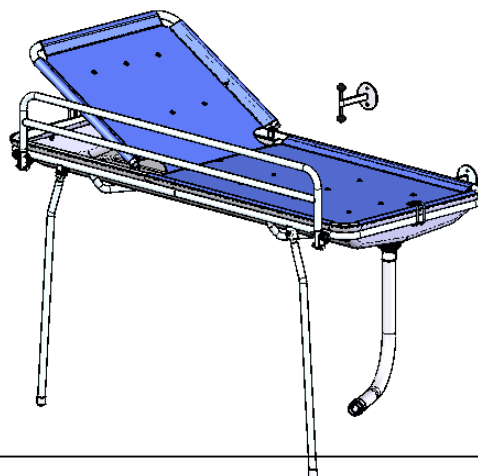

Signed	JvdL	Date	01-06-2005
Revised	5 /PS	Date	19-12-2008
Revised	6 /KA	Date	23-01-2015

## Overview parts Amfora

Copyright reserved  
by law

Unit of measure-  
ment: mm

Lopital Nederland bv

Type nr: 63004040/4045