


From Serial number 5500.20.004

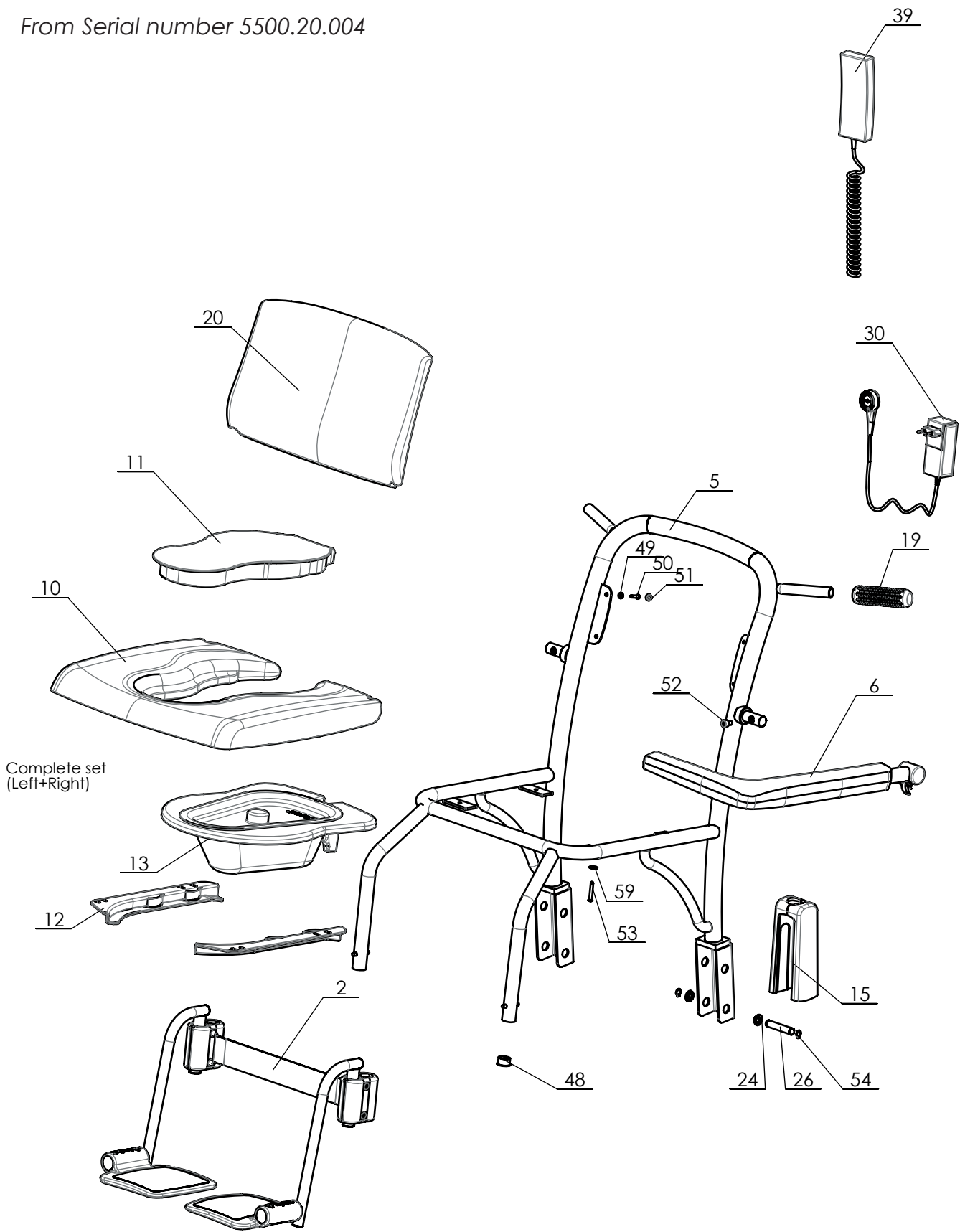

Complete set
(Left+Right)

P:\SW-Tek\Elexo 5500\

Drawn by: PS	Date	22-03-2010
Revised: 10 /KA	Date	29-09-2023
Revised: 9 /KA	Date	08-01-2020

Copyright reserved
by law


Unit of measure-
ment: mm

Overview parts Elexo

Lopital Nederland bv

Type no: 51005500

Set (left + Right)

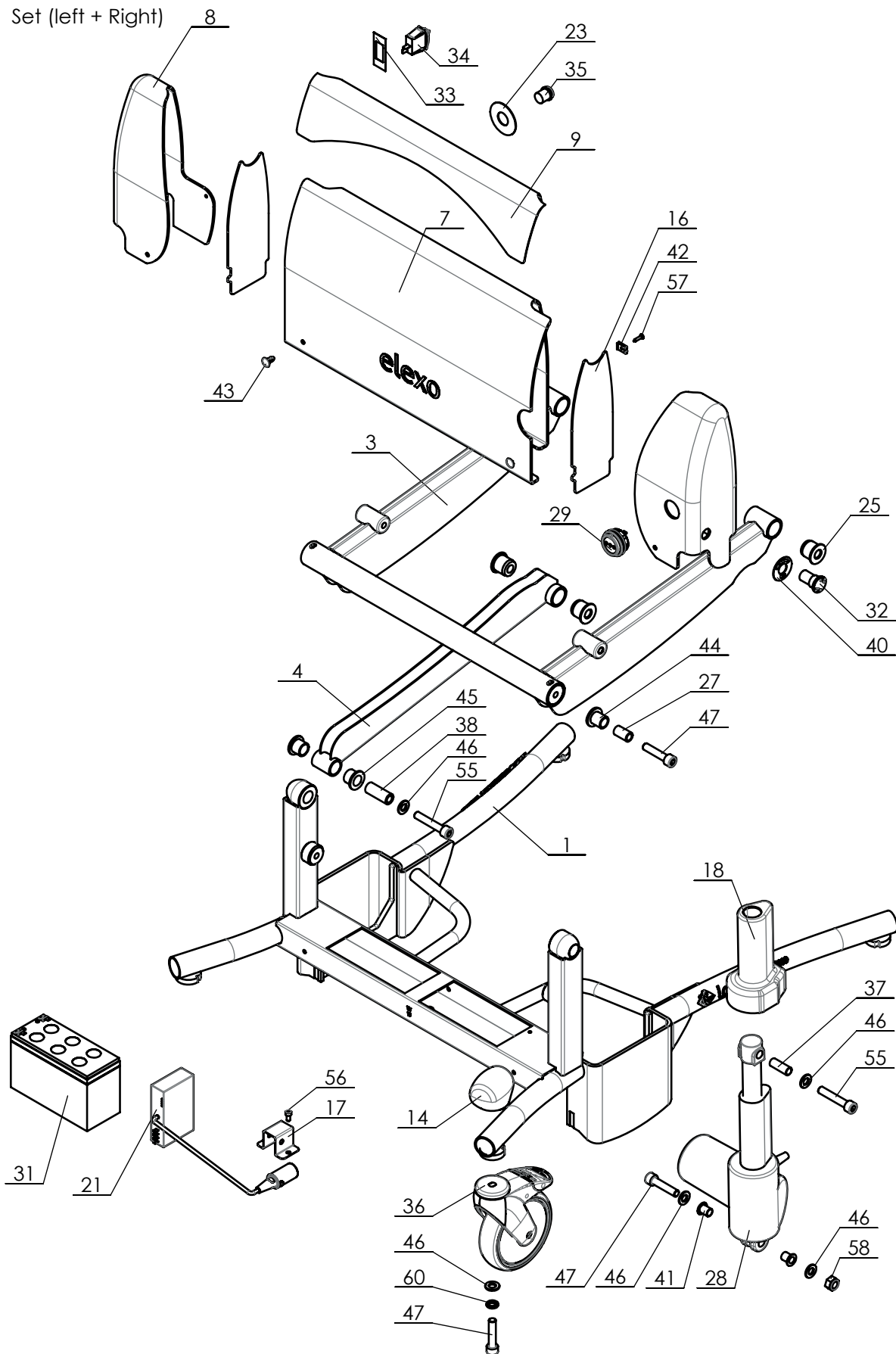

P:\SW-Tek\Elexo 5500\

Drawn by: RH	Date: 02-12-2014
Revised: 10 / KA	Date: 29-09-2023
Revised: 09 / KA	Date: 08-01-2020

Copyright reserved by law	Unit of measurement: mm
---------------------------	-------------------------

Overview parts Elexo base frame

Lopital Nederland bv

Type no: 51005500

Elexo 51005500

Part nr.	QTY.	Art.nr.	Part
1	1	70550001	Frame
2	1	69605199	Footrest complete
3	1	70550003	Main lever
4	2	70550004	Control lever
5	1	70519905	Seat frame
6	1	68405660	Front locking armrest
7	1	70550010	Cover for battery (2 pieces)
8	1	70550011	Battery Cover sides (set)
9	1	70550017	Sticker motorcover
10	1	70560046	Seat
11	1	70560045	Lid for seat
12	1	70510050	Guide for bedpan set
13	1	69501104	Bedpan + lid
14	4	70560066	Protective cover wheel
15	2	70519936	Hinge cover
16	2	70550019	Seperation plate
17	1	70550021	Bracket coupling
18	2	70550020	Cover for TA1
19	2	70560049	Handgrip
20	1	70550005	Back rest
21	1	70550013	Uni-lop Control
22	1	70530028	Battery cable with fuse
23	1	70550043	Sticker ELD
24	8	70560040	Bush bearing rear
25	8	70560041	Bush
26	4	70560042	Hinge
27	2	70519923	Bush
28	2	70550006	Actuator TA1
29	1	70550018	Chassispart charger
30	1	70550012	Charger incl. plug
31	2	70560004	Battery Set
32	1	70530030	Emergency stop
33	1	70550044	Sticker on/off
34	1	70550045	Switch On/Off
35	1	70550014	Emergency lowering button
36	4	70519928	Castor Ø 125 mm braked
37	2	70550009	Hinge bush
38	2	70550008	Bush
39	1	70550007	Manual control
40	1	70550046	Sticker E-Stop
41	4		Flanged Slide bearing 10x12x15

Drawn by: PS	Date 22-03-2010
Revised: 10 /KA	Date 29-09-2023
Revised: 9 /KA	Date 08-01-2020

Copyright reserved
by law

Unit of measure-
ment: mm

Overview parts Elexo

Lopital Nederland bv

Type no: 51005500

Elexo 51005500

Part nr.	QTY.	Art.nr.	Part
42	2		Snap nut M4
43	3		Tree clip
44	2		Flanged Slide bearing 14x18x21
45	4		Flanged Slide bearing 16x20x17
46	13		Washer M10
47	8		Cylinder head screw M10 x 45
48	2		Tube cap Ø25
49	4		Capring M4
50	4		Chipboard screw 4,5x25
51	4		Screw cap M4
52	2		Cylinder head screw M8x12 Dacromet
53	4		Chipboard screw 5x35
54	8		Retaining ring 12
55	4		Cylinder head screw M10x55
56	2		Cylinder head screw M5x10
57	2		Chipboard screw 3,9x16
58	2		Locknut M10
59	4		Washer M5 Ø15
60	4		Nord-Lock M10



P:\SW-Tek\Elexo 5500\

Drawn by: PS	Date: 22-03-2010
Revised: 10 /KA	Date: 29-09-2023
Revised: 9 /KA	Date: 08-01-2020

Overview parts Elexo

Copyright reserved
by law

Unit of measure-
ment: mm

Lopital Nederland bv

Type no: 51005500