

C:\Backup 16-03-2020\SW-Tek\Marina 2300\2300 82x204\

Signed:	ES	Date:	02-2011
Revised:	8 / KA	Date:	24-03-2020
Revised:	- /	Date:	-


## Overview parts Marina Top frame

Copyright reserved by law

Unit of measurement: mm

Lopital Nederland bv

Type nr: 61002300

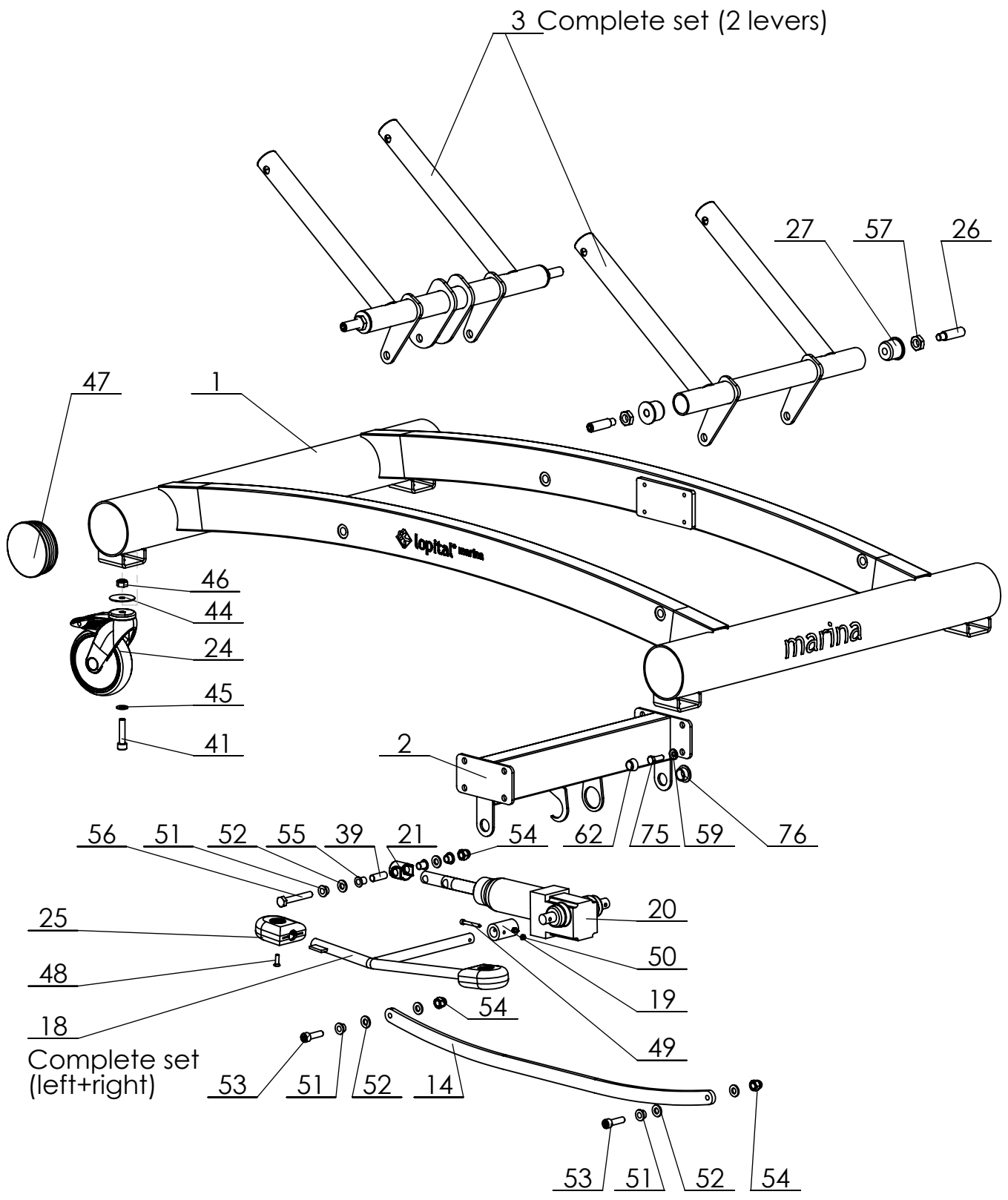

C:\Backup 16-03-2020\SW-Tek\Marina 2300\2300 82x204\

Signed: ES	Date: 02-2011
Revised: 8 / KA	Date: 24-03-2020
Revised: - /	Date: -

# Overview parts Marina Base frame

Copyright reserved by law	Unit of measurement: mm	Lopital Nederland bv	Type nr: 61002300
---------------------------	-------------------------	----------------------	-------------------

**Marina 61002300**

Part nr.	QTY.	Art.nr.	Part
1	1	70230001	Chassis
2	1	70230002	Pump frame
3	1	70230003	Lever set
4	1	70230006	Subframe
5	1	see table last page	Frame
6	2	70230013	Axis holder
7	1	70230014	Tilt lever right
8	1	70230015	Tilt lever left
9	1	70230016	Axis long
10	1	70230017	Axis short
11	1	70230018	Bush
12	2	see table last page	Side rail
13	8	70220023	Clamping plates
14	2	70230025	Pull stroke
15	1	see table last page	Lying plate
16	2	70230012	U-bracket
17	1	70210040	Gas spring
18	1	70230004	Pedal pump set
19	2	70210033	Coupling bush
20	1	70519913	Pump (hydraulic)
21	1	70230019	Pump attachment
22	2	70210041	Leaf spring incl. cover
23	2	70210039	Lock siderail
24	4	see table last page	Castor Ø 125 mm braked
25	4	70519920	Pedal cap
26	4	70307001	Custom screw
27	4	70230008	Lever bearing
28	4	70220040	Plug for buffer role
29	4	70220041	Buffer roll
30	24	70220027	Threaded bush
31	2	70210053	Cover gaspring handle
32	1	see table last page	Mattress
33	1	70210048	Pillow
34	1	70210056	Clamping ring
35	1	70210043	Clamping disk
36	1	67902000	Hose (complete)
37	1	70210055	Spout
38	1	70220028	Hose clip

Signed: ES	Date: 02-2011
Revised: 8 / KA	Date: 24-03-2020
Revised: - /	Date: -

## Overview parts Marina

Copyright reserved  
by law

Unit of measure-  
ment: mm

Lopital Nederland bv

Type nr: 61002300

**Marina 61002300**

Part nr.	QTY.	Art.nr.	Part
39	1	70550009	Hinge bush
40	1	70210058	Flat seal washer
41	4	70519935	Cylinder head screw M10 x 45
42	1		Reservoir coupling
43	4		Flanged Slide bearing 18x22x25
44	4		Ring 45x10.5x1.5
45	4		Washer M10
46	4		Locknut M10
47	4		Tube cap Ø100
48	4		Countersunk cross slotted screw M6x20
49	4		Cylinder headscrew
50	5		Locknut M5
51	8		Flanged slide bearing 10x14x10
52	10		Washer PA M10
53	4		Cylinder head screw M10x35
54	5		Capnut M10
55	2		Flanged Slide bearing 10x12x15
56	1		Hex bolt M10x70
57	4		Low Hex nut M16
58	16		Slide bearing 8x14x10
59	16		Cap ring M8
60	4		Hex bolt M8 x 100
61	8		Locknut M8
62	17		Screw cap M8
63	8		Tube cap Ø40
64	2		Cylinder head screw M5x25
65	4		Washer M5
66	8		Washer M6
67	4		Hexagonal head bolt M6 x 45
68	4		Locknut M6
69	2		Buffer 20x18
70	6		Washer M8
71	6		Tube cap 50x25
72	4		Tube cap Ø30
73	4		Hex bolt M8 x 55
74	48		Hexagon socket button head screw M5x8
75	8		Hex bolt M8x25
76	2		Flanged Slide bearing 18x22x12

Signed: ES	Date: 02-2011
Revised: 8 / KA	Date: 24-03-2020
Revised: - /	Date: -

## Overview parts Marina

Copyright reserved  
by law

Unit of measure-  
ment: mm

Lopital Nederland bv

Type nr: 61002300

The parts below differ per Marina variant.

	<b>Article number part</b>	<b>Marina variant</b>
<b>Siderail</b>	80220004 9970221604 9970232002	2300 2305 2310 2350 2355 2360 2330 2345 2380 2385 2320 2370
<b>Lying plate</b>	80220014 80230503 9975233003 9975236001 9970232003	2300 2310 2350 2360 2305 2355 2330 2380 2345 2385 2320 2370
<b>Mattress</b>	75220016 75230504 75221601 75234503 75232006	2300 2310 2350 2360 2305 2355 2330 2380 2345 2385 2320 2370
<b>Frame</b>	9970230007 9970230502 9970221602 9970234502 9970232001	2300 2310 2350 2360 2305 2355 2330 2380 2345 2385 2320 2370
<b>Castor</b>	70519928 70201002 (set p.4)	2300 2305 2320 2330 2345 2350 2355 2370 2380 2385 2310 2360